

**LECKEY**<sup>®</sup>

Horizon Stander



# LECKEY<sup>o</sup>

Stronger together.

We work with individuals, therapists and carers to design products with both a clinical and an emotional focus. Using the latest research and clinical understanding, we create practical solutions which are easily integrated into family life, because for us, life is about going, enjoying, participating and doing.



24 hour  
postural care for  
babies, kids & adults.  
Sleeping, Sitting,  
Standing, Walking,  
Moving, Bathing,  
Toileting.



---

The 3 in 1 Horizon stander is a robust, durable and adjustable supine, upright and prone stander. Available in 2 sizes covering a large growth range from 4 – 18 years of age.

---

The innovative design enables easy and safe transfer of users using either mobile or tracking hoists.

---

The unique positioning achieved by our pelvic belt and hip laterals enables de-rotation of the hips; alignment of the pelvis and centre of gravity over the base of support; accommodation or correction of flexion at the hips and knees.

---

Ideally suited for more complex users or for young adults who have not benefited from standing therapy for some time, the individually adjustable chest, hip, knee and footplates provide unsurpassed levels of adjustability and proximal positioning.

---

With the use of the non-slip adjustment knobs the product can be quickly and easily set up and re-adjusted for multiple users in a classroom, therapy room or home environment.





Supine set up

Head support

Adjustable chest and lateral support

Adjustable activity tray

Pelvic support belt or pads

Pelvic band

Adjustable knee support

Independently adjustable footplates and sandals



Flexible hip guides




Independent footplate positioning




Simple and easy to use hand adjustments




A range of head supports is available.

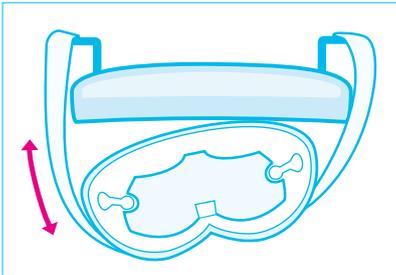


Rotation

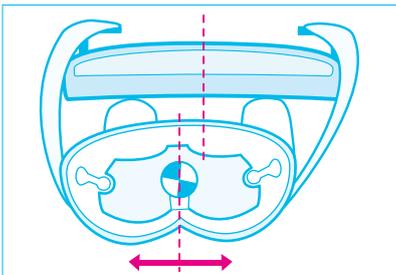

Balance

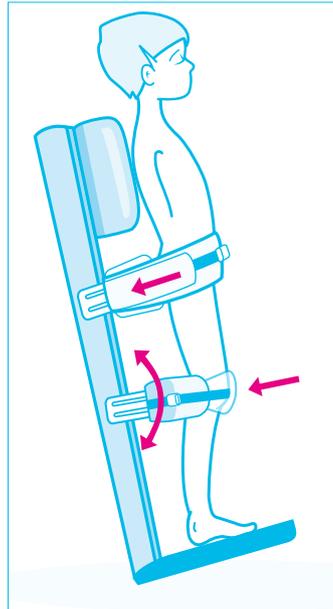

Supine Flexion

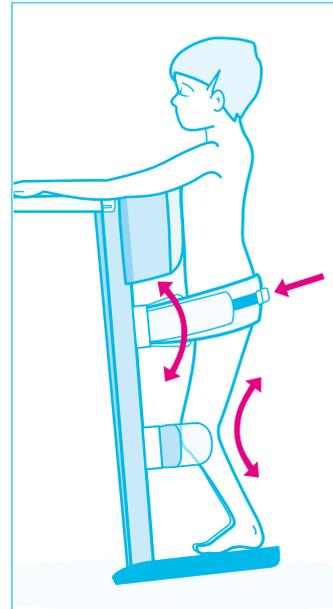

Prone Flexion



Powered angle adjustment



Prone set up

The Horizon Stander is available in two sizes.

**Horizon Supine Stander - sizes 1 and 2**  
The Horizon Supine Stander comes complete with activity tray, knee supports, laterals, head support and your choice of pelvic band and sandal size specified at time of order. The Horizon stander comes as standard in Mondeo Blue Fabric.

**Horizon Pronestander - sizes 1 and 2**  
Horizon Pronestander comes complete with activity tray, knee supports, laterals, and your choice of pelvic band and sandal size specified at time of order. The Horizon stander comes as standard in Mondeo Blue Fabric.

## Product Sizing

Size	1		2	
Age	4 - 10		8 - 18	
Codes	LHZ/1		LHZ/2	
	mm	inches	mm	inches
<b>Horizon Stander</b>				
<b>Length</b>	1135	44.7	1280	50.4
<b>Width</b>	580	22.8	640	25.2
<b>Height in Horizontal Position</b>	780	30.7	805	31.7
<b>Weight of Product</b>	41kg	90lbs	47kg	103.6lbs
<b>Max User Weight</b>	50kg	110lbs	80kg	176lbs
<b>User Standing Height</b>				
min	1000	39.4	1400	55.1
max	1400	55.1	1800	70.9
<b>Shoulder Height</b> (footplate to top of pad)				
min	820	32.3	1000	39.4
max	1150	45.3	1450	57
<b>Hip Pad Height</b> (footplate to top of pad)				
min	450	17.7	750	29.5
max	750	29.5	960	37.4
<b>Knee Height</b> (footplate to middle of pad)				
min	250	9.8	400	15.7
max	450	17.7	550	21.6
<b>Width Between Chest Laterals</b>				
min	200	7.9	240	9.4
max	300	11.8	380	14.9
<b>Footplate Angle Adjustment</b> +/-	10°	10°	10°	10°
<b>Tray Sizes</b>	420 x 440	16.5 x 17.3	520 x 590	20.5 x 23.2
<b>Angle Adjustment</b>	0-90° prone 0-90° supine		0-90° prone 0-90° supine	
<b>Tray Angle Adjustment Prone Version</b>	0° to 40°	0° to 40°	0° to 40°	0° to 40°
<b>Tray Angle Adjustment Supine Version</b>	-20° to +30°	-20° to +30°	-20° to +30°	-20° to +30°

## Leckey Servicing

Our FREE service and support includes:  
Free product training  
Free product set up  
Free product assessment  
Free product re-assessment  
Free repair within 2 year warranty

For service policies on all products outside warranty, please contact Leckey's Customer Service department. James Leckey Design Ltd as manufacturer with sole responsibility declares that all products conform to 93/42/EEC guidelines and EN12182 technical aids for disabled persons general requirements and test methods. Order forms and spare parts lists to extend the service life of the product and allow reissue are available on request or online at [www.leckey.com](http://www.leckey.com).

## Accessories



LHZ/#/112 Head lateral supports



LHZ/#/110 Hip laterals



CAS/1/109 to fit LHZ/1  
CAS/2/109 to fit LHZ/2  
Padded tray insert



LHZ/#/107 Grab rail



LHZ/#/108 Grab posts



Whitmyer head support

## LHZ/#/138 Conversion kit for **supine** standing



Adjustable activity tray  
and pelvic band



Multi-adjustable head support

## LHZ/#/139 Conversion kit for **prone** standing



Adjustable tray




Pelvic band for prone standing


## Fabric Options



Blue

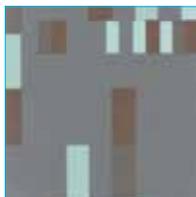

Charcoal

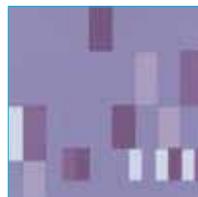

Purple



# LECKEY<sup>o</sup>



Established in 1983, Leckey is a globally recognised pioneer in the research and development of products that help adults and children with disabilities to go, do, enjoy and participate in everyday activities throughout the day and night.

We take a highly clinical approach to product design and development. Through in-depth clinical research studies with leading universities, and extensive trials with occupational therapists, physiotherapists, users and their families, we continue to develop posturally supportive, family friendly products for all day care, at every stage of life.

Through early intervention, childhood and adulthood Leckey's experienced team of designers, therapists and bioengineers work together to develop products that meet the clinical needs of the healthcare professionals and the social needs of the user.

To achieve this, we work with the healthcare professionals, the individuals and carers who use our products everyday. With their help, we create the dependable, durable, proven and high performance products that we are known for worldwide.



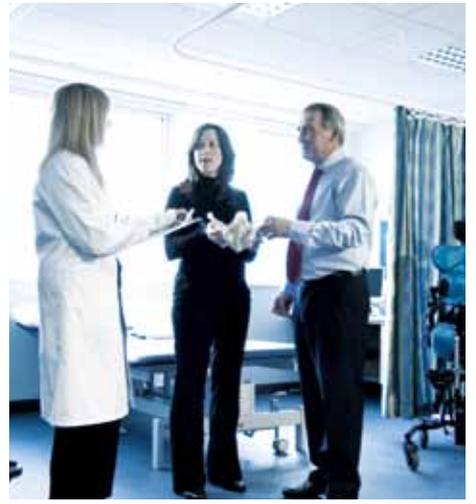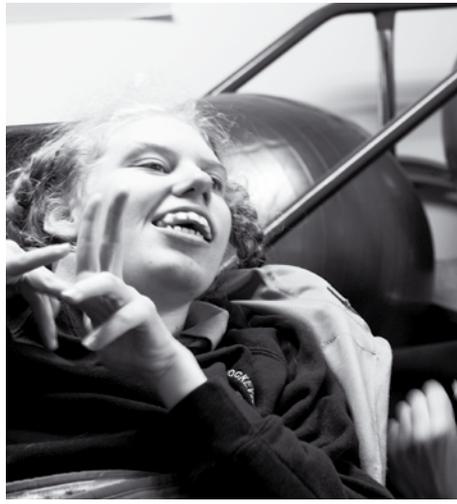

24 hour postural  
care for babies,  
kids & adults.  
Sleeping, Sitting,  
Standing, Walking,  
Moving, Bathing,  
Toileting.