AdjustmentLoop

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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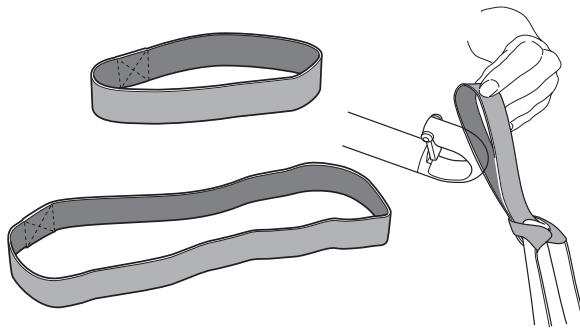
REF

49182006

49182007

SWL: 300 kg/ 660 lbs

Table of contents

English	4
Svenska	6
Norsk	8
Dansk	10
Deutsch	12
Nederlands	14
Italiano	16
Size	18
Symbols	18

**EN Always read the manual**

Keep the manual where it is easily accessible for users of the product.
Do not leave the patient unattended during a transfer situation.

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AdjustmentLoop is an extension strap, mainly used to extend the lifting sling's lift straps to the desired length; e.g., to enable a more recumbent position or a more upright seated position, or to achieve better balance, for example, for users with hemiplegia.

AdjustmentLoop is available in two lengths, both for max 300 kg/660 lbs. AdjustmentLoop, 18 cm/7.09" provides an extension of about 12 cm/4.72", while AdjustmentLoop, 28 cm/11.02" provides an extension of about 22 cm/8.66".

AdjustmentLoop can be used together with the following SystemRoMedic® lifting slings

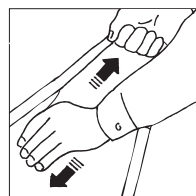
- All lifting slings for seated lifting and with divided leg supports
- All lifting slings for seated lifting and with undivided leg supports, except BariSling
- All variants of StretcherSling and SafeHandlingSheet for users weighing max 300 kg/660 lbs.
- StandingVest and WalkingVest
- LimbSling

Check Safety

Visual Inspection and stability test

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

Always read the user manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

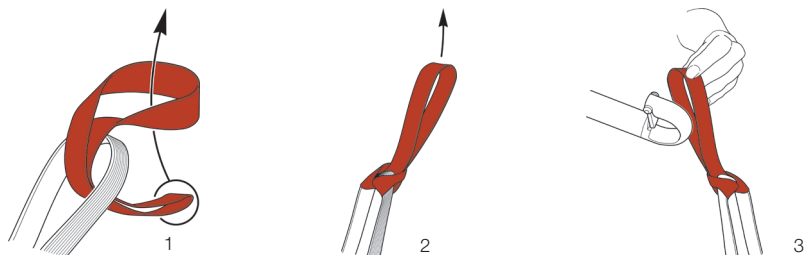
Important information

Do not leave the user unattended during a transfer situation.

To prevent discomfort and the risk of the user sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.


The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

Instructions for use of the AdjustmentLoop

1. Knot the AdjustmentLoop around the strap loop.
2. Tighten it secured around the strap loop.
3. Hang the loops on the slingbar. Start raising the lift, and then stop to ensure that everything is as it should be before lifting to the desired position.

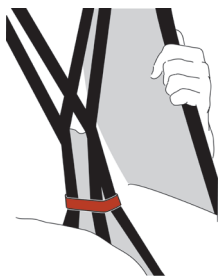Alternative placement of the AdjustmentLoop

The AdjustmentLoop can also be used on the leg sections to reduce the size of the opening and to prevent the user from sliding out of the sling.

Material

Polyester

Care of the product

Read the product label.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Size

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com