BariBar

SystemRoMedic™

Instructions for use – English

Bruksanvisning – Svenska

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
REF

70200072

Max User Weight: 454 kg / 1000 lbs

Table of contents

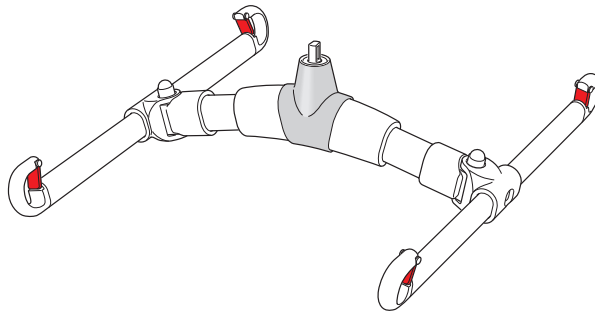English	3
Svenska	6
Norsk	9
Danske	12
Suomi	15
Deutsch	18
Netherlands	21
Français	24
Italiano	27
Espanol	30
Symbols	33

BariBar is a four-point aluminium/steel sling bar which is used, in combination with a correctly fitted lifting sling and a ceiling lift unit approved by Direct Healthcare Group, for lifting of users weighing up to 454 kg/1000 lbs.

BariBar opens up the lifting sling to provide more space and a more reclined seated position, which is suitable for larger and heavier and/or pain-sensitive users. BariBar is very easy to use. The built-in adjustment function automatically adapts the sling bar to each user for optimal comfort. Reliable safety latches prevent the lifting sling straps from creeping out of the sling bar.

Check product function

Visual inspection

Check product function regularly. Ensure that the material is free from damage. If damage is apparent, the product must be discarded.

Always read the manual

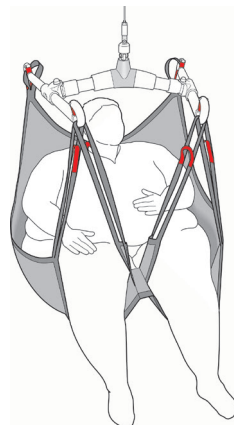
Always read the manuals for all assistive devices used during the transfer.

Keep the manual where it is easily accessible for users of the product.

Never leave a user unattended during a transfer.

Product description BariBar

BariBar is made of aluminium and steel. The lift hooks are equipped with safety latches. BariBar must be used in combination with ceiling lift units that are recommended by Direct Healthcare Group.

All four hooks must always be used.

Operation

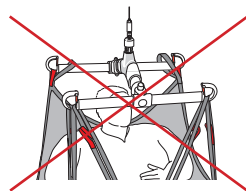
When moving the BariBar without any load applied, hold the side bar with one or both hands to prevent it from swinging.

When load is applied, use both the sling bar and the lifting sling to manoeuvre.

Lifting with BariBar

1. Use an approved lifting sling.
2. Place the BariBar with the main bar directly over the user and with the hooks marked with head symbols pointing towards the users back and head.
3. First, connect the lifting sling's back loops.
4. Then, connect the leg support loops.
5. Apply tension and ensure that everything is correct before starting to lift. If the user is uncomfortable, stop lifting.

 Head

SWL

The maximum load for BariBar is 454 kg / 1000 lbs.

Different products in the complete lift system (lift unit, sling bar, scales and other accessories) may be rated for different max. loads. The lowest rated max. load always determines the max. load for the entire system. Always check the max. load for the lift unit and lifting accessories before use, and contact your dealer when in doubt.

Cleaning

If necessary, clean with warm water or a soap solution. Do not use cleaning agents containing phenol or chlorine, as this could damage the aluminium and the plastic materials. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

BariBarWrap

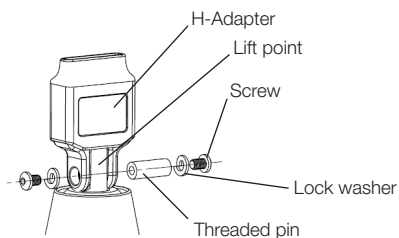
BariBar must always be fitted with BariBarWrap, a wipeable protection cover for secure handling of the sling bar.

Installing BariBar


Lower the lift strap and loosen the two screws on either end of the threaded pin. Insert the BariBar's lift point into the H-adapter and replace the threaded pin. Replace the two screws with the lock washers. Use two 4 mm Allen keys to tighten the screws.

Installation of BariBar on an overhead lift unit



- 1 x 90002955 (Threaded pin)
- 2 x 90001656 (Lock washer 6.1x11.8x1.6)
- 2 x 90001621 (Screw K6S M6x8)
- 2 x Allen key 4mm


RiseAtlas/RiseBasic

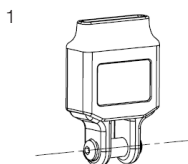2

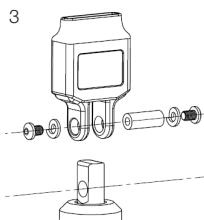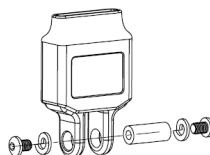4

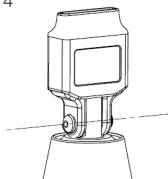Technical information

Weight: 8.6 kg / 18.9 lbs

Exterior dimensions: 197 x 682 x 609 mm, 7.7 x 26.8 x 23.9 inches

Lift dimensions, c-c lift loops: 600 x 546 mm, 23.6 x 21.4 inches

Colour: RAL 9006

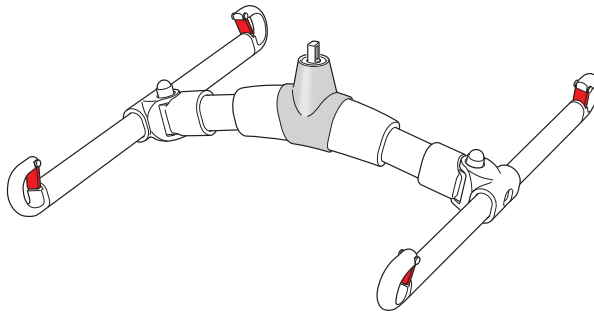
Material: Steel, aluminium, plastic

SWL: 454 kg, 1000 lbs

Angular displacement, side bars: 45°

Min. distance lift hooks, c-c lift loops: 366 mm / 14.4 inches

Max. distance lift hooks, c-c lift loops: 720 mm / 28.3 inches

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.