# BariSling

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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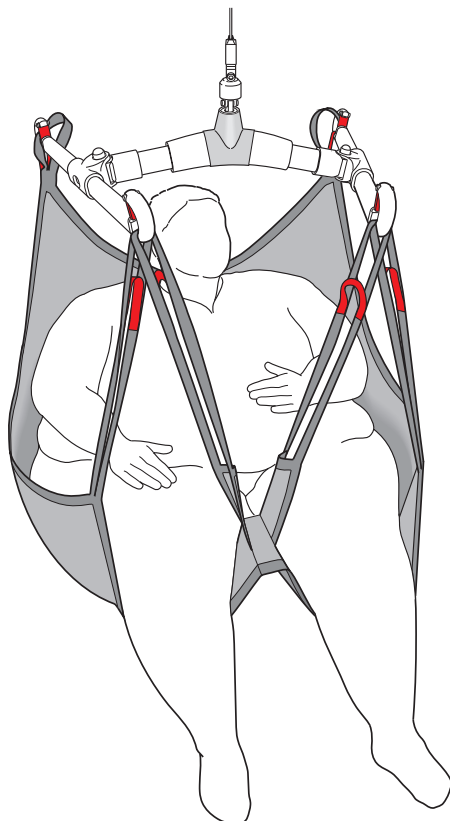
**REF**

47001010

47001011

**SWL: 570 kg/ 1257 lbs**

## Table of contents

English .....	3
Svenska .....	6
Norsk .....	9
Suomi .....	12
Deutsch .....	15
Nederlands .....	18
Français .....	21
Español .....	24
Symbols .....	27

BariSling has been designed for lifting persons weighing 200 kg/440lbs or more. It can be used in combination with mobile or stationary (overhead) lifts, with a four-point slingbar or with a BariTurn. BariSling has six suspension points. It has two slingbar loops at the back and four below, under the legs, to distribute pressure comfortably and safely.

## Functional inspection

### Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

## Always read the manual

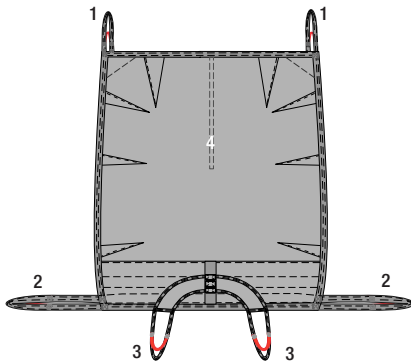
Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.


## BariSling product description

BariSling is designed as a full sling without separate leg supports. It has six slingbar loops to distribute pressure comfortably and safely: two at the back, two below on the outside of the legs, and two between the thighs.

### BariSling consists of:

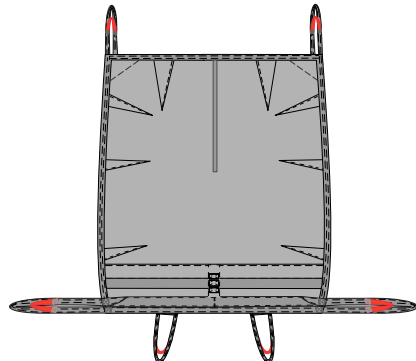1. Rear slingbar loops
2. Front slingbar loops
3. Leg harness with slingbar loops
4. Back and seat section

## Using BariSling

---

### **Application of BariSling on the patient, horizontal position in bed:**

Ideally, the patient should be able to turn him/herself. If this is not possible, we recommend the use of assistive devices for turning, such as SafeHandlingSheet, TopSheet or other approved assistive devices for turning.

Place the sling so that the midsection is aligned with the patient's spine and the lower edge is about 5-10 cm above the hollow of the knee. Place the edge of the sling as far in under the patient as possible.

Turn the patient back, so that the sling can be pulled forward and adjusted. The leg supports are then drawn up between the knees. Check to ensure that the sling is evenly distributed on both sides.

If possible, raise the head of the bed before lifting.

### **Application of BariSling on the patient, seated position at the bedside:**

The patient leans to one side of the bed and the BariSling is placed centred under the seat, as far back under the patient as possible, with the mid-section under the crotch.

Place the sling so that the mid-section is aligned with the patient's spine and the lower edge is about 5-10 cm above the hollow of the knee. Place the edge of the sling as far in under the patient as possible.

When the BariSling is correctly positioned, the patient sits up again and turns to the other side, so that the sling can be pulled forward and adjusted. The leg supports are then drawn up between the knees. Check to ensure that the sling is evenly distributed on both sides.

### **Application on a four-point slingbar:**

When lifting with a mobile or overhead lift unit, the use of a four-point slingbar is recommended. First, apply the rear loops to the wider side of the slingbar, and then the front loops to the narrower side.

### **Application on BariTurn:**

BariTurn is a turning plate that can be used in combination with two lift units to assist lifting while turning.

First, connect the four lower loops to the slingbar on one lift unit, and then connect the two rear loops to the slingbar on the other lift unit.

### **Lifting:**

Prepare the patient for lifting; explain what is about to happen. Never leave the patient unattended during a transfer situation and always be attentive to the patient's signals. Begin lifting. Stop lifting when all lift straps are taut and check to ensure that everything is as it should be before continuing. Stop lifting if the patient is uncomfortable.

The sling is left in the chair under the patient.

## Removing BariSling

---

### **Horizontal position in bed**

Lower the patient to the bed and disconnect the lift straps from the slingbar. The patient turns first to one side of the bed, so that the BariSling can be tucked in under the patient's back. The patient then turns to the other side and the BariSling is removed.

## Lifetime information & Periodic inspection

---

The product has an expected lifetime depending on how the product is used according to the manual instructions.

The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer. The periodic inspection instructions for this product are found on [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## Safe working load

---

Different products on the same lift system (lift unit, slingbar, sling, scales and other lifting accessories) may have different allowable safe working loads. The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

Product	Material	Article number	Recommended patient weight	Max. weight
BariSling XXXL	Polyester	47001010	200-350kg/440-770lbs	570kg/1257lbs
BariSling XXXXL	Polyester	47001011	250-570kg/550-1257lbs	570kg/1257lbs

**NOTE!** Recommended for use with four-point slingbar or BariTurn.

## Material:

---

Polyester: Durable material with low friction.

## Care of the product:

---

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

## NOTE!

---

Lifting and transferring a patient always involves a degree of risk. Always make sure that the patient's health permits lifting. When in doubt, do not proceed with lifting. The caregiver is always responsible for the patient's safety.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic<sup>®</sup> is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic<sup>®</sup> is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic<sup>®</sup> is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic<sup>®</sup> offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic<sup>®</sup> representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.