BariVest

SystemRoMedic

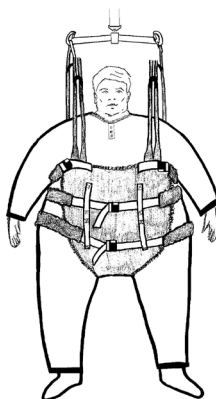
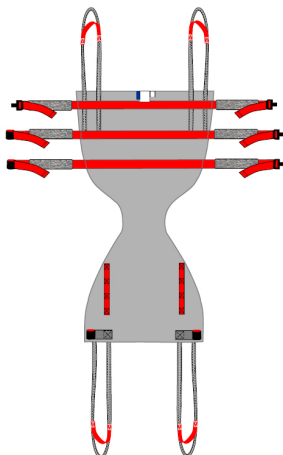
Instructions for use – English

Bruksanvisning – Svenska

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

REF

47103010
47103011

SWL: 570 kg/ 1257 lbs

Table of contents

English	3
Svenska	7
Norsk	11
Suomi	15
Français	19
Symbols	23

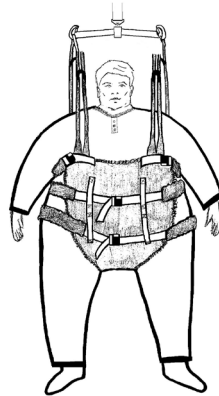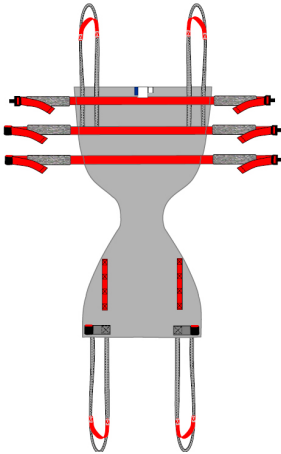BariVest is a lift vest that has been design as an aid to gait training for patients weighing 180 kg or more. It can be used with both mobile and stationary lifts, with a four-point slingbar or with BariTurn. BariVest features a back section, a front section and a narrower section in the middle to enable comfortable fit in the crotch area. It has two slingbar loops, both front and back, as well as belts at three levels.

Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort, trial fit the sling carefully.

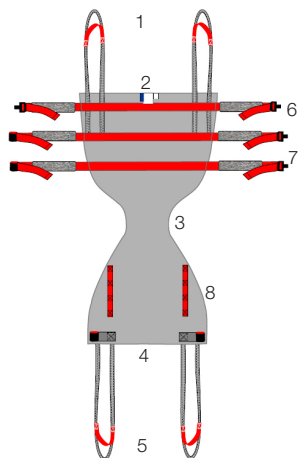
Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system.

Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

BariVest product description

BariVest features a back section, a front section and a narrower section in the middle to enable comfortable fit in the crotch area. It has two slingbar loops, both front and back, all of which are hooked to the slingbar/slingbars.

There are belts at three levels

The upper belt is secured with clasps on each side at the front for easy adjustment, and the two lower belts have a clasp that is fastened at the mid-section at the front.

BariVest consists of:

1. Rear slingbar loops
2. Back section
3. Narrow section
4. Front section
5. Front slingbar loops
6. Upper belt with clasps on both sides of the front section
7. Three belts with clasps at the front mid-section
8. Guide loops for the lower belts

Application of BariVest on the patient

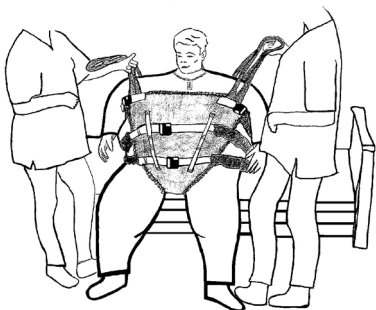
Seated position at the bedside:

Ensure that the front and back sections are placed the right way around. The product label is on the back section.

The patient leans to one side of the bed and the BariVest is placed centred under the seat, as far back under the patient as possible, with the narrow section under the crotch.

When the BariVest is correctly positioned, the patient sits up again and the front section is pulled up over the stomach as high as possible. Check to ensure that BariVest is evenly positioned both front and back.


Fasten the upper belt first, and then the two lower belts.

Tension the upper belt, so that it provides good support.


Adjust the lower belts, so that they sit comfortably. All belts may need to be adjusted again after the patient has risen.

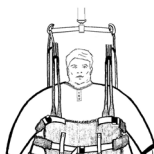Rising to a standing position

Application of the four-point slingbar:

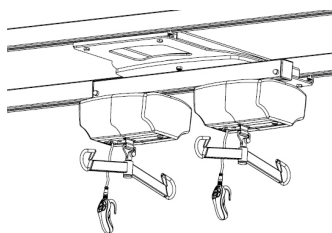For raising the patient to a standing position using a mobile lift or stationary lift unit, use of a four-point slingbar is recommended. First, apply the rear loops to the wider side of the slingbar, and then the front loops to the narrower side.


BariTurn application:

BariTurn is a turntable that enables gait training with two lift units. The lift vest can be applied in different ways, depending on whether more space is required lengthwise or width-wise.

Alternative 1: Fasten the loops on the back section to one slingbar, and then fasten the loops on the front section to the other slingbar.

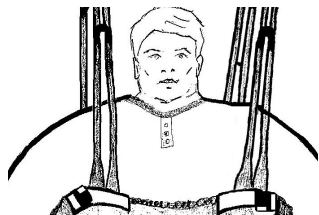Alternative 2: Hook one loop from the front section and one from the back section to one slingbar, and then hook the loops from the other side to the other slingbar.

Raising:

Prepare the patient by explaining the procedure. Never leave the patient unattended during a lifting situation. Always be attentive to the patient's signals. Begin lifting. Stop when all lift straps are tensioned, and check to ensure that everything is as it should be before continuing to lift. Stop raising if the patient is uncomfortable.

When the patient is standing, a walking support can be used.

Removing BariVest

Seated position at the bedside:

Lower the patient to the edge of the bed and disconnect the loops from the slingbar. Allow the front section to hang down between the patient's legs.

The patient leans first to one side of the bed, so that the BariVest can be tucked under the seat. The patient then leans to the other side and the BariVest can be removed.

Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions.

The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on www.directhealthcaregroup.com

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer. The periodic inspection instructions for this product are found on www.directhealthcaregroup.com

Product	Article number	Recommended user weight	Lift vests range	SWL
BariVest XXXL	47103010	180-300kg/397-660lbs	190-280cm/74.8-110In	570kg/1257lbs
BariVest XXXXL	47103011	250-570kg/551-1257lbs	230-310cm/90-122In	570kg/1257lbs

NOTE! Recommended for a four-point slingbar or BariTurn.

Accessories

BariVestCover, Protective Covers in disposables. Non-woven, can not be laundered. Discard after use.

Care of the product:

Read the product label.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

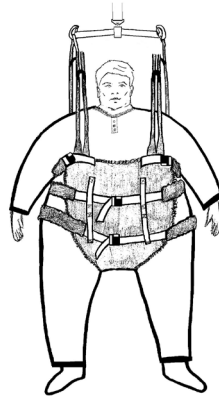
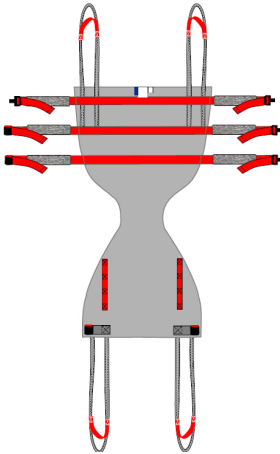
NOTE!

Mobility training always involves a measure of risk. Always make sure that the patient's health permits mobility training. When in doubt, do not proceed with mobility training. The caregiver is always responsible for the patient's safety.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors. Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

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Symbols

	<p>This product complies with the requirements of the Medical Device Regulation 2017/745</p>		<p>Visual Inspection</p>
	<p>Medical Device</p>		<p>Read the manual</p>
	<p>Caution</p>		<p>Product Code</p>
	<p>Manufacturer information</p>		<p>Batch Code</p>

SystemRoMedic[®]

Simple solutions for great results

SystemRoMedic[®] is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic[®] is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic[®] is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic[®] offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic[®] representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com