USER MANUAL  
BRUGERMANUAL  
BRUKERVEILEDNING

# TROPHY

handicare

9002432E

---

---

<b>English .....</b>	<b>5</b>
<b>Dansk .....</b>	<b>27</b>
<b>Norsk.....</b>	<b>49</b>

---

---

---

# English

© 2008 Handicare

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Handicare.

The information provided is based on general data concerning the constructions known at the time of the publication of this manual. Handicare executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Handicare cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but Handicare cannot be held liable for possible errors in the information or the consequences thereof. Handicare accepts no liability for loss resulting from work executed by third parties.

Names, trade names, etc. used by Handicare may not, as per the legislation concerning the protection of trade names, be considered as being available.

2008-01

## Table of contents

---

<b>Introduction</b> .....	<b>7</b>
This manual .....	7
Product reference .....	7
Service and technical support.....	7
Identification plate.....	7
Approval .....	8
CE declaration.....	8
Warranty .....	8
Liability .....	10
Used scooters and the environment .....	10
<b>1 Safety</b> .....	<b>11</b>
1.1 General safety instructions .....	11
1.2 Decals and instructions on the scooter .....	11
1.3 Electronic safety .....	11
<b>2 General description</b> .....	<b>12</b>
2.1 Product description .....	12
2.2 Main components .....	13
2.3 Tiller console and operating panel.....	14
2.4 Battery indicator.....	15
2.5 Speed .....	15
2.6 Indicators.....	15
2.7 Warning lights.....	15
2.8 Speed, daily distance and total distance.....	15
2.9 Brake button (option).....	15
<b>3 Operating instructions</b> .....	<b>16</b>
3.1 Inspection before use .....	16
3.2 Getting in and out .....	16
3.3 Free wheel lever .....	17
3.4 Switching on the scooter.....	17
3.5 Driving .....	18
3.6 Obstacles.....	18
3.7 Operation.....	19
3.8 Reducing the scooter.....	20
3.9 Transport .....	21
3.10 Seat adjustments.....	21
3.11 Adjustment options .....	22
3.12 Coding the combination lock.....	22
<b>4 Maintenance</b> .....	<b>23</b>
4.1 Maintenance table .....	23
4.2 Batteries .....	23
4.3 Checking and inflating the tyres.....	24
4.4 Cleaning the scooter.....	24
<b>5 Troubleshooting</b> .....	<b>25</b>
5.1 Troubleshooting table .....	25
<b>6 Technical specification</b> .....	<b>26</b>


## Introduction

### This manual

With this manual you can safely use and maintain (clean) the product. In case of doubt always contact your dealer.

**Strictly follow the procedures with a caution symbol! These procedures can - if they are not executed properly - result in human injury or damage to the product or the environment.**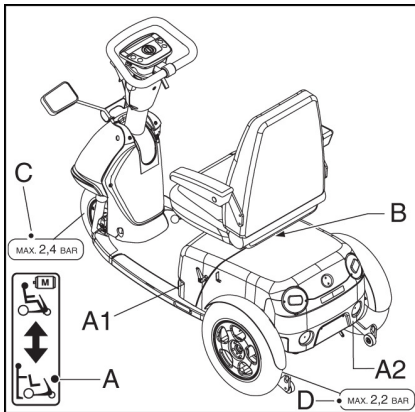


### Product reference

- A1 Free wheel Trophy Alpine (and models < 2012)
- A2 Free wheel Trophy (if not A1)
- A Identification plate
- B Tyre pressure front wheel(s)
- C Tyre pressure rear wheels

## Service and technical support

For information concerning specific settings, maintenance or repair work, please contact your dealer. Make sure you always mention the model, year of manufacture and identification number. This information is provided on the identification plate of the product. A service manual is available for the dealer.

## Identification plate

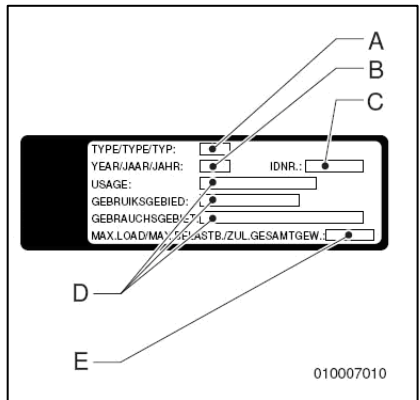- A Model
- B Year of manufacture
- C Identification number
- D User area indoors or outdoors
- E Maximum load in kg

### Approval

The product meets the following standards:

1. EN12184 (1999) Electrically driven wheelchairs and scooters, class C.
2. ISO7176-8 Requirements for impact, static and fatigue strength.
3. ISO7176-9 Climate tests for electric wheelchairs and scooters.
4. ISO7176-14 Standards and test methods for controller systems of electrically driven wheelchairs.
5. ISO7176-16 Requirements of resistance to fire.

The product is EMC (Electro Magnetic Compatibility) approved in accordance with EN12184 (1999).

### CE declaration

The product is in conformity with the provisions of the Medical Devices Directive and thus has CE marking.

### Warranty

In the following warranty and liability stipulations the terms and definitions as summed up hereafter must be explained as follows:

**Product:** The hand-operated or electric wheelchair or scooter manufactured and delivered by Handicare.

**Customer:** The person who directly obtains a Product from Handicare.

**Dealer:** The person who delivers a Product obtained from Handicare to customers or third parties.

**User:** The person who uses a Product manufactured by Handicare.

Irrespective of what is determined concerning warranty conditions applicable to the Product, in any case

the following applies with regard to the warranty:

1. Except insofar as described otherwise Handicare guarantees the Product for its suitability for the purpose for which the Product is intended - all of these points as described in this manual - and for the quality of the material of which the Product is made and the manner in which the Product is manufactured.
2. Repairs or replacement of parts of the Product that may be necessary as a result of faults that are based on qualitatively faulty material or manufacturing errors will be executed free of charge, as long as such faults occurred within one (1) year after the date of delivery of the Product to the Customer. The parts to be replaced must be shipped post-paid to Handicare. Disassembly or assembly of these parts is for the expense of the Customer. Therefore the following cases are not eligible for free repair or replacement as meant above:
  - The repair or replacement that is necessary in connection with faults that arise after one (1) year after the date of delivery of the Product to the Customer;
  - The repair or replacement that is required in connection with faults due to improper or careless use of the Product or that are based on the Product being used for another purpose than the one for which it is intended, in which regard it applies that if the Customer is a Dealer, this Dealer will indemnify Handicare against possible claims from Users or other third parties for faults based on an incorrect or careless use of the Product;

- Parts that are subject to wear, and the need for repair or replacement of the parts are the actual consequence of normal wear.
3. Irrespective of that stipulated under 2, it applies as far as an electric Product is concerned that with regard to the battery that forms part of the Product warranty is only given in case of faults or non-functioning of the battery that are demonstrably the direct consequence of material or manufacturing errors. A fault or non-functioning of the battery as a result of normal wear is not covered by the warranty as meant in these warranty stipulations. Similarly not covered by the warranty are faults or non-functioning that are the consequence of improper or unprofessional use of the Product or the battery that is part of the Product, including the incorrect charging of the battery and the failure to carry out timely and good maintenance, in which context it also applies that if the Customer is a Dealer, this Dealer will indemnify Handicare against possible claims from Users or other third parties that are based on the above mentioned improper or unprofessional use of the Product or the battery that is part of the Product.
  4. The warranty conditions as mentioned in above stipulations in any case become null and void if:
    - The guidelines of Handicare for the maintenance of the Product are not, or insufficiently, followed;
    - A necessary repair or replacement of parts is based on neglect, damage or abuse of the Product or a use of the Product for another purpose than the one for which it was intended;
  - Parts of the Product are replaced by parts of another origin than those which Handicare uses and/or parts of the Product are replaced without the permission of Handicare.
  5. The warranties as mentioned in stipulations 1 through 3 also become null and void if what is involved is re-used by a new user within the warranty period and that such re-use necessitates adaptations to the product and those adaptations are carried out without the instructions and/or at the order of Handicare.
  6. To retain rights under the above delineated warranties the Customer must, in case of damage or other calamities, as rapidly as possible contact Handicare and inform them as fully as possible. The possibility of taking recourse to the above-mentioned warranty conditions becomes null and void for the Customer in any case after 20 workdays after the claim situation or the calamity arises that was the reason for the recourse to the guarantee.
  7. The replacement of a part or the repair or the reconditioning of the Product within a current warranty period does not extend the warranty period.
  8. Handicare gives no warranty on repair to or reconditioning of the Product carried out other than under order of and/or at the instructions of Handicare. If repairs and/or reconditioning are executed by or on behalf of a Customer, the Customer indemnifies Handicare with respect to the claims of third parties who result, in the broadest

## Introduction

---

sense of the word, from such repairs or reconditioning.

electronic waste. Used batteries belong to chemical waste.

## Liability

Irrespective of what is determined regarding liability in the general conditions applicable to the Product, with regard to liability in any case the following applies:

1. Taking into consideration the following stipulations, Handicare only accepts liability for loss due to death or physical injury that is the result of a defect in the Product for which Handicare is responsible and for damage to another object that is the private property of the user of the Product, as long as said loss is the direct result of a fault in the Product. At all times, the liability of Handicare is limited to the amount which is being covered by the third-party insurer of Handicare.
2. Handicare accepts no other or further liability than delineated under 1. In particular Handicare accepts no liability for consequential damage, in any form whatsoever.

## Used scooters and the environment

If your scooter has become superfluous or needs to be replaced, it can usually be taken back by your dealer. If this is not possible, please contact your local authorities for the possibilities of recycling or an environmentally friendly way of disposing of the used materials.

For the production of the scooter several plastics and metals have been used. The scooter also contains electronic components that must go to

# 1 Safety

Handicare accepts no liability whatsoever for damage or injury caused by the failure to (strictly) adhere to the safety guidelines and instructions or else due to carelessness during the use and cleaning of the scooter and possible accessories. Depending on the specific work circumstances or the accessories used, additional safety instructions may be required. Please contact your dealer immediately if you observe a potential danger during the use of the product.

## 1.1 General safety instructions

- **The user of the scooter is at all times fully responsible for the fulfilment of the locally applicable safety regulations and guidelines.**
- **The technical specifications must not be altered.**
- **Modifications to (parts of) the scooter are not permitted.**

## 1.2 Decals and instructions on the scooter

- **Signs, symbols and instructions placed on this scooter comprise part of its safety facilities. They must never be covered or removed and must remain present and clearly legible throughout the entire lifespan of the scooter.**

- **Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your dealer with regard to this.**

## 1.3 Electronic safety

- The standard version of your scooter has been tested according to the strictest EMC requirements. Mobile telephones have no influence on the driving behaviour of the scooter.  
When using a mobile telephone in the vicinity of a scooter with special adaptations, you are advised to first switch off the scooter.
- Your scooter may influence electromagnetic fields such as alarm systems.
- If the electronics of the scooter is not properly screened off, this may affect sensitive electrical appliances, such as shop alarms and garage openers. The scooter has been tested for this. Should any problems of this nature occur, please report these to your dealer immediately.

## 2 General description

### 2.1 Product description

The Trophy is an electrically driven scooter for longer distance outdoor use. The unique suspension system, stability and manoeuvrability offer excellent driving characteristics and maximum comfort. Fully adjustable, the Trophy scooter can be tailored to optimally meet your specific requirements, ensuring a relaxing position of the hands and shoulders. With controls on the right- or left-hand side the clear control panel can be easily operated with very little operating force. The Trophy scooter provides maximum independence!

The Trophy has been designed for:

- Transport of persons to a maximum weight of 160 kg.
- Use on paved roads, sidewalks, foot- and bicycle paths.
- Use in and around the house.

Your dealer must give you good driver's instructions before you can independently operate the product and participate in traffic.

You must be able to correct the consequences of actions while driving the scooter.

Consider operating the scooter under the supervision of an experienced dealer during first operations.


The scooter is not a motorised vehicle in the sense of traffic regulations.

The Trophy has a maximum speed of 15 kph.

If you use the scooter for other purposes than those for which it is intended, Handicare accepts no liability whatsoever for damage or injury resulting from such use other than for which the scooter was developed and designed.


**Please study this manual carefully before you use the scooter.**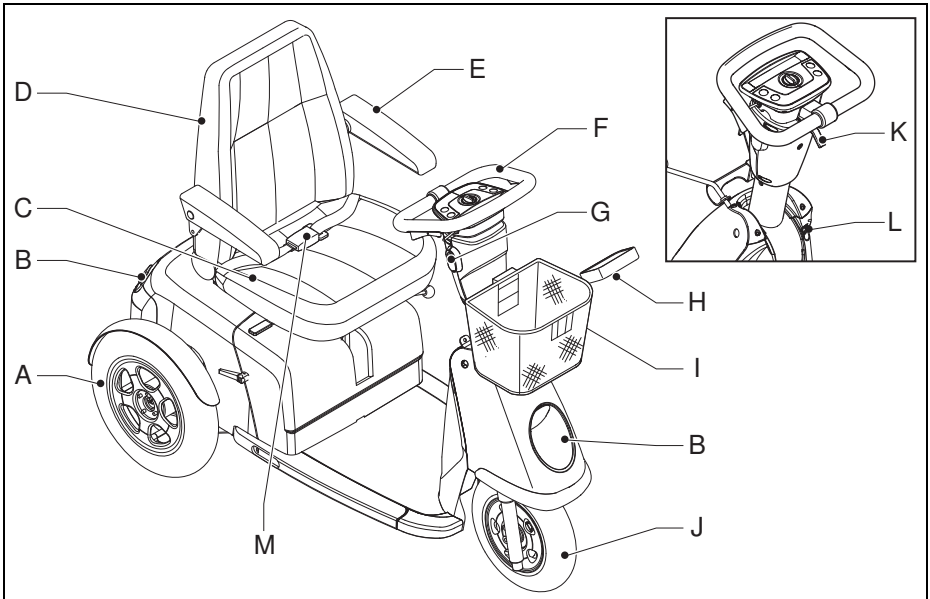

## 2.2 Main components

- |   |                |   |   |
|---|----------------|---|---|
| A | Rear wheels    | I | Basket  |
| B | Lighting       | J | Front wheel(s)                                  |
| C | Seat           | K | Tiller angle adjustment                         |
| D | Backrest       | L | Charge connection for the batteries             |
| E | Armrest        | M | Lap belt, to be provided by the dealer (option) |
| F | Tiller console |   |   |
| G | Ignition lock  |   |   |
| H | Mirror         |   |   |

## General description
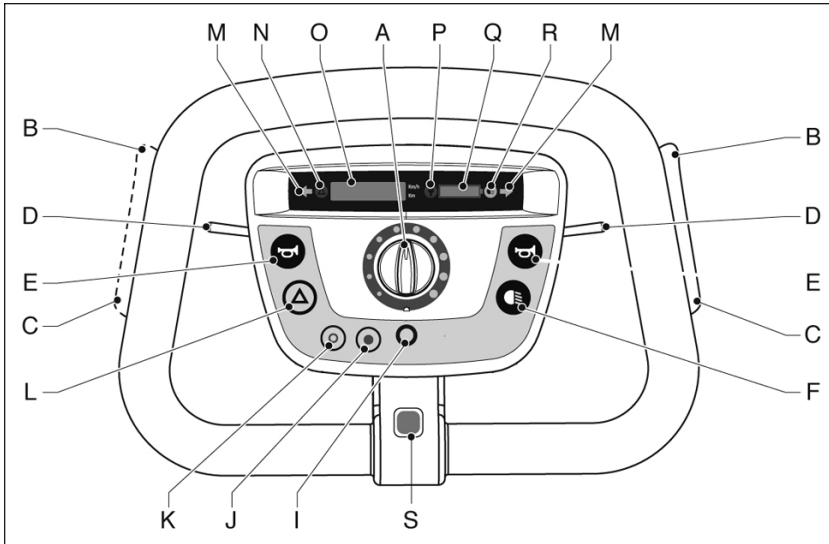


### 2.3 Tiller console and operating panel

- A Speed control
- B Throttle: drive forward
- C Throttle: reverse
- D Indicators left/right
- E Horn
- F Lighting
- G Option: forward (when using foot throttle)
- H Option: reverse (when using foot throttle)
- I Selector switch for speed, daily distance or total distance
- J Option: 'down' (e.g. for seat with electrical high/low)
- K Option: 'up'
- L Warning lights
- M Indicator 'indicators switched on'
- N Warning lights on
- O Speed, daily distance and total distance indicator
- P Indicator light 'charge battery'
- Q Battery indicator
- R Lights switched on
- S Option: brake button

#### With dual control

- T Right throttle
- U Left throttle

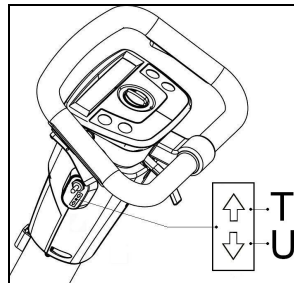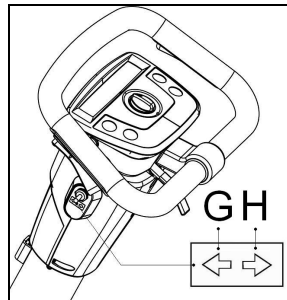## 2.4 Battery indicator

The battery indicator shows the condition of the batteries.

When all the lights are on, the batteries are fully charged. As the batteries are depleted, lights from right to left will go out.

- One or more lights are on: OK.
- Lights are flashing: the batteries must be charged as soon as possible.
- Lights are flashing quickly: malfunction, see 5.

- **If only the red lights are on, the batteries must be charged immediately.**
- **If the indicator light 'charge battery' is on: charge batteries.**

## 2.5 Speed

The speed can be set as follows:

1. Set the desired maximum speed with the speed control (D).
2. Dose with the throttle.

## 2.6 Indicators

The left and right turn indicator can be operated on both sides.

The indicator will automatically switch off after approximately 12 seconds.

The indicator can be switched off manually by pushing it once again.

## 2.7 Warning lights

Switch on the alarm lights if you fear you will not be seen by other traffic, or if you have stopped because of a malfunction. The alarm lights can also be switched on if the key is not in the ignition.


## 2.8 Speed, daily distance and total distance

The display shows the speed (kph), daily distance (km) or total distance (km). Press the selector switch to switch between speed, daily distance and total distance.

Select daily distance and keep the key pressed in for 2 seconds to reset the daily distance.

## 2.9 Brake button (option)

1. Push the brake button.
2. After having used the brake button, switch off contact and reboot to drive again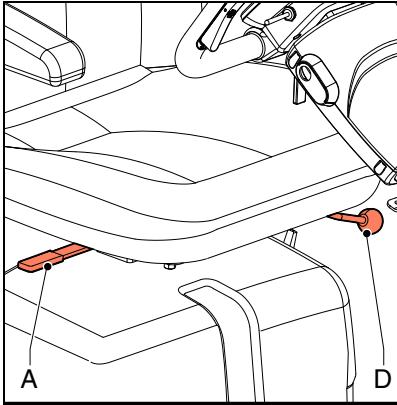

### 3 Operating instructions


#### 3.1 Inspection before use

It is important to check the following points before you start driving the scooter:

- If the seat is properly locked (A).
- If the lights and indicators are working properly.
- If the tyres are sufficiently inflated.
- If the batteries are charged sufficiently. The green battery indicator lights must be on.

- **In winter the batteries have a lower capacity. In case of light frost the capacity will drop to approximately 75% of the normal capacity. At a temperature of less than  $-5^{\circ}$  the capacity will be approximately 50% of the normal capacity. This will affect the range.**
- **If you observe anything unusual while driving, release the throttle and take the key out of the ignition once you have stopped.**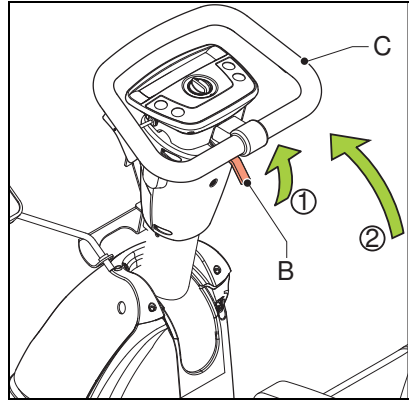

#### 3.2 Getting in and out

1. Make sure the scooter is switched off electrically.
2. Put the free wheel handle in the DRIVE position.

#### Tiller angle adjustment

Pull the handle (B) for the tiller angle adjustment (C) and push the tiller forward.

#### Turning the seat

1. Move handle (A) forward and turn the seat  $90^{\circ}$  to the left or the right. The seat will lock automatically when turned  $90^{\circ}$ .
2. Now seat yourself on the seat.
3. Turn the seat back as described above. When the seat has been turned to its original position, it will click back into place.

#### Folding up armrests

The armrest can be folded up so that you can get in and out sideways.


#### Sliding seat adjustment

Pull up the handle (D) and move the seat forward or backward into a comfortable position.

Switch of the scooter electrically before you get in or out! This way you will avoid that the scooter starts to move when you accidentally squeeze an operating handle.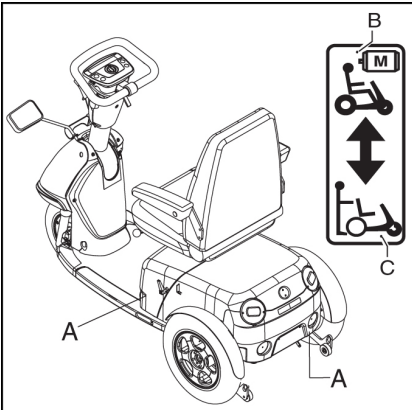


### 3.3 Free wheel lever

Put the free wheel lever (A) in the DRIVE position (B) to be able to drive the scooter electrically.

To be able to push the scooter, put the free wheel lever in the Push (C) position.

- Never put the scooter in the Drive position when you are parked on a slope. When the scooter is parked on a slope that is too steep, it can start moving down the slope.
- After pushing, the free wheel lever should immediately be returned to the Drive position.
- The free wheel lever should only be put in the Push position if the scooter is to be pushed. In the Drive position the automatic parking brake is activated.

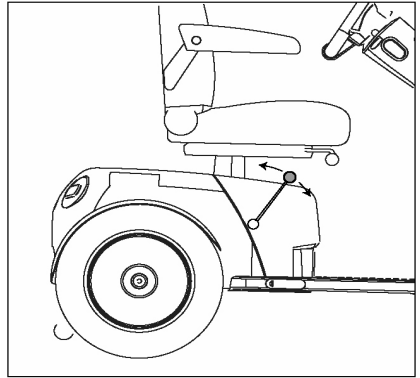### 3.4 Switching on the scooter

To switch on the scooter put the key in the ignition and turn the key a quarter of a turn.


When an electronics error is detected, the battery indicator will start flashing. The electronics are blocked.

Make sure that the hand brake isn't still in its parking position when you take off.

Make sure no other keys / key ring are attached to the scooter's key ring while driving. This way you will avoid that that the scooter is accidentally switched off and comes to a standstill.

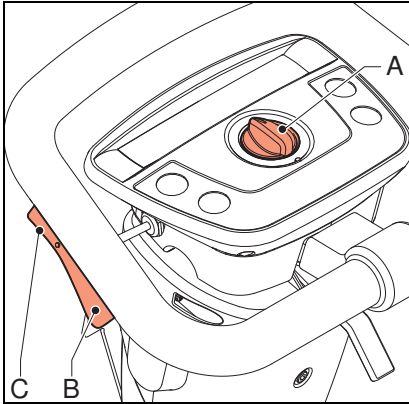### 3.5 Driving

- As a scooter driver you are vulnerable in traffic. Other traffic participants may not always notice you.
- Keep to the applicable traffic regulations.
- Avoid out-of-the way routes, so that help can be on its way quickly when needed.
- Use lap strap if applicable

#### Driving forward


1. Switch on the scooter, see 3.4.
2. Use the speed control (A) to select the desired speed.
3. Squeeze the bottom part (B) of one of the operating handles. The further you squeeze the handle, the faster you will drive.

#### Reversing

1. Release the operating handle. The scooter will come to a standstill.
2. Slowly squeeze the top part (C) of one of the operating handles. The scooter will drive in reverse.
3. When you release the operating handle, you will brake automatically.

Select speed position 2 using the speed control (A). In the lowest speed position, the scooter will not drive.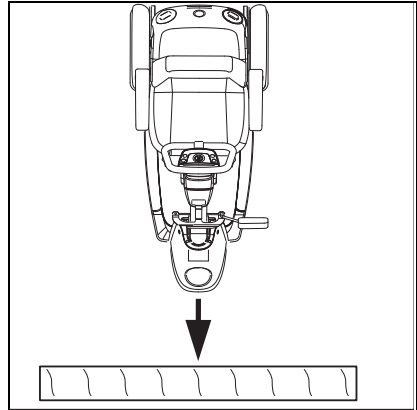

### 3.6 Obstacles

#### Driving up obstacles

1. Drive straight up to the kerb.
2. Slowly squeeze the operating handle. Drive up the sidewalk without changing direction.
3. As soon as the front wheel is on the sidewalk, you must keep your speed for the rear wheels to be able to move up the sidewalk as well. If you cannot drive up onto the sidewalk, find a lower place where you can.

#### Driving down obstacles

1. Drive straight up to the edge of the sidewalk.
2. Slowly squeeze one of the operating handles. Drive carefully down the kerb without changing direction.

- Do not drive down stairs!

- In panic, release immediately the throttle. The scooter will stop automatically.

### 3.7 Operation


- Switch of the scooter electrically before you get in or out! This way you will avoid that the scooter starts to move when you accidentally squeeze an operating handle.
- Maximum slope witch can safely be driven with maximum weight according to ISO 7176-2 is 10° at 160 kg.

#### Driving up a slope

- Slopes with an angle of more than 10° must not be taken.
- Always drive slowly and with awareness on slopes.
- Do not drive down slopes with loose gravel or a sandy surface, as one of the rear wheels might slip.
- Avoid sudden and jerky movements.
- Avoid directional changes and turns on slopes if you notice a strong decrease in speed when driving up a slope, take a less steep route.
- Driving on a slope too long can cause overheating.

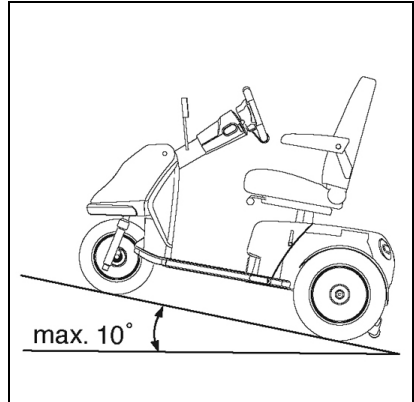#### Driving down a slope

- If you need to drive down a slope, make sure the speed control is in the lowest position and drive as slowly as possible and with the utmost control.
- It can be very dangerous to reverse down a slope.
- Move the upper part of your body slightly backward and always brake slowly. If you brake fast, you may cause the scooter to tip over forward.

#### Curves


- Never take curves at full speed. Decrease speed before taking a curve.
- Use the indicators when changing direction.
- When making difficult movements, turn the speed controller in a low position.

### Miscellaneous

- Drive carefully on slippery roads, resulting from rain, ice or snow!
- Prevent the scooter from coming into contact with seawater: seawater is caustic and may damage the scooter.
- Prevent the scooter from coming into contact with sand: sand may affect the moving parts of the scooter, resulting in unnecessarily rapid wear.
- Do not drive the scooter when you are under the influence of drugs, alcohol or medication that could affect your ability to drive.
- You must have sufficient eye sight to be able to safely drive the scooter.
- It is imperative to turn on the lights in case of reduced visibility.
- When driving at a high speed you must be extra careful. Select a lower maximum speed on sidewalks and in pedestrian areas.
- Be sure that no items of clothing hang loose. These could be caught between the wheels.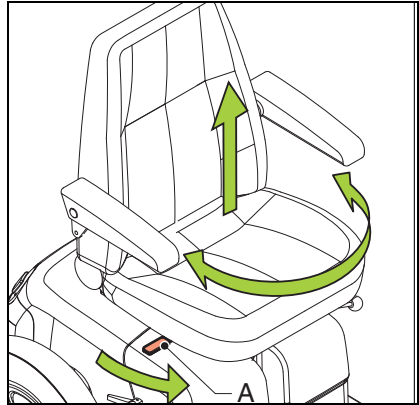


### 3.8 Reducing the scooter

#### Removing the seat

1. Move the handle (A) forward.
2. Lift the seat from the socket in which it turns.  
The seat can be removed more easily if you slightly turn the seat while lifting it.


#### Folding down tiller

Pull up the handle (A) and fold down the tiller (B).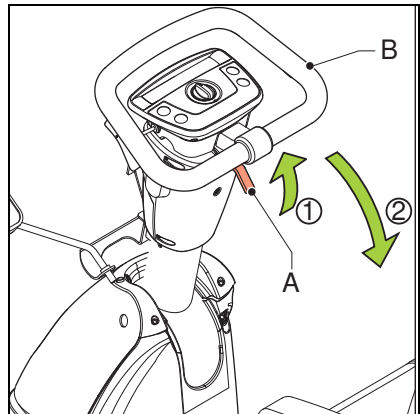

### 3.9 Transport

The scooter can be placed in a car. The best way to do this is by using steel planking, across which the scooter is pushed into the car. The scooter may also be lifted; this requires at least two strong persons.

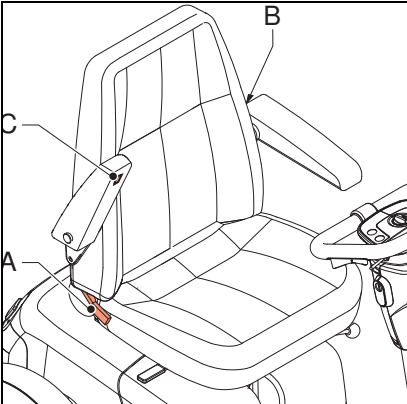- Considering the nature of the scooter, the user is supposed to make a transfer to a regular car seat. You may not be transported in a car or taxi while sitting on the scooter, even if the vehicle concerned has been adapted for the transport of scooters. The reason for this is that a scooter cannot offer the level of safety that is offered by standard car seats, no matter how well the scooter is fixed to the vehicle concerned.
- After the scooter has been placed in the car, you must make sure that the free wheel lever is in the Drive position.
- The scooter must be secured with lashing straps at the tie-down points on the front and back of the frame.

### 3.10 Seat adjustments

#### Backrest

1. Seat yourself on the seat and pull up the handle (A). The backrest will move forward.
2. Press the backrest backward by leaning back to a position that is most comfortable for you. Then push the handle down to secure the backrest in the position you have chosen.

**Do not operate the handle if the seat is unoccupied, because then the backrest will come forcefully forward.**

#### Lower-backrest


Seat yourself on the seat and turn the knob (B) in such a way that the lower-backrest reaches the most optimal position for you.

#### Armrest

Turn the wheel (C) to adjust the height and angle of the armrest.

**If the position of the backrest is adjusted, this will affect the position of the armrests.**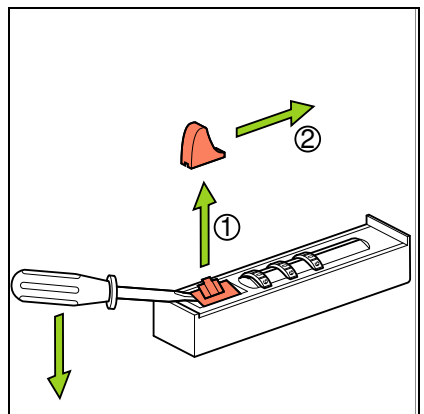

### Headrest height

Choose a comfortable height for the headrest.

### 3.11 Adjustment options

To ensure optimal posture and positioning as well as a relaxing position of the arms and shoulders, your dealer can adjust the:

- Seat height
- Tiller console height
- Tiller console angle
- Spring tension (suspension of the scooter)

### 3.12 Coding the combination lock

The code is set to 0-0-0 ex-works.

#### Adjusting the code

1. Remove the button (1) using a small, sharp object.
2. Move the released button (2) in the direction of the wheels.
3. Keep the button in this position and set your personal code by turning the wheels.
4. Remember this code.
5. Release the button: your code has been set.
6. Firmly push the button (1) back in its position.
7. To unlock and release the basket, move the button to 'open'.

## 4 Maintenance

### 4.1 Maintenance table

It is recommended to have your scooter serviced by your dealer once a year or, in case of intensive use, every six months. The table below shows the maintenance activities you can perform yourself.

Frequency	Description
Daily	Charge the batteries after each use, see 4.2.
Weekly	Check tyre pressure and inflate the tyres if necessary, see 4.3
Monthly	Clean the scooter, see 4.4.
	Clean the upholstery, if necessary, see 4.4.
	Check the batteries (in summer: once every fortnight; in winter: once a month), see 4.2
Every three months	Lubricate the turning system of the seat.

### 4.2 Batteries

The scooter has 'dry' gel batteries. These dry batteries are entirely closed and maintenance free.

**The use of 'wet' batteries is not permitted.**

#### Charging the batteries

Consult the owner's manual of the battery charger.

In case of normal use the batteries should be charged every night.

1. Switch off the scooter.

2. Put the plug of the charge cord in the charge connection, see chapter 2.2.
3. Put the plug of the battery charger in the wall outlet of the electricity mains.

When the batteries have been charged:

1. Remove the plug from the charge connection.
2. Remove the plug of the battery charger from the wall outlet of the electricity mains.

- **As soon as the plug of the charge cord is connected to the charge connection, the operation of the scooter is switched off.**
- **Always remove the charge cord when the batteries have been charged. This way, you will prevent slow depletion of the batteries.**

#### Maintenance of the batteries

Refer to the battery instructions.

- **Make sure that the batteries are always well charged.**
- **Do not use the scooter if the batteries are almost depleted and never completely deplete the batteries. This can seriously damage the batteries and you run the risk to come to an unintended standstill.**

#### Cleaning the batteries

Keep the batteries clean and dry. Dirt and water can cause a leak, reducing the capacity of the batteries. After cleaning grease the poles with acid-free vaseline.

### Replacing the batteries

If the capacity of the batteries is reduced to the extent that the scooter can only make very short trips, the batteries are at the end of their lifespan. The batteries must be replaced as soon as possible. Contact your dealer for this.

### 4.3 Checking and inflating the tyres

- **Make sure the tyres are kept at the correct pressure.**
- **The tyre pressure may never exceed the maximum value indicated on the tyre.**

Inflate the tyres with a foot pump or a bicycle pump, in combination with the reducing nipple included in the delivery. You can also have your tyres inflated at your local petrol station.

- Tyre pressure front wheel: 2.4 bar
- Tyre pressure rear wheel: 2.2 bar

### 4.4 Cleaning the scooter

#### Removing dry dirt

Upholstery, metal parts and frame parts can usually easily be cleaned with a dry soft cloth.

#### Removing mud and other wet dirt

The dirty parts can best be cleaned by first wiping them with a wet sponge and then wiping them dry with a dry soft cloth.

#### Cleaning the upholstery

Clean the upholstery with a damp cloth and household soap. Then wipe the upholstery dry with a soft dry cloth.

- **Never use abrasive or aggressive cleansers. They can scratch the scooter.**
- **Do not use organic solvents such as thinner, benzene or white spirit.**
- **Be careful with water in view of the electronic system.**
- **Do not dry clean, iron or spin-dry the upholstery.**

## 5 Troubleshooting

### 5.1 Troubleshooting table

If the scooter will not function, while the batteries are fully charged, check the following points before contacting your dealer.

1. Switch off the scooter and then switch it on again. Check if the problem has been solved.
2. Check if all charge plugs are properly connected and secured.
3. Check if the scooter is in the DRIVE position (not in NEUTRAL).
4. Check if the position of the speed control has not changed.
5. In case of a malfunction the battery indicator lights will be flashing. Use the table below to find out the cause and report this information to your dealer.

Number of lights	Fault description	Action
6	Scooter is connected to the battery charger.	Disconnect the scooter from the battery charger.
7	Throttle error.	Make sure the throttles are not activated when the scooter is switched on.

In all other cases contact your dealer.

### 6 Technical specification

Model		Trophy 3W	Trophy 4W	Trophy Alpine
Max. user weight	kg	160	-	-
Total length	mm	1400	-	-
Total width	mm	660	-	-
Total weight without batteries	kg	155	166	155
Transport weight of heaviest part	kg	90	101	90
Static stability downward	°	15	-	-
Static stability upward	°	15	-	-
Static stability sideways	°	15	-	-
Range (ISO 7176) 85 Ah batteries	km	60	-	-
Range (ISO 7176) 56 Ah batteries	km	40	-	-
Dynamic stability on slopes	°	14	-	-
Climbing capacity for obstacles	mm	100	-	-
Starting on a slope	°	10	10	14
Maximum speed forward	km/u	15	15	11
Minimum brake distance at maximum speed	mm	2100	-	-
Idem, maximum	mm	2200	-	-
Seat angle	°	2	-	-
Effective seat depth	mm	440	-	-
Effective seat width	mm	460	-	-
Seat height front, minimum	mm	420	-	-
Idem, maximum	mm	550	-	-
Back angle, minimum	°	90	-	-
Idem, maximum	°	180	-	-
Back height	mm	550	-	-
Armrest height, minimum	mm	180	-	-
Idem, maximum	mm	220	-	-
Front armrest to backrest	mm	260	-	-
Minimum turning radius	mm	1250	1480	1250
Capacity to drive over obstacles	mm	100	-	-
Ground clearance	mm	70	-	-
Turning space	mm	1650	1940	1650

Batteries	
Maximum dimensions batteries	334x178x253 mm
Battery type	Gel/AGM
Scooter weight incl batteries	170 kg
Maximum battery capacity	110 Ah
Maximum battery capacity with 5 hours charging time	27.5 Ah
Maximum permissible charging voltage	13.6 V
Maximum charging current	10 A
Connector type	A DIN 72311
Connector type	A DIN 72311

Operating force	
Throttle	< 60 N
Electronic switches	< 13.5 N
Parking brake	< 60 N
Insert charge plug	< 60 N

Wheels	
Diameter front wheel	12.5" x 2.25"
Diameter rear wheels	3.00x10
Tyre pressure front wheel	2.4 Bar
Tyre pressure rear wheels	2.2 Bar

---

# Dansk

© 2008 Handicare

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorization of Handicare.

The information provided is based on general data concerning the constructions known at the time of the publication of this manual. Handicare executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Handicare cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but Handicare cannot be held liable for possible errors in the information or the consequences thereof. Handicare accepts no liability for loss resulting from work executed by third parties.

Names, trade names, etc. used by Handicare may not, as per the legislation concerning the protection of trade names, be considered as being available.

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Handicare B.V.