

# **Bure Standard/S**

Bure Standard and Bure S are available in three versions: Electric, Manual and Gas. Bure S is a narrower version of the Bure Standard specially adapted to work in confined spaces. Both models have stepless forearm support height adjustment.

#### Ergonomic handles

Ergonomically designed handles that are steplessly adjustable in every direction for optimum individual comfort. This is extremely important for stroke patients who can thus reach their "usual" hand position.

Easily reachable hand controller

The hand controller is located such that both of the carer's or user's hands can easily reach it.

## Power Rise

Bure Standard EL and Bure S provide excellent assistance when helping patients and users stand. The strain on personnel, and thus repetitive strain injuries, are minimised. Some patients can even get up on their own with the assistance of the StandTall Walker.

## Eminently suitable at home

Bure S is perfectly suited to confined spaces such as at home or in some care accommodation. Bure S will pass through most doorways that are 70 cm or wider.

#### Electric

Bure Standard and
Bure S electric are ideal for users who
need a walkers equipped with power rise
and which are capable of height
adjustment once a patient is standing.

#### Gas

Bure Standard and Bure S gas are ideal for users who need a quality walker capable of stepless height adjustment but who do not need assistance from a power rise actuator.

### Sustainable

StandTall Walkers are designed to last. For example, they have extra reinforcement bands around the arm rest cushions to prevent them from splitting from impacts to the side.

#### Stable

Height-adjustable handles

See accessories.

The walker has anatomical pads that provide relief for shoulders, arms and the neck and shift the focus to larger muscle groups such as leg and chest muscles instead. This improves stability and makes the walker easier to steer.

# Flexible stepless adjustment

Electrically driven height adjustment makes it easy to pre-set the StandTall Walker to the desired height. Once the patient is standing in position the height can be steplessly adjusted for comfort and the proper support for an optimal walking configuration.

## Brakes and directional locks

There are optional one or two-handed brakes that can be positioned so that the user can reach them from the walking position. The handbrake acts on the rear castors and the brake assembly also includes directional locks that make it easier for the user to stay on course when walking.

## Open design

The open design provides the same stability but with greater room for manoeuvre. The patient does not feel shut in and it makes assistance with standing up and dressing easier.

#### Manual Bure Sta

Waterproofing

wet spaces.

All Bure Standard EL and

Bure S EL parts have IP 65

international protection ratings

and can therefore be used in

Bure Standard and Bure S manual are great for users who need good, individually adaptable walkers but who do not need to adjust the height very often and who have no need of assistance from a power rise actuator.

## **Technical Specification**

Part No.	Product	Width	Max user weight	Length	Height	Armrest/Internal dimension
56-312T	Bure Standard EL	78cm	150kg	80cm	90-130cm	25-50cm
56-311	Bure Standard Gas	78cm	150kg	80cm	90-129cm	25-50cm
56-310	Bure Standard Manual	78cm	150kg	80cm	90-133cm	25-50cm
56-302T	Bure S EL	66cm	150kg	79cm	90-130cm	30-45cm
56-301	Bure S Gas	66cm	150kg	79cm	90-129cm	30-45cm
56-300	Bure S Manual	66cm	150kg	79cm	90-133cm	30-45cm

All walkers may be ordered with 75, 100 or 125mm castors – specify when ordering. (Unless otherwise specified, the heights shown below refer to products with 125 mm castors). Examples of possible parts combinations are shown below.