Bure Standard/S Bure Low/S

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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SWL: 150 kg / 330 lbs

Table of contents

English	4
Svenska	18
Danske	32
Suomi	46
Deutsch	60
Français	75

Product Overview

REF

56-310	56-312T/100
56-310/100	56-321
56-310/75	56-321/75
56-310/75-B/0	56-321/75-PU
56-310-B/0	56-321-S
56-311-PARSONS-KD	56-321-S/100
56-311	56-321-S/75
56-311-PARSONS-KD	56-321-S/75-PU
56-311/100	56-321-S/75-UK
56-311/100-PU	56-322/75
56-311/75	56-322-S
56-311/75-B/0	56-322-S-B/0
56-311/75-B/1	56-322-S/75
56-311/75-PU	56-301/75-RS
56-311-B/1	56-302-W
56-311-BRAKE/1-RED	56-302T/75-PU-AUS
56-311-BRAKE/0-PU	56-302T
56-311-PU	56-302T-FKN_R
56-311-RED	56-302T/75
56-311-RS	56-302T/75-FKN_R
56-300	56-302T/75-PU
56-300/100	56-302T-PU
56-300/75	56-302T-TK
56-300/75-DIR	
56-300/75-B/0	
56-300/75-B/0-PU	
56-300/75-B/1-PU	
56-300-B/0	
56-300-B/1	
56-301	
56-301-B/0	
56-301-PU	
56-301/75	
56-301/75-PU	
56-312T	
56-312T-B/0-PU	
56-312T-BRAKE/0	
56-312T-B/0-TKN_R	
56-312T-PU	
56-312T-TK	
56-312T-US	
56-312T/75	
56-312T/75-PU	

Bure Standard/S

Bure Low/S

Instructions for use - English

System**RoMedic**

1. Intended use

Walkers are to be used as indoor, movable and portable assistive aids, which together with approved accessories are intended to be used for transfer from sitting to standing positions, and walking training. Walkers are to be used by persons with reduced muscular strength, who shall be able to stand and perform simple leg movements. Walkers are to be used for leaning on when walking and standing during early mobilization training. Walkers and accessories are intended for indoor use only in Home Healthcare environment and Professional Healthcare facility environment.

2. Contraindications, Precautions and Warnings

Contraindications
The device may not be used by patients above the maximum weight indicated on the label of the device.
Precautions
Check that the device is used on a dry and flat surface.
Check the device is correctly mounted/assembled before its first use.
Check the device after every folding/disassembly after any transport.
Check height adjustment and leg spreading motions and inspect the actuators' or gas pistons' full ranges.
Activate the brakes whenever the device is not in use.
Check the function of the brakes before use.
Inspect the device regularly to detect any signs of damage. If there are any visible signs of damage, contact Direct Healthcare Group Customer Service.
In case of an electrical model of the device, check that the hand control does not show signs of wear before every use.
Check that the hand control markings are in accordance with the lifting functions.
Check the battery status of an electrically operated model, before using the device.
It is important to never leave the user alone during the transfer.
Warranty applies only if repairs or alterations are made by personnel who are authorized by Direct Healthcare Group.
Ensure that there are no obstacles or people in the way of the device, when moving.
Handle batteries with care. Do not drop.
Use only batteries and cables that are intended for the device as instructed by the manufacturer.
Verify that potential accessories that might be used are properly attached to the device before use.
Low speed is recommended when moving with the device.
Take care not to drive the device over thresholds with high speed or force. Approach the threshold with lower speed and communicate the coming threshold to the user.
Keep the instructions for use together with the product so that it can be used as a reference if necessary.
Beware of moving parts during usage.
Perform a risk assessment on the user before using the Walker to ensure they can stand and perform simple movements.

Warnings
The caregiver must be able to read and understand the Manual/IFU of the product.
It is important to use only approved accessories to prevent unintended detachment of components and subsequently a fall that may lead to patient injury.
Use careful and gentle maneuvers when moving the device.
Perform maintenance/service of the device, according to the instructions in the manual/IFU, at least once every 12 months.
Accessories must be properly fitted and tested in relation to the user's needs and functional ability.
Special care must be taken when using strong electrical power sources such as diathermy and the like so that diathermy cables are not placed on or near the device. In case of doubt consult with a Direct Healthcare Group representative.
When lifting the device, only use the denoted handholds. At least two people are required for lifting the device.
Do not leave a care recipient unattended during a transfer.
Never move the lift by pulling on the actuator or gas piston.
The device must not be lowered into water.
The device must not be cleaned using steam.
The device must not be left or stored in a damp or humid environment.
The device must not be charged in a wet room.
The device must not be used outdoors, only indoors.
To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth.
No modification of this equipment is allowed
Do not modify this equipment without authorisation of the manufacturer
If this equipment is modified, appropriate inspection and testing must be conducted to ensure continued safe use of this equipment.

Before use:

- Inspect the packaging for any damage.
- Check for correct product being delivered.
- Check to ensure that the device is free from damage.
- Check if all the parts/components are included in the packaging.
- Check if all approved accessories are included in the package.
- Check if Product Quality Approval document is included in the packaging. Save this document for future contact with manufacturer.
- Make certain the device is properly assembled.
- Check lifting function and bottom-frame adjustment function.
- Check if it is possible to move the device in all directions.

3. Standing Up Using the Bure

With electrical components:

1. Ask the user to sit on the edge of the bed or chair. Bring out the walking table and place it in front of the user. Lock the brakes on the wheels.
2. Ask the user to place their arms on the armrests. If necessary, ask the user to move forward on the seat base so that the user reaches the handles.
3. Ask the user to lean forward. To start raising the walking table, press the UP button on the controller. Raise the walking table to a desired height.
4. Unlock the brakes. Assist the user while walking.

With gas piston:


1. Ask the user to sit on the edge of the bed or chair. Bring out the walking table and place it in front of the user. Lock the brakes on the wheels.
2. Raise the walking table by pressing the handle on the gas piston.
3. Ask the user to lean forward, grasp the armrests and stand up.
4. Adjust the height of the walking table to a desired position

4. Using Bure Walker with Gait Training Kit 56-388

Top view

- The gait support harness has four blue straps with a plastic buckle allowing for adjustments in length, having steel carabiner clips at the ends to be fastened to the gait walker. The carabiner clips on to the metal mounts placed under the table of the Gait walker.
- Apply the gait walker support harness by clipping on the two front carabiners to the front mounters, one on each side of the table.
- Place the user standing up in the Gait walker. Pull back the harness in between the legs and clip on the carabiners to the back mounters, one on each side of the table. Adjust the harness to the user by pulling in the back strap loops.
- **Observe!** Intended use of the Gait trainer support harness is to practise walking. **It is not** a sitting or lifting aid.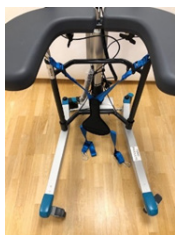

4. Raising/Lowering the Frame - Electric

The desired height is adjusted with Arrow <UP> or Arrow <DOWN>.

Handset TiMOTION Plus also have charge indicator / battery (low, medium, high) and Service indicator (see further under "17 Hand Control Plus").

5. Raising/Lowering the Frame - Gas

Release the gas strut by moving the handle upward (the gas strut will raise the Walker). Let go of the handle at the desired height (the gas strut is locked).

Release the gas strut by moving the handle upward. Use your body weight to press the frame down. Let go of the handle at the desired height (the gas strut will lock).

6. Adjusting the Handles

To adjust handles to the desired position; undo knob, adjust handle, tighten knob.

7. Left and Right Adjustment of Armrests

The armrests can be adjusted sideways for optimal support; undo knob, adjust armrest, tighten knob.

8. Locking/Unlocking Castors

Depress the foot brake to set it and lock the castor.
Press down on the protruding part of the brake to release it.

9. Control Box Wiring

Remove the control box cover to make the connections as described below. Use a screwdriver or similar; refer to the special labels on the control box.

NOTE: Make sure that plugs are connected to the correct sockets otherwise the actuators can be damaged.

1. Socket for the raise/lower actuator.
2. Hand control socket.
3. Battery terminal socket.
4. Plug socket. NOTE: The plug must ALWAYS be connected to the control box for IP protection to be valid.

10. Battery Charging

- Plug needs to connect to 230V wall socket. Charging starts automatically.
 - LED / battery flashes green.
- The battery is of the LiOn type and insensitive to how long / how often you charge the battery. Full charge on discharged battery takes about 24 hours.
- We recommend that the battery plug be removed from the control box if the product will be out of use for an extended period (more than one week) or if the battery is switched off by means of the ON/OFF switch. This eliminates the risk of disabling the battery as the control box always uses a small quantity of electricity even when the Walker is not in use.

11. LEDs on Control Box and Battery

Control box – green light when power is available from battery/mains socket.

Battery:

- The LED flashes green during charging.
- ON/OFF: To use the product, press ON.
- The green lamp will begin to flash.
- Use the OFF button (hold down for 5 s) to switch off the battery if the product will not be used for an extended period.
- The LED will shows a constant green light when the battery is fully charged.
- When the battery is in normal state/in operation, the LED flashes green at REGULAR intervals, namely 0.5 s ON, 4 s OFF.
- When the battery requires charging (remaining charge <25%) the LED will flash orange/red at REGULAR intervals, namely 0.5 s ON, 4 s OFF.

NOTE: The battery has an in-built device that automatically switches it off when remaining charge is <20%. The LED is then extinguished. The battery must now be fully charged before the electronics can be used again.

12. Maintenance

Check that everything is clean and working before using.

Recommended at least once per year

Check that	Action
<p>General The Walker feels firm/free from play. The Walker does not rattle during manoeuvres. The Walker is level and all of its castors are in contact with the floor. The Walker is not dirty.</p>	<p>Clean with lukewarm soap solution or alcohol-based cleaning agent (no petroleum products).</p>
<p>Armrest Armrests are intact and clean. Armrest widening is functional.</p>	<p>Clean; replace if damaged. Fit lock screws/plates as required.</p>
<p>Handle Grips are not damaged/dirty. Handle adjustment is functional.</p>	<p>Clean with lukewarm soap solution (no petroleum products). Replace handle grips. Fit lock screws/plates as required.</p>
<p>Battery Check that the charging cable is connected to the control box. Check that the hand control is connected to the control box. Check that connections are made according to the wiring diagram. Check that the battery, actuators and control box are not loose.</p>	<p>Fit new charging cable; this must always be connected to the control box (IP65 protection). Connect or fit new hand control as required. Refer to the wiring diagram in the control box wiring section. Tighten; replace with new fasteners as required.</p>
<p>Height adjustment Raising and lowering are functional. The Walker feels stable at maximum settings. There is no play between the vertical frame and bottom frame. Height adjustment clamps lock. The electric actuator fitting must be free from play. Height adjustment via hand control is functional. The electric actuator must run smoothly at a constant speed.</p>	<p>Fit new guide sleeves (in frame) or end plugs (in chrome tubes) as required. Tighten all lock bolts. Replace clamps. Tighten the attachment fitting concerned; replace bolts/lock nuts as necessary. Check that the battery/hand control/ actuator are connected in accordance as in the control box wiring section. Charge the battery. Contact customer services. Refer to the wiring diagram in the control box wiring section.</p>
<p>Frame parts There is no mechanical damage. There are no scratches. End plugs/lower frame fitted</p>	<p>If damage is present, contact DHG customer services. Touch up as necessary. Fit new end plugs.</p>
<p>Castors/brakes Castors roll easily/tread not damaged. The castors are firmly fastened to the lower frame. Castor brakes functional on all castors. Handbrake functional.</p>	<p>Clean or replace castors. NOTE: Castors are always fixed to the lower frame with thread locker or lock nuts. Do not dismantle the castors; replace the whole castor assembly instead. Tighten the castor bolt and fit locking nut or use thread locker (depends on model). Replace castor assembly. Adjust the brake or fit new castors.</p>

DHG can provide upon request necessary technical information in order to assist maintenance and repair of the device.

13. General Maintenance Advice & Cleaning of Walker

- Clean the walker armrests and handles after every use. Wipe using a mild soap solution/surface disinfectant. However, the product is not corrosion proof.
- With environments with high risk of infections use the walker with PU-cushions (Accessory number 57-361-L and 57-361-R)
- Replace damaged parts immediately; must be done by experienced personnel.
- The product is only intended for indoor use.

14. General Maintenance for gait training Sling:


Read the product label.

Do not use rinsing agent.
To prolong product life,
avoid tumble-drying.


15. Storage and transportation

If the walker is not to be used for some time or e.g., during transport, we recommend that the emergency stop button be pressed in. The walker should be transported and stored in $-10\text{ }^{\circ}\text{C}$ to $+50\text{ }^{\circ}\text{C}$ and in normal humidity, 20% to 80% non-condensing. The air pressure should be between 700 and 1060 hPa. Leftmost symbol indicates storage and transportation. Let the walker reach room temperature before the batteries are charged or the walker is used. The walker should not be stored so that it is exposed to dust, or so that the battery or gas piston is exposed to direct sunlight.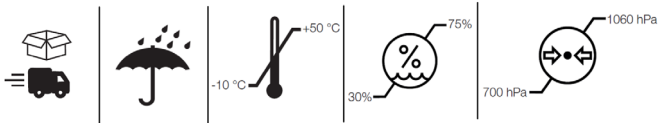

Operation

The operating environment should be $5\text{ }^{\circ}\text{C}$ to $40\text{ }^{\circ}\text{C}$, relative humidity 20% to 80% non-condensing, and atmospheric pressure 700 to 1060 hPa. See also Technical Information below. Leftmost symbol indicates operating condition.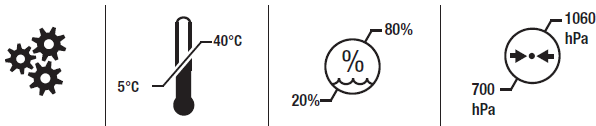

16. Technical Specifications

Dimensions apply to Walkers with 125 mm castors.


Part No.	Product	Product width	Max user weight	Length	Height	Armrest/ internal dimension
56-311	Standard GAS	78cm	150kg	80cm	90-129cm	25-50cm
56-301	S GAS	66cm	150kg	79cm	90-129cm	30-45cm
56-322	Low GAS	78cm	150kg	80cm	75-95cm	25-50cm
56-322-S	Low S GAS	66cm	150kg	79cm	75-95cm	30-45cm

Product weight:

Manual 15 kg, Gas 15 kg, Electric 21 kg

"LIFT" denotes suitable handhold for lifting the Walker.

NOTE: At least two people are required for lifting.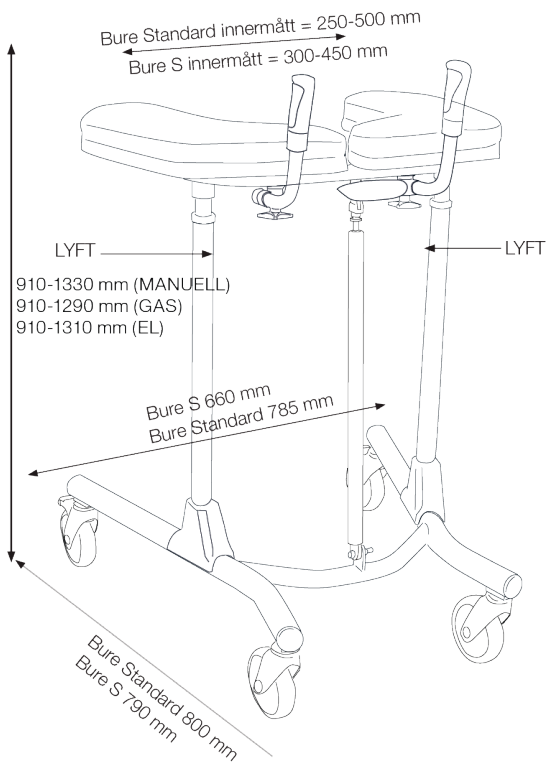

17. Recycling/Disposal

When the product is no longer usable it should be recycled in accordance with legislation and regulations in the country concerned. All electrical parts including batteries must be removed and recycled as electrical components. (Ask your local recycling station for further information about how the different materials in the product (metals, plastics, electronics) should be recycled).

18. Warranty

For further information or enquiries please contact:

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info.se@directhealthcaregroup.com

www.directhealthcaregroup.com

19. Electrical System - Technical Data

Actuator	Lift
Producer	TIMOTION
Voltage	12V
Push	2000N/1500N
Pull	2000N/1500N
Self-locking push	1000N
Current	6.6 (+/-1,0)A
Colour	Black
Stroke	400mm
Mounting dim	605mm
Encapsulation	IP66

IEC60601-1 3rd revision

Control box + Battery

Producer	TIMOTION
Voltage	24V
Capacity	2.0AH
Encapsulation	IP66
Colour	Black
Electrical safety	Class II



IEC60601-1 3rd revision


21. Expected lifetime:

The life expectancy of the product is 8 years.

22. Parts List

1	A	Armrest	6	M	Protective Cover	
	B	Knob		N		Bolts
	C	Plastic Washer				
	D	Washer, Black				
2	D	Handle & Grip	7	O	Wheel frame	
	E	Clamp		P	End bushing	
	F	Knob		Q	Wheel	
				R	Bolt	
				S	Washer	
				T	Nut	
		U	Serrated washer			
3		Upper frame	8		Locking handle, height adjustment	
4	G	Chromium-plated tube	9	A	Gas strut	
	H	Attachment bolt (2 pcs)		B	Handle	
	I	Serrated washer		C	Arm/housing	
	J	End bushing		D	Bolt, washer (2 pcs), nut (attachment at bottom and top)	
5	K	Frame	10	A	Battery shelf & attachment bolts	
	L	Sleeve guide		B	Battery & nut/ bolt (for attachment to battery shelf)	
				C	Control box TIMOTION	
				D	Hand control TIMOTION	
				E	Actuator TIMOTION	

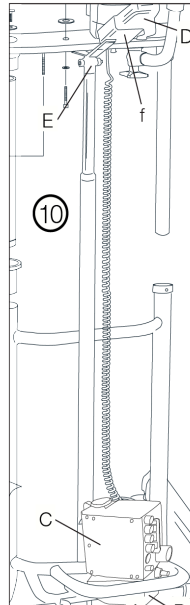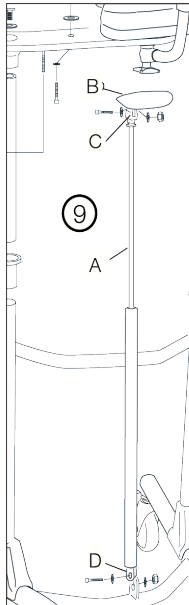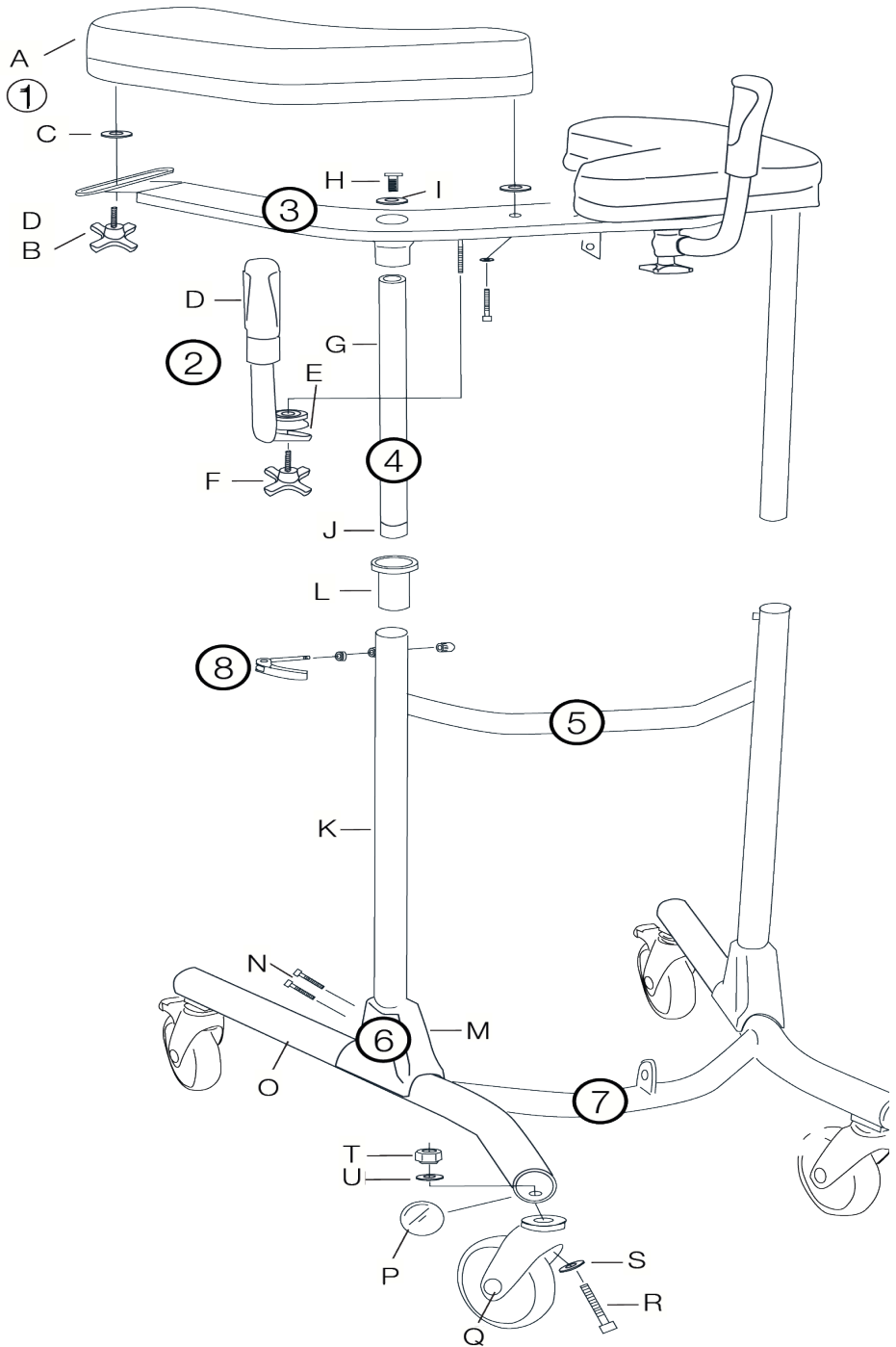23. Spare Parts

Nr.	Part no.	Description
-	56-305	Touch-up paint Bure, Silver 8
10F	56-328	Holder, hand controller (EL only)
L	56-340	Guide sleeve
G	56-341	Standard guide tube/chrome
9B/C	56-342	Plastic handle Bure GAS (GAS only)
J	56-343	End piece, guide tube
P	56-344	End piece, lower frame, Black/D45
10B	56-389	TiMOTION - Battery
10E	58-377	TiMOTION- Actuator (raise/lower)
10C	58-373	TiMOTION- Control box 220 V
10A	56-351	Battery shelf Std (EL only)
O	57-300	Lower frame, Bure Std
O	57-300-2	Lower frame, Bure S
2	56-353	Handle kit, Bure Std (2 pairs incl. lock knob/clamps)
-	56-357	Wheel without brake 100 mm
-	56-358	Wheel with brake 100 mm
-	56-360	Wheel without brake 125 mm
-	56-387	Wheel with brake 75 mm
Q	56-361	Wheel with brake 125 mm
8	56-362	Lock lever, manual height adjustment
8	56-362-1	Lock lever, clamp, bolt for manual height adjustment
2F	56-363	Knob for adjusting handle
1B	56-364	Knob for adj. width of armrest plates
1A	56-366	Armrest, right, Bure Standard/S
1A	56-367	Armrest, left, Bure Standard/S
10D	58-374-X1	TiMOTION- Hand control (raise/lower)
M	56-375	Protective cover, plastic, Bure Standard
-	58-326	TiMOTION- Coiled cable
1C	57-792	Washer, nylon
1D	57-918	Washer, black

24. Accessories

Part no.	Description
56-339-1-0	Handbrake kit/2 handles
56-339-1-1	Handbrake kit, one-handed brake
56-327	Frame elevation kit 70 mm
56-330	Gas strut kit 150N incl. attachment/lever
56-330-LOW	Gas strut kit 150N incl. attachment/lever for Bure Low
56-368	Gas strut 150N (gas strut only)
56-381	Gas strut kit 200N incl. attachment/lever
56-331	Stirrup
56-334	Drip holder
56-335	Attachment for drip holder
56-336	Oxygen holder
56-337	Side support, adjustable
56-338	Drip rod complete with attachment
56-389	Soft basket with attachment, Bure Standard/S
56-388-KIT	Kit/Gait training

25. Table of Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code
	Type BF, according to the degree of protection against electric shock		The device is intended for indoor use
	WEEE Symbol May not be discarded in domestic waste		Class II Equipment
 ON 2min OFF 18min	Duty Cycle: 2 min in active (ON) mode. 18 min in rest (OFF) mode.		Wash at 75 degrees
	Do not iron		Do not dry clean
	Do not bleach		

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Electromagnetic Compatibility (EMC)

Electromagnetic emissions and test levels

The product is intended to be used in the stated environments with electromagnetic levels as specified below. The caregiver and/or user of the product assure that the product is used in such an environment.

Emission test	Standards	Compliance
RF emissions	CISPR 11	Group 1
RF emissions	CISPR 11	Class B
Harmonic current emissions	IEC 61000-3-2	Class A
Voltage fluctuations and Flicker emissions	IEC 61000-3-3	Complies

Electromagnetic Immunity

Acceptance criteria for the EMC pass criteria is unintentional movement above $\pm 10\text{mm}$ is not allowed. The product is intended to be used in the stated environments with electromagnetic levels as specified below. The caregiver and/or user of the product assure that the product is used in such an environment.

Acceptance criteria for the EMC pass criteria is unintentional movement above $\pm 10\text{mm}$ is not allowed. The product is intended to be used in the stated environments with electromagnetic levels as specified below. The caregiver and/or user of the product assure that the product is used in such an environment.

Radiated Fields in close proximity, Immunity test and compliance. Standard: IEC 61000-4-39			
Dwell time: 3 sec			
Test levels (A/m)	Modulation Pulse modulation	Mod. Frequency (kHz)	Test Frequency
8	CW	CW	30 kHz
65	50% Duty cycle	2.1	134.2 kHz
7.5	50% Duty cycle	50	13.56 MHz

Immunity test	Basic EMC standard or test method	IEC 60601-1-2 Edition 4 Test levels and compliance	
		Professional healthcare facility environment†	Home healthcare environment
Surge	IEC 61000-4-5	± 0.5 kV, ± 1 kV	
Voltage dips	IEC 61000-4-11	0% U _T ; 0.5 cycle At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°	
		0% U _T ; 1 cycle and 70% U _T ; 25/30 cycles. Single phase: at 0°	
Voltage Interruptions	IEC 61000-4-11	0% U _T ; 250 / 300 cycle	
Conducted disturbances induced by RF fields	IEC 61000-4-6	6V in ISM bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz	6V in ISM and amateur bands between 0.15 MHz and 80 MHz 80% AM at 1 kHz
		Dwell time: 3 sec Frequency step size: 1%	
Radiated RF Electromagnetic fields	IEC 61000-4-3	3 V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz	10 V/m 80 MHz – 2.7 GHz 80% AM at 1 kHz
		Dwell time: 3 sec Frequency step size: 1%	
Electrical fast transients / burst	IEC 61000-4-4	± 2 kV 100 kHz repetition frequency	
RATED Power frequency magnetic field	IEC 61000-4-8	30 A/m 50 Hz or 60Hz	
Electrostatic Discharge (ESD)	IEC 61000-4-2	Contact: ± 8 kV Air: ± 2 kV, ±4 kV, ± 8 kV, ± 15 kV Number of discharges: 10 for each polarity	

Radiated Radio – Frequency, Proximity fields from wireless communication equipment & compliance. (Dwell time: 3 sec.)

Standard: IEC 61000-4-3

	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Immunity test levels (V/m)
Spot Frequencies	385	380 to 390	TETRA 400	Pulse modulation 18 Hz	27
	450	430 to 470	FMRS 460, FRS 460	FM ± 5 kHz Deviation 1 kHz sine	28
	710	704 to 787	LTE Band 13, 17	Pulse modulation 217 Hz	9
	745				
	780				
	810	800 to 960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz	28
	870				
	930				
	1720	1700 to 1990	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 217 Hz	28
	1845				
	1970				
	2450	2400 to 2570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz	28
	5240	5100 to 5800	WLAN 802.11 a/n	Pulse modulation 217 Hz	9
	5500				
5785					
Other Identified frequencies	433	-	-	Pulse modulation 2 Hz	3
	-	860-960	-	Pulse modulation 2 Hz	54
	2450	-	-	Pulse modulation 2 Hz	54

SystemRoMedic™

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Type BF Applied Parts

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.