Bure Standard/S Bure Low/S

SystemRoMedic®

Instructions for use – English

Bruksanvisning – Svenska

SWL: 150 kg / 330 lbs

Table of contents

English	3
Svenska	17

Product Overview

REF

56-310	56-301	56-302
56-310-DIR	56-301-B/0	56-302-AUS
56-310-KD	56-301-B/1	56-302-US
56-310/100	56-301-PU	56-302-W
56-310/75	56-301/100	56-302T/75-PU-AUS
56-310/75-B/0	56-301/100-RS	56-302/100
56-310-B/0	56-301/75	56-302/100-US
56-310-B/1	56-301/75-PU	56-302T
56-311-PARSONS-KD	56-312/100	56-302T-FKN_R
56-311	56-312/100-UK	56-302T/100
56-311-KD	56-312/100-US	56-302T/75
56-311-PARSONS-KD	56-312/75	56-302T/75-FKN_R
56-311/100	56-312/75-AUS	56-302T/75-PU
56-311/100-PU	56-312L-AUS	56-302T-PU
56-311/75	56-312L-KD-AUS	56-302T-TK
56-311/75-B/0	56-312L-UK	
56-311/75-B/1	56-312T	
56-311/75-PU	56-312T-B/0-PU	
56-311/75-RS-KD	56-312T-BRAKE/0	
56-311-B/1	56-312T-B/0-TKN_R	
56-311-BRAKE/1-RED	56-312T-PU	
56-311-BRAKE/0-PU	56-312T-TK	
56-311-PU	56-312T-US	
56-311-RED	56-312T/75	
56-311-RS	56-312T/75-PLUS-PU	
56-312	56-312T/75-PU	
56-312-AUS	56-312T/100	
56-312-CAN	56-321	
56-312-CAN-KD	56-321/75	
56-312-KD	56-321/75-PU	
56-312-TK	56-321L-AUS	
56-312-UK	56-321L-UK	
56-312-UK-KD	56-321-S	
56-312-US	56-321-S/100	
56-300	56-321-S/75	
56-300/100	56-321-S/75-PU	
56-300/75	56-321-S/75-UK	
56-300/75-DIR	56-322	
56-300/75-B/0	56-322/75	
56-300/75-B/0-PU	56-322-S	
56-300/75-B/1-PU	56-322-S-B/0	
56-300-B/0	56-322-S/75	
56-300-B/1	56-301/75-RS	

Bure Standard/S

Bure Low/S

SystemRoMedic

Instructions for use - English

1. Intended use

Walkers are indoor, movable and portable assistive aids, which together with approved accessories are intended to be used for transfer from sitting to standing positions, and walking training. Walkers are to be used by persons with reduced muscular strength, who shall be able to stand and perform simple leg movements. Walkers are to be used for leaning on when walking and standing during early mobilization training. Walkers and accessories are intended for indoor use only in Home Healthcare environment and Professional Healthcare facility environment.

2. Contraindications, Precautions and Warnings

Contraindications
The device may not be used by patients above the maximum weight indicated on the label of the device.
Precautions
Check that the device is used on a dry and flat surface.
Check the device is correctly mounted/assembled before its first use.
Check the device after every folding/disassembly after any transport.
Check height adjustment and leg spreading motions and inspect the actuators' or gas pistons' full ranges.
Activate the brakes whenever the device is not in use.
Check the function of the brakes before use.
Inspect the device regularly to detect any signs of damage. If there are any visible signs of damage, contact Direct Healthcare Group Customer Service.
In case of an electrical model of the device, check that the hand control does not show signs of wear before every use.
Check that the hand control markings are in accordance with the lifting functions.
Check the battery status of an electrically operated model, before using the device.
It is important to never leave the user alone during the transfer.
Warranty applies only if repairs or alterations are made by personnel who are authorized by Direct Healthcare Group.
Ensure that there are no obstacles or people in the way of the device, when moving.
Handle batteries with care. Do not drop.
Use only batteries and cables that are intended for the device as instructed by the manufacturer.
Verify that potential accessories that might be used are properly attached to the device before use.
Low speed is recommended when moving with the device.
Take care not to drive the device over thresholds with high speed or force. Approach the threshold with lower speed and communicate the coming threshold to the user.
Keep the instructions for use together with the product so that it can be used as a reference if necessary.

Warnings
The caregiver must be able to read and understand the Manual/IFU of the product.
It is important to use only approved accessories to prevent unintended detachment of components and subsequently a fall that may lead to patient injury.
Use careful and gentle maneuvers when moving the device.
Perform maintenance/service of the device, according to the instructions in the manual/IFU, at least once every 12 months.
Accessories must be properly fitted and tested in relation to the user's needs and functional ability.
Special care must be taken when using strong electrical power sources such as diathermy and the like so that diathermy cables are not placed on or near the device. In case of doubt consult with a Direct Healthcare Group representative.
When lifting the device, only use the denoted handholds. At least two people are required for lifting the device.
Do not leave a care recipient unattended during a transfer.
Never move the lift by pulling on the actuator or gas piston.
The device must not be lowered into water.
The device must not be cleaned using steam.
The device must not be left or stored in a damp or humid environment.
The device must not be charged in a wet room.
The device must not be used outdoors, only indoors.

3. Raising/Lowering the Frame - Electric

The desired height is adjusted with Arrow <UP> or Arrow <DOWN>.

On products with Linak it is possible to select normal or high speed (single or double arrow).

Handset from Linak and TiMOTION Plus also have charge indicator / battery (low, medium, high) and Service indicator (see further under "17 Hand Control Plus").

Linak

TiMOTION

4. Raising/Lowering the Frame - Gas

Release the gas strut by moving the handle upward (the gas strut will raise the Walker). Let go of the handle at the desired height (the gas strut is locked).

Release the gas strut by moving the handle upward. Use your body weight to press the frame down. Let go of the handle at the desired height (the gas strut will lock).

5. Adjusting the Handles

To adjust handles to the desired position; undo knob, adjust handle, tighten knob.

6. Left and Right Adjustment of Armrests

The armrests can be adjusted sideways for optimal support; undo knob, adjust armrest, tighten knob.

7. Locking/Unlocking Castors

Depress the foot brake to set it and lock the castor.

Press down on the protruding part of the brake to release it.

8. Control Box Wiring

TIMOTION

Remove the control box cover to make the connections as described below. Use a screwdriver or similar; refer to the special labels on the control box.

NOTE: Make sure that plugs are connected to the correct sockets otherwise the actuators can be damaged.

1. Socket for the raise/lower actuator.
2. Hand control socket.
3. Battery terminal socket.
4. Plug socket. NOTE: The plug must ALWAYS be connected to the control box for IP protection to be valid.


TIMOTION

Linak

The door on the control box can be opened with a thumb grip.

1. Hand control socket.
2. Battery outlet.
3. Socket for actuator up / down.
4. Socket for plug.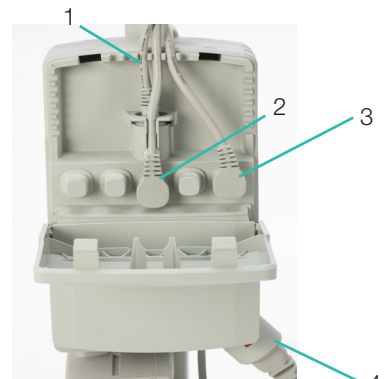

Linak

9. Battery Charging

- Plug needs to connect to 230V wall socket. Charging starts automatically.
 - o TIMOTION: LED / battery flashes green.
 - o Linak: Full level battery indicator flashes yellow until the battery is fully charged.
- The battery is of the LiOn type and insensitive to how long / how often you charge the battery. Full charge on discharged battery takes about 24 hours.
- TIMOTION: We recommend that the battery plug be removed from the control box if the product will be out of use for an extended period (more than one week) or if the battery is switched off by means of the ON/OFF switch. This eliminates the risk of disabling the battery as the control box always uses a small quantity of electricity even when the Walker is not in use.

10. LEDs on Control Box and Battery

TIMOTION

Control box – green light when power is available from battery/mains socket.

Battery:

- The LED flashes green during charging.
- ON/OFF: To use the product, press ON.
- The green lamp will begin to flash.
- Use the OFF button (hold down for 5 s) to switch off the battery if the product will not be used for an extended period.
- The LED will show a constant green light when the battery is fully charged.
- When the battery is in normal state/in operation, the LED flashes green at REGULAR intervals, namely 0.5 s ON, 4 s OFF.
- When the battery requires charging (remaining charge <25%) the LED will flash orange/red at REGULAR intervals, namely 0.5 s ON, 4 s OFF.

NOTE: The battery has an in-built device that automatically switches it off when remaining charge is <20%. The LED is then extinguished. The battery must now be fully charged before the electronics can be used again.

Linak

Control box - LED lights up yellow during operation, is off when the table is not in use.

When the control box is connected to the mains, the LED lights up green.

When driving / connected to the mains, the LED lights up yellow.

Battery:

- LED lights YELLOW during charging (at the same time as the charging indicator on the hand control flashes yellow).
- The LED goes out when the battery is fully charged (the same applies to the charge indicator on the hand control).

11. Maintenance

Recommended at least once per year

Check that	Action
General The Walker feels firm/free from play. The Walker does not rattle during manoeuvres. The Walker is level and all of its castors are in contact with the floor. The Walker is not dirty.	Clean with lukewarm soap solution or alcohol-based cleaning agent (no petroleum products).
Armrest Armrests are intact and clean. Armrest widening is functional.	Clean; replace if damaged. Fit lock screws/plates as required.
Handle Grips are not damaged/dirty. Handle adjustment is functional.	Clean with lukewarm soap solution (no petroleum products). Replace handle grips. Fit lock screws/plates as required.
Battery Check that the charging cable is connected to the control box. Check that the hand control is connected to the control box. Check that connections are made according to the wiring diagram. Check that the battery, actuators and control box are not loose.	Fit new charging cable; this must always be connected to the control box (IP65 protection). Connect or fit new hand control as required. Refer to the wiring diagram in the Technical manual. Tighten; replace with new fasteners as required.
Height adjustment Raising and lowering are functional. The Walker feels stable at maximum settings. There is no play between the vertical frame and bottom frame. Height adjustment clamps lock. The electric actuator fitting must be free from play. Height adjustment via hand control is functional. The electric actuator must run smoothly at a constant speed.	Fit new guide sleeves (in frame) or end plugs (in chrome tubes) as required. Tighten all lock bolts. Replace clamps. Tighten the attachment fitting concerned; replace bolts/lock nuts as necessary. Check that the battery/hand control/ actuator are connected in accordance with the wiring diagram. Charge the battery. Contact customer services.
Frame parts There is no mechanical damage. There are no scratches. End plugs/lower frame fitted	If damage is present, contact DHG customer services. Touch up as necessary. Fit new end plugs.
Castors/brakes Castors roll easily/tread not damaged. The castors are firmly fastened to the lower frame. Castor brakes functional on all castors. Handbrake functional.	Clean or replace castors. NOTE: Castors are always fixed to the lower frame with thread locker or lock nuts. Do not dismantle the castors; replace the whole castor assembly instead. Tighten the castor bolt and fit locking nut or use thread locker (depends on model). Replace castor assembly. Adjust the brake or fit new castors.

12. General Maintenance Advice & Cleaning

- Clean the Walker and its armrests clean as necessary - Wipe using a soap solution/surface disinfectant.
- The product may be cleaned in a tunnel washer or similar as electrical systems have IP65 protection. However, the product is not corrosion proof which means washing by machine may reduce its lifetime.
- The life expectancy of the product is 8 years.
- If the product is to be cleaned in a washing tunnel / washing cabin, use PU-cushions.
- Replace damaged parts immediately; must be done by experienced personnel.
- The product is only intended for indoor use.

13. Technical Specifications

Dimensions apply to Walkers with 125 mm castors.


Part No.	Product	Product width	Max user weight	Length	Height	Armrest/ internal dimension
56-311	Standard GAS	78cm	150kg	80cm	90-129cm	25-50cm
56-301	S GAS	66cm	150kg	79cm	90-129cm	30-45cm
56-322	Low GAS	78cm	150kg	80cm	75-95cm	25-50cm
56-322-S	Low S GAS	66cm	150kg	79cm	75-95cm	30-45cm
56-312T/56-312L	Standard EL	78cm	150kg	80cm	90-130cm	25-50cm
56-302T/56-302L	S EL	66cm	150kg	79cm	90-130cm	30-45cm
56-321/56-321L	Low EL	78cm	150kg	80cm	75-100cm	25-50cm
56-321-S/56-321L-S	Low S EL	66cm	150kg	79cm	75-100cm	30-45cm

Product weight:

Manual 15 kg, Gas 15 kg, Electric 21 kg

"LIFT" denotes suitable handhold for lifting the Walker.

NOTE: At least two people are required for lifting.

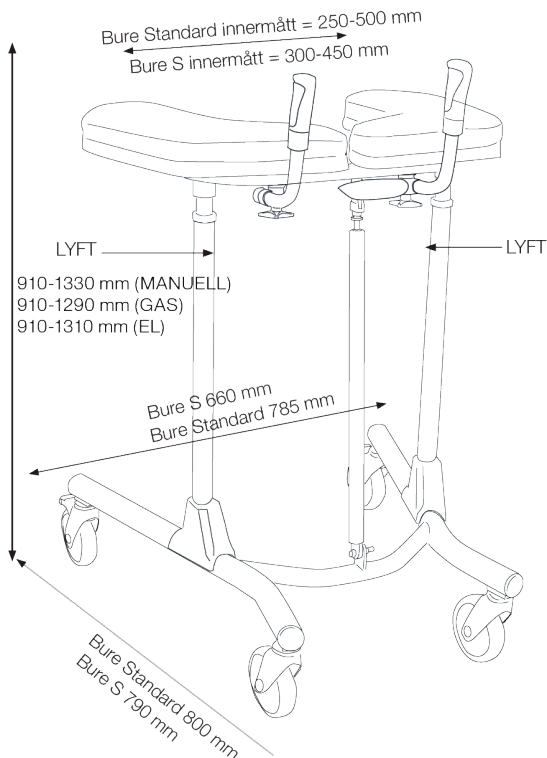14. Recycling/Disposal

When the product is no longer usable it should be recycled in accordance with legislation and regulations in the country concerned. All electrical parts including batteries must be removed and recycled as electrical components. (Ask your local recycling station for further information about how the different materials in the product (metals, plastics, electronics) should be recycled).

15. Warranty

We provide two years' warranty for manufacturing faults (does not apply to wear parts).

For further information or enquiries please contact:

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info.se@directhealthcaregroup.com

www.directhealthcaregroup.com

16. Electrical System - Technical Data (TiMOTION/Linak)

Actuator	Lift	Widening	Lift	Widening
Producer	TiMOTION	TiMOTION	Linak	Linak
Voltage	12V	24V	24V	24V
Push	2000N/1500N	1000N	1000N/1500N/ 2000N	1200N
Pull	2000N/1500N	1000N	1000N/1500N/ 2000N	1200N
Self-locking push	1000N	1000N	7A	3A
Current	6.6 (+/-1,0)A	2.6 (+/-0,4)A	7A ut/7 A in	3A ut/3 A in
Colour	Black	Aluminium	White	White
Stroke	400mm	55mm	400mm	55mm
Mounting dim	605mm	167mm	606 +/-2 mm	165 +/-2 mm
Encapsulation	IP66	UIP	IPX6	IPX6

IEC60601-1 3rd revision

Control box + Battery

Producer	TiMOTION	Linak
Voltage	24V	230V/24V
Capacity	2.0AH	2,25 ah
Encapsulation	IP66	IPx6
Colour	Black	White
Electrical safety	Class II	Class II (double insulated)

IEC60601-1 3rd revision

17. Hand Control Plus

TI-MOTION

**1. Service indicator – flashes red after 11 months' operation.

Constant light (red) after 12 months' operation.

The indicator is a reminder to review the maintenance instruction (see Maintenance Instructions). This should be done at least once every 12 months.

Reset: hold down buttons <2a> + <2b> for at least 30 seconds.

**The indicator will go out around 30 seconds after the hand control is activated (battery conservation function).

Linak

The service indicator lights up solid yellow after about 12 months of use. The indicator is a reminder to go through the Maintenance instruction (should be done once every 12 months).

Reset: Simultaneously hold down the buttons <Fast speed up> + <Fast speed down> for about 10 seconds until the control box makes a beeping sound as an acknowledgment that the reset has been done.



The service indicator flashes yellow when the actuator has been overloaded or if another critical fault has occurred.


Reset: Simultaneously hold down the <Low Speed Up> + <Low Speed Down> buttons for about 5 seconds until the Service indicator goes out.

18. Parts List

1	A B C D	Armrest Knob Plastic Washer Washer, Black	6	M N	Protective Cover Bolts
2	D E F	Handle & Grip Clamp Knob	7	O P Q R S T U	Wheel frame End bushing Wheel Bolt Washer Nut Serrated washer
3		Upper frame	8		Locking handle, height adjustment
4	G H I J	Chromium-plated tube Attachment bolt (2 pcs) Serrated washer End bushing	9	A B C D	Gas strut Handle Arm/housing Bolt, washer (2 pcs), nut (attachment at bottom and top)
5	K L	Frame Sleeve guide	10	A B C D E	Battery shelf & attachment bolts Battery & nut/ bolt (for attachment to battery shelf) Control box (Linak/TIMOTION) Hand control (Linak/TIMOTION) Actuator (Linak/TIMOTION)

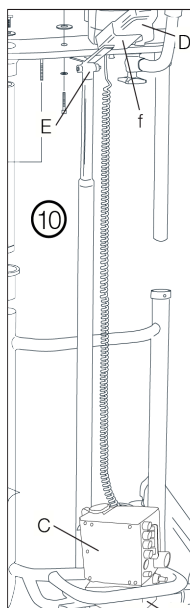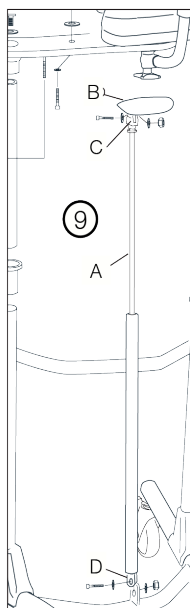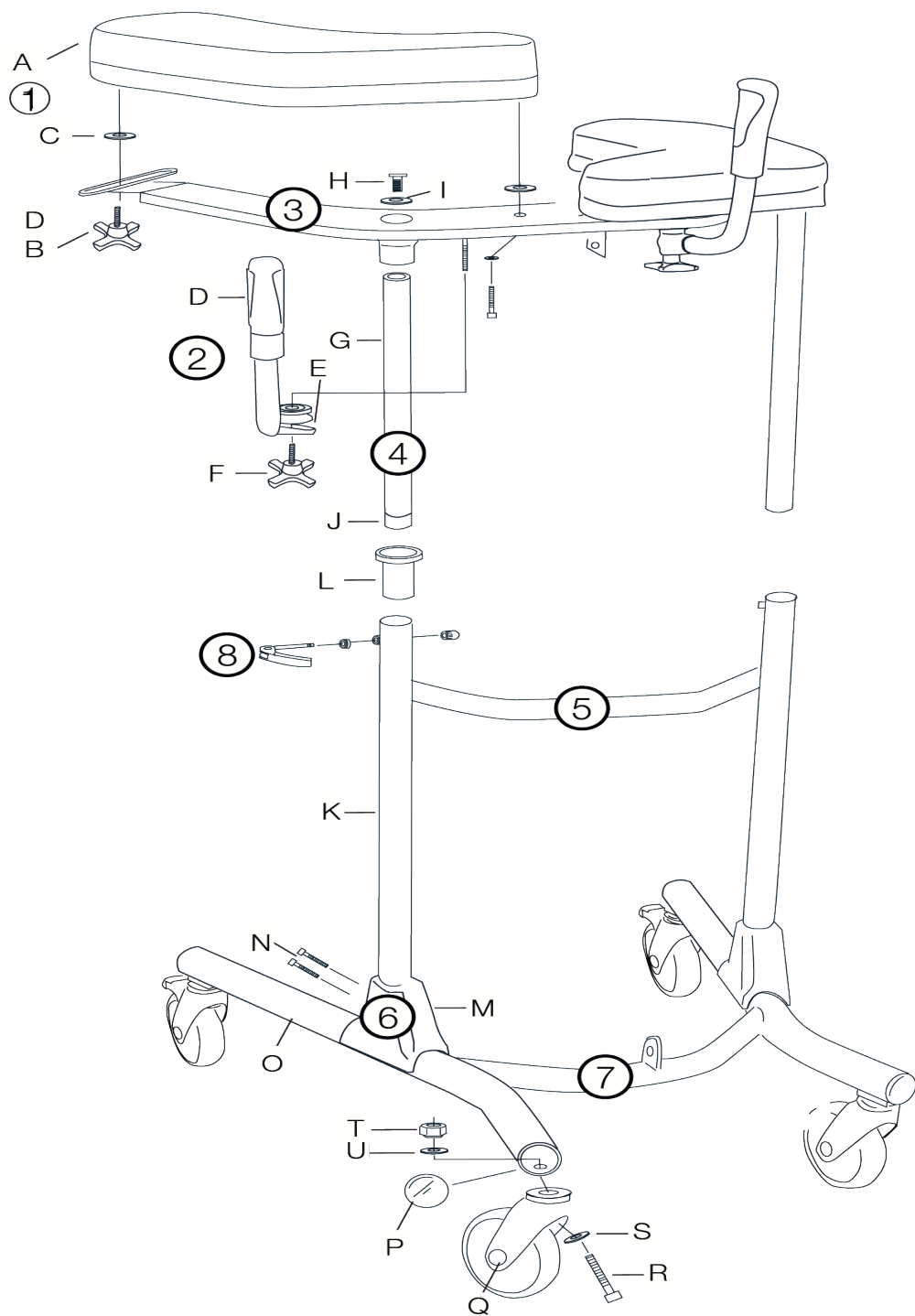19. Spare Parts

-	56-305	Touch-up paint Bure, Silver 8
10F	56-328	Holder, hand controller (EL only)
L	56-340	Guide sleeve
G	56-341	Standard guide tube/chrome
9B/C	56-342	Plastic handle Bure GAS (GAS only)
J	56-343	End piece, guide tube
P	56-344	End piece, lower frame, Black/D45
10B	56-402	Linak - Battery (LiOn)
10B	56-389	TiMOTION - Battery
10E	56-398	Linak- Actuator (raise/lower) (EL, LA31 only)
10E	58-377	TiMOTION- Actuator (raise/lower)
10C	56-397	Linak- Control box 220 V (EL, LA31 only)
10C	58-373	TiMOTION- Control box 220 V
10A	56-351	Battery shelf Std (EL only)
O	57-300	Lower frame, Bure Std
O	57-300-2	Lower frame, Bure S
2	56-353	Handle kit, Bure Std (2 pairs incl. lock knob/clamps)
-	56-357	Wheel without brake 100 mm
-	56-358	Wheel with brake 100 mm
-	56-360	Wheel without brake 125 mm
-	56-387	Wheel with brake 75 mm
Q	56-361	Wheel with brake 125 mm
8	56-362	Lock lever, manual height adjustment
8	56-362-1	Lock lever, clamp, bolt for manual height adjustment
2F	56-363	Knob for adjusting handle
1B	56-364	Knob for adj. width of armrest plates
1A	56-366	Armrest, right, Bure Standard/S
1A	56-367	Armrest, left, Bure Standard/S
10D	58-411	Linak- Hand control (raise/lower), (EL, LA31 only)
10D	58-374-X1	TiMOTION- Hand control (raise/lower)
M	56-375	Protective cover, plastic, Bure Standard
-	58-406-EUR/AUS/UK/US	Spiral Cable, gray 300mm, Linak
-	58-326	TiMOTION- Coiled cable
1C	57-792	Washer, nylon
1D	57-918	Washer, black

20. Accessories

Part no.	Description
56-339-1-0	Handbrake kit/2 handles
56-339-1-1	Handbrake kit, one-handed brake
56-327	Frame elevation kit 70 mm
56-330	Gas strut kit 150N incl. attachment/lever
56-330-LOW	Gas strut kit 150N incl. attachment/lever for Bure Low
56-368	Gas strut 150N (gas strut only)
56-381	Gas strut kit 200N incl. attachment/lever
56-331	Stirrup
56-334	Drip holder
56-335	Attachment for drip holder
56-336	Oxygen holder
56-337	Side support, adjustable
56-338	Drip rod complete with attachment
56-389	Soft basket with attachment, Bure Standard/S

21. Table of Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visuell inspection
	Medical device		Read the manual
	Caution		Product code
	Manufacturer information		Batch Code
	Type BF, according to the degree of protection against electric shock		The device is intended for indoor use
	WEEE Symbol May not be discarded in domestic waste		Class II Equipment
	Duty Cycle: 2 min in active (ON) mode. 18 min in rest (OFF) mode.		

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Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.