Protective Covers Disposable Covers

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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Protective Cover

Disposable Cover

REF	
1051	1055
1151	1155
1251	1255
1451	2055
1481	9136
2051	9138
2052	9156
	9168

SWL EasySlide: 300 kg/ 661 lbs
SWL RollerSlide: See Board

Table of contents

English	3
Svenska	4
Norsk	5
Dansk	6
Suomi	7
Deutsch	8
Nederlands	9
Français	10
Italiano	11
Español	12
Český	13
Product information	14
Symbols	15

Protective Covers Disposable Covers

SystemRoMedic

Different protective covers are available for SystemRoMedic's EasySlide and RollerSlide. Protective Cover is a multi-use cover that is made of Ultra Low Friction (ULF) material. Disposable Cover, a single-use cover that is made of plastic, can be used to advantage in e.g., emergency care or on isolation wards. The multi-use protective cover for RollerSlide is also a glide cover, since it enables the slide to glide freely.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

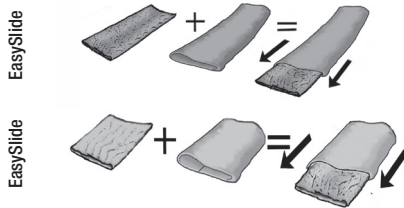
Always read the user manual

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product. Always make sure that you have the right version of the manual.


The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

Placement

Disposable Covers are on convenient rolls and are torn off one by one.

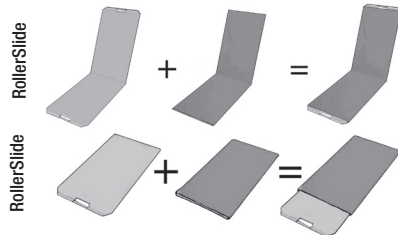Disposable Cover and Protective Cover are pulled over EasySlide or RollerSlide (see images).

When using Disposable Cover on RollerSlide, remember that a glide cover is always needed, since it is what allows the slide to glide freely.

NOTE! Disposable Cover cannot be laundered, but must be discarded after use.

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Expected lifetime

Up to 10 years of normal use

Protective Covers Disposable Covers

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.