Moving Health Forward

EN [User Manual](#)

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
LEJRELET

Pad High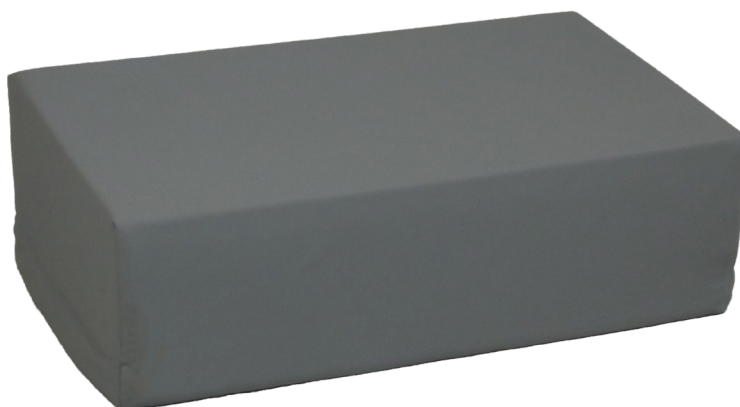

EN

Application


LEJRELET Pad High is a rectangular cushion made from a mixture of polyurethane foam and viscoelastic foam. It is used for various positioning situations and during lower hygiene procedures for the bedridden patient.

LEJRELET Pad High has a large support surface which in combination with viscoelastic foam offers good pressure relief and comfort for the bedridden patient.


Pad High can be used alone or in combination with other LEJRELET products.

Examples for use of LEJRELET Pad High:


- Support of leg in lateral position
- Support of leg during lower hygiene procedures
- Support of arm in lateral position
- Support of legs and pressure relief of heels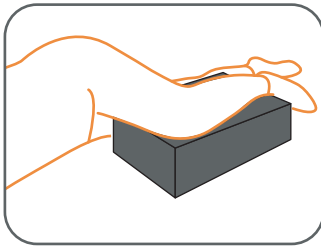


Support of leg in lateral position

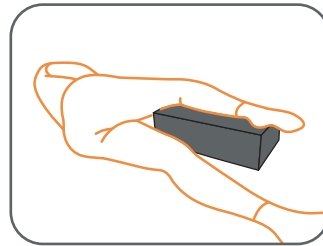Support of leg during lower hygiene situations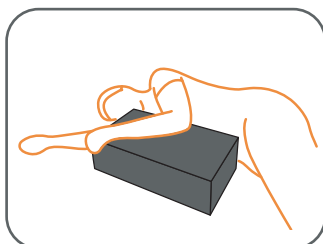

Support of arm in lateral position

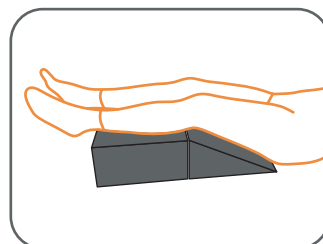Support of legs and pressure relief of heels

Technical Information

Filling:

LEJRELET Pad High is pressure relieving and made with a core of poly-urethane foam covered with 2 cm of flameretardant viscoelastic foam on the top.

This combination means, that the cushions are firm and supportive, yet they are still flexible and provide effective pressure relief and comfort.

Fabric:

- Waterproof
- Water vapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

Size: H 15 x W 30 x L 50

Item no.: 5000003

Weight: 800 g

Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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Direct Healthcare Group Ltd
Direct Healthcare Group Sverige AB,
Direct Healthcare Group Pty Ltd.,