Moving Health Forward

EN [User Manual](#)

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LEJRELET

Tube 125

EN

Application


The LEJRELET Tube 125 has the same structure and positive qualities as the LEJRELET Tube 250. The great flexibility of the material affords many application options.

It is, however, only half the length, i.e. 125 cm long and therefore weighs less.


The LEJRELET Tube 125 is suitable for smaller adults and large children. It is also suitable for leg amputees who need good support for the shortened leg. The LEJRELET Tube 125 can be folded very tightly, thus providing great support for, inter alia, contractures.

Examples for use of LEJRELET Tube 125:


- To support the body in a lateral position
- For 30-degree positioning for leg amputees
- To support legs and arms while seated
- Stable lateral positioning for flexion contractures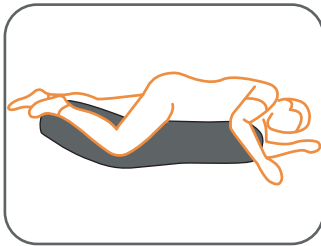


Support of the entire body in lateral position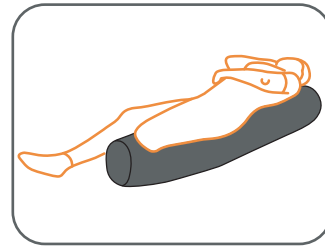

For 30-degree positioning for leg amputees

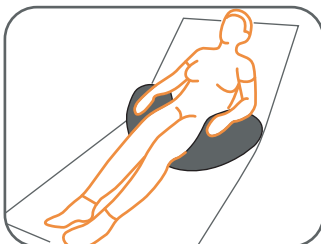To support legs and arms while seated

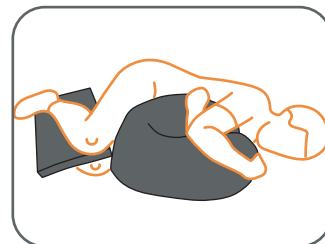Stable lateral positioning for flexion contractures

Technical Information

Filling:

LEJRELET Tube 125 is filled with viscoelastic foam granules preventing decubitus.

Fabric:

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

Size: Ø 25 x L 125

Item no.: 5000000

Weight: 2350 g

Be aware that the patient is positioned such that airways are kept clear.

Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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This Page is Left Intentionally Blank

This Page is Left Intentionally Blank

Moving Health Forward

PRESSURE ULCER PREVENTION
SAFE MOVING & HANDLING
SPECIALIST THERAPIES
BATHROOM SAFETY
RENTAL & SERVICE SOLUTIONS

Direct Healthcare Group Ltd
Withey Court, Western Industrial Estate
Caerphilly, United Kingdom
CF83 1BF

T: +44 (0) 800 043 0881
F: +44 (0) 845 459 9832
E: info@directhealthcaregroup.com

EC **REP**

EU Authorised Representative, Importer, and Distributor:

Direct Healthcare Group Sverige AB,
Torshamnsgatan 35, SE-164 40, Kista
Sweden
T: +46 (0) 8 557 200

Australian Sponsor:

Direct Healthcare Group Pty Ltd.,
68 Howe Street, Osborne Park,
Western Australia 6017
T: +61 (0) 423 852 810

Issue 3
Date: June 2022

DHG-HEALTHCARE.COM