Moving Health Forward

EN [User Manual](#)

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LEJRELET

Tube 250

EN

Application


LEJRELET Tube 250 is a long soft cylinder filled with pressure relieving viscoelastic foam granules.

It is easy to shape and adapt Tube 250 for the individual user and care situation and thus possible to adapt Tube 250 to various situations and conditions.


The viscoelastic foam granules give great adaptability to the body and good support / pressure prevention. Further more, Tube 250 is very comfortable and calming for the bedridden person.

Examples for use of LEJRELET Tube 250:


- Support of the entire body in lateral position
- Positioning in a 30 degree angle
- Pressure relief for scapula and arms in supine position
- Support of legs and pressure relief of heels
- Comfort and calming positioning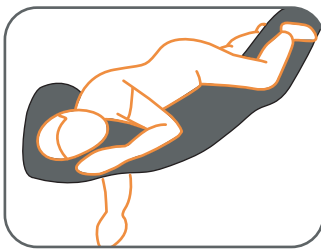


Support of the entire body in lateral position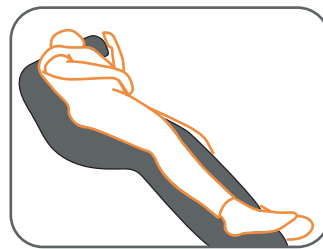

Positioning in a 30 degree angle

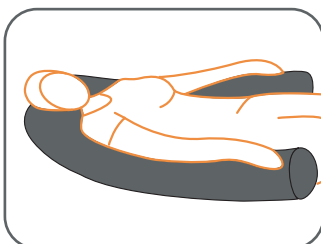Pressure relief for scapula and arms

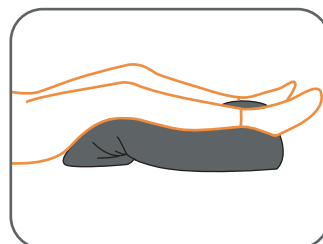Support of legs and pressure relief of heels

Technical Information

Filling:

LEJRELET Tube 250 is filled with viscoelastic foam granules preventing decubitus.

Fabric:

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

Size: Ø 25 x L 250

Item no.: 5000001

Weight: 4700 g

Be aware that the patient is positioned such that airways are kept clear.

Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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Direct Healthcare Group Ltd
Direct Healthcare Group Sverige AB,
Direct Healthcare Group Pty Ltd.,