Moving Health Forward

DK

Manual

VENDLET

V5S

V5S Speed Adjust Bari

Tillykke med Deres nye VENDLET system.

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Dismounting, Storage & Remounting

Dismounting

The VENDLET may be dismantled and transferred to another bed.

1. Remove the turning sheet and the slide sheet.
2. Lower the VENDLET bars.
3. Disconnect the power cable from the outlet.
4. Then the VENDLET can be removed by the same procedure as described in the mounting section on page 33-35, ie:
 - Remove the two strips that hold the cables in place on all four actuators.
 - Dismount the actuator.
 - Strap the VENDLET together so that the side rail can not run up and down.
 - Loosen the bolts in the mounting plates and turn aside to lift VENDLET off the bed frame.

Storage

The VENDLET should be stored in a dry location.

Temperature: -10°C - +50° C

Relative air humidity: 20% - 90% @ 30° C. No condensation.

Atmospheric pressure: 700 - 1060 hPa.

Remounting

Follow the procedure described in this manual if remounting is required.

Before the mounting, you should check all VENDLET parts for damage and that the date of the annual service check has not been exceeded.

Maintenance/Check

Annual check

The VENDLET system must be serviced annually by a person with relevant training. This check will ensure that the safety issues pertaining to the VENDLET system are in compliance with the standard as well as a long service life.

The annual check includes the following:

Mechanical parts

	Check for signs of damage and crooked fittings or brackets and missing parts.
	Check the bars and support bars for dents/deformations.
	Check whether the bar velcro tapes are attached and in good condition.

Sheets

	Is the turning sheet in good condition?
	Is the slide sheet in good condition?

Functional tests

	Do all bolts and nuts match the parts list and are they evenly tightened? Is there a 1 mm clearance between the end of the bar and the slide sleeve of the top fitting?
	Does the mounting meet all side rail dimension requirements?
	Check whether it is possible to raise and lower the bars freely and in parallel. It might be necessary to remount the brackets to ensure proper height and width.
	Check whether the VENDLET works as intended when the hand control is activated.

Electrical parts

Control unit no.:			
Check the control unit including plugs and mounting fittings for signs of damage.			
Check whether the timer deactivates after 90 seconds of idle time.			
Transformer no.:		Hand control no.:	
Check the transformer for signs of damage.		Check the hand control, cables and mounting fixtures for signs of damage.	
Check all cables, plugs, and relief cables for signs of damage.		Check the functioning of all buttons and whether the green diode is illuminated when the system is active.	
Check whether the green diode is illuminated.		Check all plugs, cables, and relief cables.	
When using an extension cable, check whether the cable has the necessary quadrant (1.0 mm2).		Check the stop function.	
Bars	Red side	Blue side	
Check the motors for signs of damage.			
Check all cables, plugs, and relief cables for signs of damage.			
Check whether the bar can be operated manually when the motor is disconnected from the control unit.			
Check whether the motor has full pulling power when you press the  /  and reduced pulling power when you press the  /  by stopping the bar with your hand.			
Actuators			
Check the plastic covers for signs of damage.			
Check all cables, plugs, and relief cables for damage. Check whether the cables are properly secured.			
Check whether the cables are properly secured to the bed frame with cable ties.			
Check the slide function – lubricate, if necessary.			

Troubleshooting

Problem	Checklist
VENDLET does not work	Has the power been turned on?
	Is the green diode on the control unit under the bed illuminated?
	Is the green diode on the hand control illuminated?
	Are all cables properly mounted and intact?
	Replace the hand control.
	Reset the system by simultaneously pressing the two top buttons on the hand control. You will hear 10 rhythmic beeps. Press and hold the buttons until the beeping stops.
The bar starts but stops pulling immediately afterwards	Contact a person with relevant training.
	Does the client weigh more than max. user weight?
	Is part of the mattress or blanket rolled onto the bar with the sheet?
One of the bars does not work.	Is the turning sheet attached correctly?
	Is the cable from the bar intact and correctly mounted?
	Switch the plugs of the two bars. If the same bar still does not work, then the motor of that bar is defective. If the other bar is not working, then the hand control or the control unit is defective. Replace the hand control. If this does not solve the problem, replace the control unit.
	Reset the system by simultaneously pressing the two top buttons on the hand control. You will hear a series of rhythmic beeps. Press and hold the buttons until the beeping stops.
	Replace the bar motor.
One or both bars will not work when a load is applied	Contact a person with relevant training.
	Does the sheet run over the bar, cf. page 24 of the user manual? If the sheet runs under the bar, the motor disconnects when used. This is a safety feature.
	Switch the plugs of the two bars. If the same bar still does not work, then the motor of that bar is defective. If the other bar is not working, then the hand control or the control unit is defective. Replace the hand control. If this does not solve the problem, replace the control unit.
	Replace the bar motor.
	Contact a person with relevant training.
One of the bars will only move in one direction	Is the plug properly connected to the control unit and the actuator?
	Is the cable intact?
	Switch the plugs of the two bars. If the same bar still does not work, then the bar motor is defective. If the other bar is not working, then the hand control or the control unit is defective. Replace the hand control. If this does not solve the problem, replace the control unit.
	Reset the system by simultaneously pressing the two top buttons on the hand control. You will hear a series of rhythmic beeps. Press and hold the buttons until the beeping rhythm changes.
	Replace the bar motor.
	Contact a person with relevant training.
The hand control buttons are not working	Is the plug properly connected to the control unit?
	Is the cable intact?
	If the actuator still does not work, try changing the channel. Replace the actuator if this does not solve the problem.
	Reset the system by simultaneously pressing the two top buttons on the hand control. You will hear a series of rhythmic beeps. Press and hold the buttons until the beeping rhythm changes.
	Replace the hand control.
	Contact a person with relevant training.

One or more actuators will not go up or down	Is the plug properly connected to the control unit?
	Is the cable intact?
	If the actuator still does not work, try changing the channel. Replace the actuator if this does not solve the problem.
	Reset the system by simultaneously pressing the two top buttons on the hand control. You will hear a series of rhythmic beeps. Press and hold the buttons until the beeping rhythm changes.
	Contact a person with relevant training.
One or more actuators / actuator fittings will not move entirely down	Are the actuator fittings perpendicular to the bed frame?
	Have the side rail dimensioning requirements been met, cf. page 31-32? If not, the free movement of the actuator fittings may be affected.
	Dirt, dust etc. may obstruct the sliding function of the actuator fitting. Clean the actuator fitting and lubricate it with silicone spray.


Repairs

Only a limited number of VENDLET system components need regular replacement. On the next pages, we will explain the more general repairs. Please do not hesitate to contact us or our consultants if you need assistance or guidance or to perform repairs or troubleshooting.


All repairs must be carried out by a person with relevant training.

Replacing the motor in a bar


1. Raise the bar and remove the turning sheet from the bar on which the motor is to be replaced.
2. Disconnect the power cable from the mains outlet.
3. Disconnect the motor bar cable from the control unit and remove the bar from one of the fittings (you can also choose to disconnect the bar in both ends).
4. Remove the Allen screw at the motor end of the bar and pull the motor out.
5. Unscrew the small pivot screw on the motor fitting and remove it.
6. Mount the motor fitting on the new motor and attach it using the pivot screw.
7. Place the motor inside the bar and attach it using the Allen screw you removed earlier.
8. Mount the bar on the two actuator fittings and connect the cable to the control unit. Remember the plastic fittings.
9. Reconnect the power cable to the mains outlet.
10. Reset the system by simultaneously pressing the two top buttons on the hand control. You will hear a series of rhythmic beeps. Press and hold the buttons until the beeping stops.
11. Check whether the bar is working properly.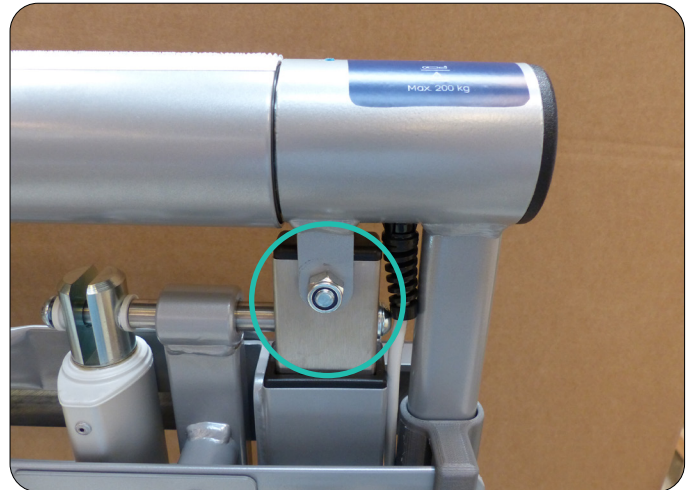


Dismount the bar from the actuator fitting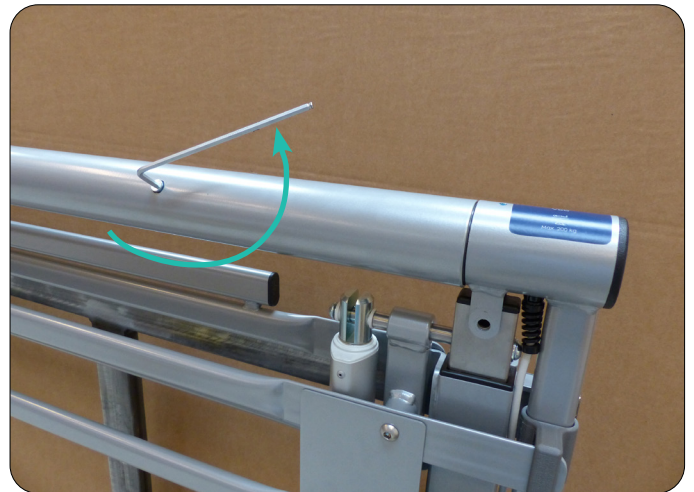

Remove the Allen screw

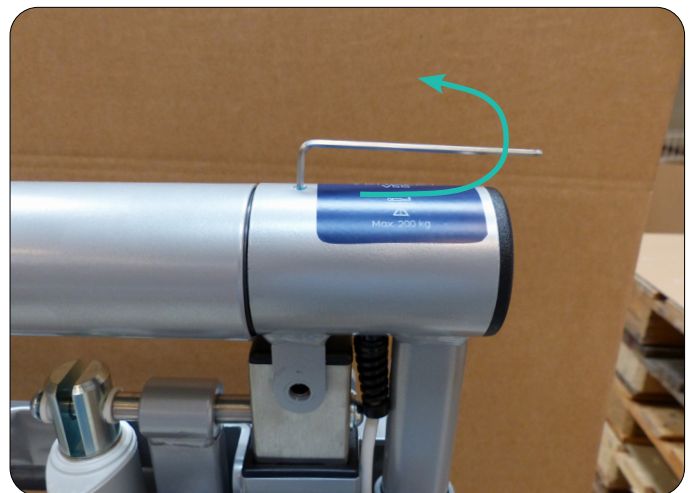Remove the pivot screw

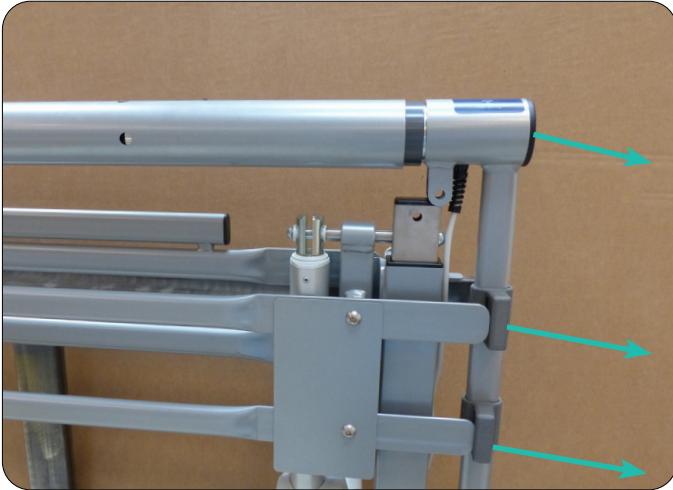Remove the motor bracket and motor

using the two Allen screws.

8. Secure the actuator cable with two new cable ties and route it along the inside of the bed frame to the control unit. Use cable ties to secure the actuator cable along the bed frame in such a way that the cable will not get caught by the moving parts of the bed.
9. Reconnect the power cable to the mains outlet.
10. Reset the system by simultaneously pressing the two top buttons on the hand control. You will hear a series of rhythmic beeps. Press and hold the buttons until the beeping stops.
11. Check whether the actuator is working properly.

Change the motor

Remove the Allen screws and free the actuator

Replacing the actuator

1. Disconnect the power cable from the mains outlet.
2. Disconnect the cable from the control unit.
3. Disconnect the actuator cable from the control unit and release it from the bed, allowing it to hang freely.
4. Remove the 2 Allen screws securing the actuator to the actuator fitting and remove the actuator.
5. Remove the actuator cable from the defective actuator if you want to re-use the cable. The actuator cable is attached to the actuator with a locking device. Release the locking device by pressing a small straight blade screwdriver against the side of the locking device on both sides.
6. Remove the actuator cable from the defective actuator and install it on the new actuator.
7. Mount the new actuator on the actuator fitting and attach it

Release the locking device

Replacing the velcro tape on a bar

The bar velcro bands are wearing parts that wear out over time. The velcro bands will have to be replaced when they no longer properly secure the turning sheet.

1. Release one end of the velcro band and pull it off the bar.
2. Use a sharp knife to remove the glue residue. Clean and degrease the bar.
3. Glue the new velcro strip onto the bar. Once the velcro tape is secured, run a hard roller over the velcro tape several times to ensure optimal adhesion to the bar and remove air pockets from the underside of the velcro tape.
4. You should always use original velcro tape from DHG.

Spare Parts

All individual components are delivered as complete units as spare parts.

Please contact DHG for further information.

Technical Data

In accordance with the Medical Device Regulation (EU) 2017/745, VENDLET Standard is a class I product.

	VENDLET V5S VENDLET V5S Speed Adjust	VENDLET Bari
Product no.	6100000 / 6100050	6100150
Max. user weight	200 kg	400 kg
Power supply:		
Transformer	Class 1	Class 1
Mains supply	230 Volt AC/50 Hz	230 Volt AC/50 Hz
Power rating	Up to 200 VA	Up to 200 VA
Output voltage	24 Volt DC	24 Volt DC
IP class	IPX6	IPX6
Intermittence	10%	10%
Battery (optional):	1,2 AH / 24 Volt DC	1,2 AH / 24 Volt DC
IP class	IPX6	IPX6
Control unit:		
Output voltage	24 Volt DC	24 Volt DC
IP class	IPX6	IPX6
Intermittence	10%	10%
Weight:		
Total system	approx. 58 kg	approx. 58 kg
Red side incl. actuator	27 kg	27 kg
Blue side incl. actuator	27 kg	27 kg
Control unit	3,9 kg	3,9 kg
Length:	200 cm	200 cm
Width:	May be adapted to beds between 70 and 140 cm	
Expected service life	10 years	10 years
Noise level max. load	65 db(A)	65 db(A)
Turning sheet:		

Regular length and width	350 cm / 180 cm	350 cm / 180 cm
Material	100 % cotton	100 % cotton
Slide sheet:		
Regular length and width	210 cm / 95 cm	210 cm / 95 cm
Material	100% nylon	100% nylon
System service check	Annually	Annually

Environmental Conditions & Disposal

Environmental conditions

Operation:	
Temperature	5 – 40 °C
Air humidity	20 – 90% at 30°C – no condensation
Atmospheric pressure	700 – 1060 hPa
Storage:	
Temperature	-10 - +50 °C
Air humidity	20 – 90% at 30°C – no condensation
Atmospheric pressure	700 – 1060 hPa
Transport:	
Take the necessary precautions to protect the electric components against rough jolts, shocks etc.	
After transportation in low temperatures, the actuators and motors in the bars might work irregularly until they have been warmed up to normal room temperature.	

Power consumption

	VENDLET V5S VENDLET V5S Speed Adjust		VENDLET Bari	
Standby		5 Watt		5 Watt
Working	Bars raise / lower	190 Watt	Bars raise / lower	190 Watt
	Tighten sheet (without person)	50 Watt	Tighten sheet (without person)	50 Watt
	Slacken sheet	20 Watt	Slacken sheet	20 Watt
	Moving a person at 90 kg	115 Watt	Moving a person at 90 kg	115 Watt
	Moving a person at 200 kg	150 Watt	Moving a person at 400 kg	220 Watt

Disposal

Do not discard a VENDLET with general household waste. On disposal of a VENDLET, DHG recommends that the system be divided into individual parts and components with a view to recycling.

All mechanical steel components may be disposed of as metal parts.

The control unit must be disposed of as electronic waste. Furthermore, it may be divided into metal waste, cable waste,

electronic waste, and reusable plastic.

The actuators may be disposed of as electronic waste. Furthermore, they may be divided into metal waste, electronic waste, and reusable plastic.

The hand control may be disposed of as electronic waste. Furthermore, it may be divided into cable waste, electronic waste, and reusable plastic.

The turning sheet and the slide sheet should be disposed of as combustible waste.

This Page is Left Intentionally Blank

This Page is Left Intentionally Blank

Moving Health Forward

PRESSURE ULCER PREVENTION
SAFE MOVING & HANDLING
SPECIALIST THERAPIES
BATHROOM SAFETY
RENTAL & SERVICE SOLUTIONS

Direct Healthcare Group Ltd
Withey Court, Western Industrial Estate
Caerphilly, United Kingdom
CF83 1BF

T: +44 (0) 800 043 0881

F: +44 (0) 845 459 9832

E: info@directhealthcaregroup.com

EC	REP
----	-----

EU Authorised Representative, Importer, and Distributor:

Direct Healthcare Group Sverige AB,
Torshamnsgatan 35, SE-164 40, Kista
Sweden
T: +46 (0) 8 557 200

Australian Sponsor:

Direct Healthcare Group Pty Ltd.,
68 Howe Street, Osborne Park,
Western Australia 6017
T: +61 (0) 423 852 810

Issue 1
Date: April 2022

DHG-HEALTHCARE.COM