

Operating Instructions for Electronic Wheelchair Scale




Caution! Load Cell damage can occur if scale is allowed to fall from storage position.

- ① From transport position, lay scale on floor.

DO NOT remove the Storage Pin or operate Column Release Lever prior to this operation.

DO NOT USE SCALE ON CARPETED FLOORS!

- ② Remove the Storage Pin.

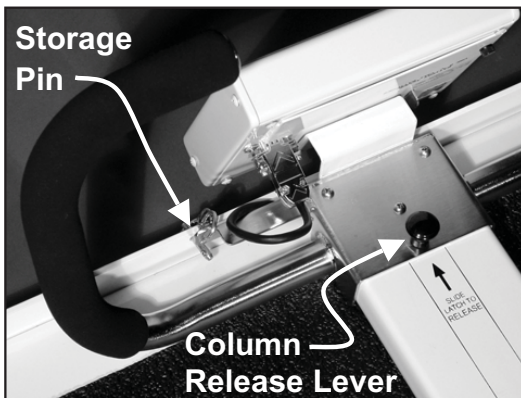

- ③ Slide the Column Release Lever to release column and lift to full up-right and locked position.



 **Cardinal
DETECTO**

Operating Instructions for Electronic Wheelchair Scale

- ④ Press the ON key located on front of the indicator.
- ⑤ When the indicator reads Zero, press the **TARE** key, enter the wheelchair weight, then press the **ENTER** key. The indicator will show a negative weight equal to the weight of wheelchair.



- ⑨ For storage, use column release to release column and fold down to locked position. Replace storage pin (see *warning label on scale base*). Store in an up-right position, preferably near a wall for maximum protection against damage.

HINT: If indicator is on when storing, press the ZERO key to zero the indicator which activates the automatic shutoff.

- ⑥ Position patient on scale by pulling wheelchair onto scale. **NEVER PUSH** patient on or off scale. Lock wheelchair wheels.
- ⑦ When weight reading is steady, read the patient's weight.
- ⑧ Remove patient from scale by backing down the ramp.

