# EasyBelt/FlexiBelt

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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
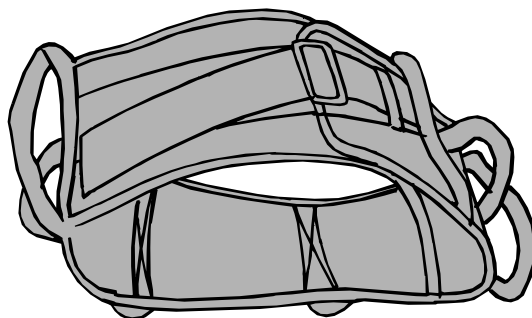
## REF

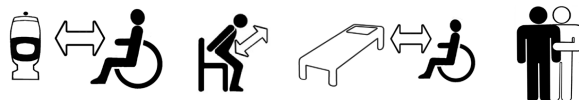6002	6023
6003	6024
6004	6025
6005	6026
6006	6072
6012	6073
6013	6074
6014	6075
6015	6076

SWL: 170 kg/ 375 lbs

## Table of contents

English .....	3
Svenska .....	5
Norsk .....	7
Dansk .....	9
Suomi .....	11
Deutsch .....	13
Nederlands .....	15
Français .....	17
Italiano .....	19
Español .....	21
<hr/>	
Product information .....	23
Accessories .....	23
Symbols .....	23

# EasyBelt/FlexiBelt

SystemRoMedic

Instructions for use - English

EasyBelt and FlexiBelt are used as support in different transfer situations. They are not to be used as assistive devices for lifting. The inside of EasyBelt is lined with nylon, making it easy to launder or wipe clean with disinfectant. FlexiBelt's soft inside feels comfortable against the skin. Both materials have an anti-slip function.

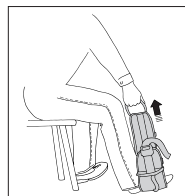
## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

## Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

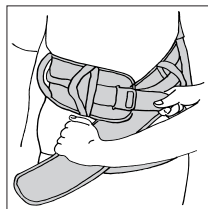
Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

## Placement


Apply the belt around the user, below the waist.

Fasten the belt and tighten it.


## Removing

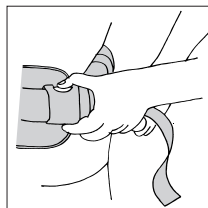Release the clasp, first loosening any straps, and then remove the belt.

## Using

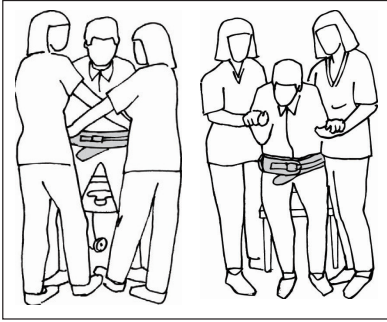
When standing up from sitting to standing the caregiver / caregivers can choose to work from the side or from the front of the user. One advantage of working in front of the user is to give a feeling of a more secure and safe standing up.

In raising with two helpers, it is advisable to work with cross-grip front of the user.

It is important to lean the user forward to get a natural pattern of movement, working in towards the body with the belt.

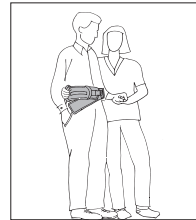When two caregivers are assisting, it is appropriate if one caregiver stands on each side, grasping the cross-grips at the back and holding the user by the hand at the front. Cross-grips at the back provide support and enable a good working stance.

Caregivers can work in pairs or alone, depending on the user's needs, since EasyBelt/FlexiBelt have several vertical and horizontal handles.

## Material

EasyBelt: Polyamid, nylon

FlexiBelt: Polyamid, polyester

## Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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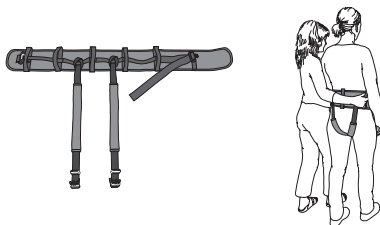
## Accessories

### SupportStraps

SupportStraps can be used as a leg harness in combination with EasyBelt/FlexiBelt.

## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

# SystemRoMedic®

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.