EasyRoll2Way

SystemRoMedic

EasyRoll4Way

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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
EasyRoll2Way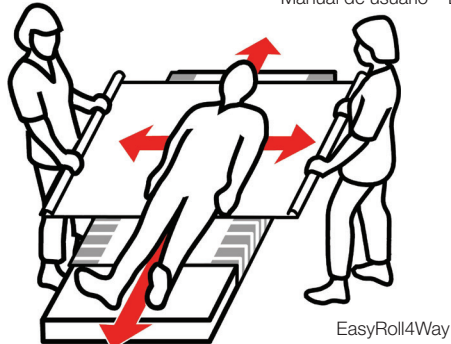

EasyRoll4Way

REF

2101	4011SSL	41402LS	4113	4111T	4112LT
2101S	41401S	41403LS	4111L	4112T	4113LT
4011S1	41402S	4111	4112L	4113T	41402SET
4011S	41403S	4112	4113L	4111LT	41403SET
4011L	41401LS				

SWL: 200 kg/440 lbs

Table of contents

Product information	3
English	6
Svenska	9
Norsk	12
Dansk	15
Suomi	18
Deutsch	21
Nederlands	24
Français	27
Italiano	30
Español	33
Symbols	36

**Product information/ Produktinformation/ Produktinformasjon/ Produktinformation/
Tuotetiedot/ Produktinformation/ Productinformatie/ Information produit/
Informazioni sul prodotto/ Información del Producto**

Art. no.	Product/Produkt/ Tuote/ Produktname/Product/ Produit/ Prodotto/ Producto	Size/ Storlek/ Størrelse/ Koko/ Größe/ Maat/ Taille/ Dimensione/ Tamaño
----------	--	---

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(Can be combined with EasyRoll 2way drawsheet or EasyRoll 4way sliding drawsheet)

411051P	EasyRoll Sliding sheet	100cm long (for 105cm bed)
4106	EasyRoll Sliding sheet	190cm long (for 90cm bed)
4404IB	EasyRoll Sliding sheet	100cm long (for 90cm bed)
418011	EasyRoll Sliding sheet	100cm long (for 80cm bed)
418021	EasyRoll Sliding sheet	140cm long (for 80cm bed)
418031	EasyRoll Sliding sheet	200cm long (for 80cm bed)
419011	EasyRoll Sliding sheet	100cm long (for 90cm bed)
419021	EasyRoll Sliding sheet	140cm long (for 90cm bed)
419031	EasyRoll Sliding sheet	200cm long (for 90cm bed)
4110511	EasyRoll Sliding sheet	100cm long (for 105cm bed)
4110521	EasyRoll Sliding sheet	140cm long (for 105cm bed)
4110531	EasyRoll Sliding sheet	200cm long (for 105cm bed)
4112011	EasyRoll Sliding sheet	100cm long (for 120cm bed)
4112021	EasyRoll Sliding sheet	140cm long (for 120cm bed)
4112031	EasyRoll Sliding sheet	200cm long (for 120cm bed)
41701PS	EasyRoll Sliding sheet	100cm long (for 70cm bed)
41702PS	EasyRoll Sliding sheet	140cm long (for 70cm bed)
41703PS	EasyRoll Sliding sheet	200cm long (for 70cm bed)
41801PS	EasyRoll Sliding sheet	100cm long (for 80cm bed)
41802PS	EasyRoll Sliding sheet	140cm long (for 80cm bed)
41803PS	EasyRoll Sliding sheet	200cm long (for 80cm bed)
41901PS	EasyRoll Sliding sheet	100cm long (for 90cm bed)
41902PS	EasyRoll Sliding sheet	140cm long (for 90cm bed)
41904TV	EasyRoll Sliding sheet	160cm long (for 90cm bed)
41903PS	EasyRoll Sliding sheet	200cm long (for 90cm bed)
411051PS	EasyRoll Sliding sheet	100cm long (for 105cm bed)
411052PS	EasyRoll Sliding sheet	140cm long (for 105cm bed)
411053PS	EasyRoll Sliding sheet	200cm long (for 105cm bed)
411201PS	EasyRoll Sliding sheet	100cm long (for 120cm bed)
411202PS	EasyRoll Sliding sheet	140cm long (for 120cm bed)
411203PS	EasyRoll Sliding sheet	200cm long (for 120cm bed)
41902PSB	EasyRoll Sliding sheet	140x200cm (for 90cm bed)
419031D	EasyRoll Sliding sheet	200cm long (for 90cm bed)
418031R	EasyRoll Sliding sheet	200cm long (for 80cm bed)
419031R	EasyRoll Sliding sheet	200cm long (for 90cm bed)
4110531R	EasyRoll Sliding sheet	200cm long (for 105cm bed)
519031R	EasyRoll Sliding sheet	200cm long (for 90cm bed)
45901S	EasyRoll Sliding sheet Anticubitus	100cm long (for 90cm bed)
45902S	EasyRoll Sliding sheet Anticubitus	140cm long (for 90cm bed)
45903S	EasyRoll Sliding sheet Anticubitus	200cm long (for 90cm bed)
451051S	EasyRoll Sliding sheet Anticubitus	100cm long (for 105cm bed)
451052S	EasyRoll Sliding sheet Anticubitus	140cm long (for 105cm bed)

451053S	EasyRoll Sliding sheet Anticubitus	200cm long (for 105cm bed)
451201S	EasyRoll Sliding sheet Anticubitus	100cm long (for 120cm bed)
451202S	EasyRoll Sliding sheet Anticubitus	140cm long (for 120cm bed)
451203S	EasyRoll Sliding sheet Anticubitus	200cm long (for 120cm bed)
51901FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 90cm bed)
51902FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 90cm bed)
51903FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 90cm bed)
511051FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 105cm bed)
511052FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 105cm bed)
511053FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 105cm bed)
511201FAH	EasyRoll Sliding sheet w Anti-slip edge	100cm long (for 120cm bed)
511202FAH	EasyRoll Sliding sheet w Anti-slip edge	140cm long (for 120cm bed)
511203FAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 120cm bed)
519031DAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 90cm bed)
5110531RAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 105cm bed)
5112031RAH	EasyRoll Sliding sheet w Anti-slip edge	200cm long (for 120cm bed)
42901PS	EasyRoll Sliding sheet w Barrier	100cm long (for 90cm bed)
42902PS	EasyRoll Sliding sheet w Barrier	140cm long (for 90cm bed)

EasyRoll 2Way - drawsheet/ draglakan/ stikkklakenet/ træklagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera
(To be combined with EasyRoll sliding sheet)

2101	EasyRoll 2way Drawsheet	100x180cm
2101S	EasyRoll 2way Drawsheet	140x200cm

EasyRoll 4Way - drawsheet/ draglakan/ stikkklakenet/ træklagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera
(To be combined with EasyRoll sliding sheet)

4011S1	EasyRoll 4way Sliding Drawsheets	150cm
4011S	EasyRoll 4way Sliding Drawsheets	140x200cm
4011L	EasyRoll 4way Sliding Drawsheets	200x200cm
4011SSL	EasyRoll 4way Sliding Drawsheets	150x200cm
4111T	EasyRoll 4way Sliding Drawsheets	140cm long (for 90cm bed)
4112T	EasyRoll 4way Sliding Drawsheets	140cm long (for 105cm bed)
4113T	EasyRoll 4way Sliding Drawsheets	140cm long (for 120cm bed)
4111LT	EasyRoll 4way Sliding Drawsheets	200cm long (for 90cm bed)
4112LT	EasyRoll 4way Sliding Drawsheets	200cm long (for 105cm bed)
4113LT	EasyRoll 4way Sliding Drawsheets	200cm long (for 120cm bed)
4111B	EasyRoll 4way Sliding sheet	140x200cm
4112B	EasyRoll 4way Sliding sheet	140x215cm
4113B	EasyRoll 4way Sliding sheet	140x230cm
4111LB	EasyRoll 4way Sliding sheet	200x200cm
4112LB	EasyRoll 4way Sliding sheet	200x215cm
4113LB	EasyRoll 4way Sliding sheet	200x230cm

EasyRoll - pillowcase/ örngott/ putetrek/ hovedpudebetræk/ tynnyliina kahvat/ Kissenbezug/ kussensloop/ Taie d'oreiller/ federa/ unda de almohada

4303	EasyRoll Sliding Pillow case	50x60cm
4304	EasyRoll Pillow case w Barrier	50x60cm

EasyRoll 2way Inco, EasyRoll 4way Inco, EasyRoll Chair pad, EasyRoll Underpad and EasyRoll Mattress Protector

10991	EasyRoll Inco 2way Sliding sheet	185x90cm
10992	EasyRoll Inco 4way Sliding sheet	120cm
10992-5	EasyRoll Inco 4way Sliding sheet	140cm
1299k	EasyRoll Chair pad	45x45cm
14545GR	EasyRoll Chair pad	45x45cm
16060GR	EasyRoll Chair pad	45x45cm
16090GR	EasyRoll Chair pad	45x45cm
80011-1	EasyRoll Underpad	85x90x180 cm
80011B	EasyRoll Underpad	80x180cm
80011P	EasyRoll Underpad	80x180cm
80011PL	EasyRoll Underpad	90x210cm
80017-1	EasyRoll Underpad	85x90 cm
80017P	EasyRoll Underpad	85x90 cm
80018	EasyRoll Underpad	150x90 cm
80018P	EasyRoll Underpad	150x90 cm
80019	EasyRoll Underpad	120x90x180cm
80011PS	EasyRoll Underpad w Barrier	70x190cm
80010P	EasyRoll Underpad w Sliding sheet	80x85x180 cm
90.200J01	EasyRoll Mattress protector	90x200cm
105.200J01	EasyRoll Mattress protector	105x200cm
120.200J01	EasyRoll Mattress protector	120x200cm

Product combinations, Produktkombinationer, Produkt kombinasjon, Produkt kombination, Tuoteyhdistelmä, Produktkombination, Product-combinatie, Combinaison de produit, Combinazione di prodotti, Combinación del producto

Art. no.	Combination	Art. no.	Combination
4111	4111 + 4111T	41401S	41401S + 4011S
4112	4112 + 4112T	41402S	41402S + 4011S
4113	4113 + 4113T	41403S	41403S + 4011S
4111L	4111L + 4111LT	41401LS	41401LS + 4011L
4112L	4112L + 4112LT	41402LS	41402LS + 4011L
4113L	4113L + 4113LT	41403LS	41403LS + 4011L
41402SET	41402SET + 4011S	10990	10990 + 10992
41403SET	41403SET + 4011L	10990-5	10990-5 + 10992-5

EasyRoll2Way

SystemRoMedic

EasyRoll4Way

SystemRoMedic

EasyRoll2Way and EasyRoll4Way are two different draw sheets that can be used together with a EasyRoll slide sheet. EasyRoll2Way is used for lateral repositioning and EasyRoll4Way is used for both lateral repositioning and repositioning higher up or lower down in bed. EasyRoll4Way has an anti-friction material on the underside and has been developed mainly for heavy users. Available in several sizes.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.


Always read the manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Placement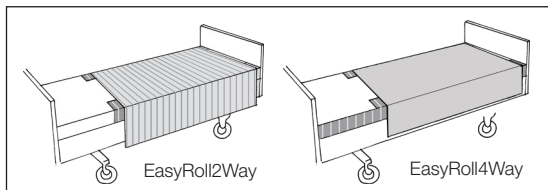

Place EasyRoll with the smooth surface in the middle of the bed and with EasyRoll2Way on top. After turning, gliding movement can be inhibited by bedding EasyRoll2Way in under the mattress on both sides of the bed.

A tag on EasyRoll4Way indicates which side should be placed facing upwards in bed. After turning, gliding movement can be inhibited by bedding EasyRoll4Way in under the mattress on both sides of the bed.

Turning with EasyRoll2Way

Perform the turn by grasping the draw sheet at the level of the hips and the shoulders. For best ergonomic technique, grasp a lot of fabric.

Work with straight wrists and shift weight by leaning backwards.

Turning with the aid of a draw sheet can be performed by two caregivers. This is a collaborative effort whereby one caregiver pulls the draw sheet (as above) and the other presses down somewhat near the shoulders and hips while at the same time pushing.


Turn the user. After turning, gliding is prevented by tucking the draw sheet under the mattress on both sides.

Turning with EasyRoll4Way

The EasyRoll4Way draw sheet, which has an anti-friction material on the underside, has been developed specially for heavy users. The procedure for turning is the same as for EasyRoll2Way (above). After turning, gliding is prevented by tucking the draw sheet under the mattress on both sides. To prevent chafing, smooth out the bedding under the user before tucking the draw sheet in under the mattress. The caregivers can do this by pulling diagonally on the draw sheet from either side. Raise the bed rails before leaving the user.


Higher up in bed with EasyRoll4Way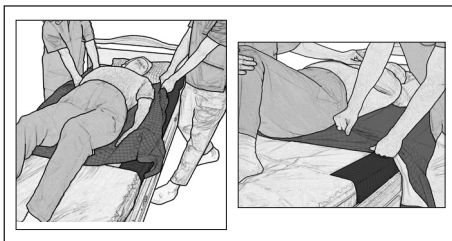

To reposition the user higher up in bed, grasp the draw sheet on both sides near the shoulders, stand in a walking stance and take one step backwards.

To prevent the user from gliding down in bed, tuck in part of the draw sheet, so that it is folded double under the user's seat.




Sitting up in bed with EasyRoll4Way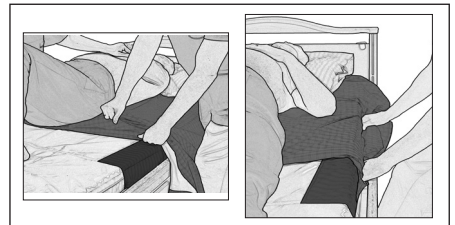

With a EasyRoll4Way in the bed, downward gliding can be prevented even when the user sits up in bed. Tuck in part of the draw sheet, so that it is folded double under the user's seat. Then, tuck the draw sheet in under the mattress, but only at the hip section. When the head end of the bed is subsequently raised, the draw sheet acts as a friction surface to prevent sliding.



Lift-assisted turning

When manual repositioning seems to place too much strain on the body, EasyRoll4Way with handle straps (Art. No. 4111, 4112, 4113, 4111L, 4112L, 4113L) can be used in combination with a user lift and a two-point slingbar.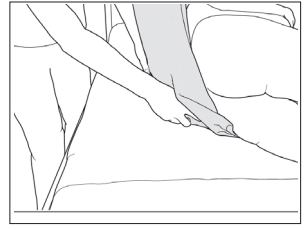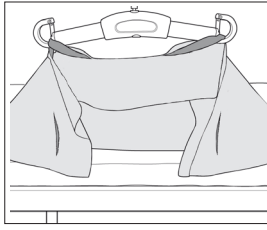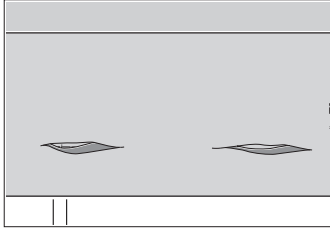

Select two of the handle straps on one side and hook them to the slingbar. Prepare the user for turning.

Start turning carefully and stop to make any necessary adjustments; e.g., move the user closer to the middle of the bed to free more space for turning. Then, raise the slingbar until the user is lying on his or her side.

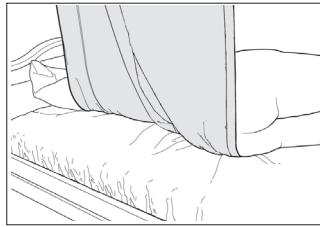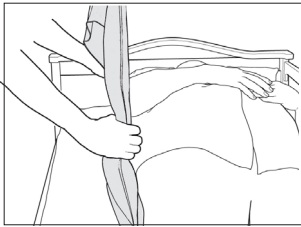With one hand under the hip, pull the lower edge of the EasyRoll4Way in, so that it is folded double.

The user's back is now supported and the caregiver is easily able to attend to the user's personal hygiene needs.

Note

Be aware that there is a risk of the user sliding out of bed. Raise the bed rails before leaving the user.

Material

Polyester, Cotton, Duchesse

Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

Contact your local distributor if you have any questions about the product and its use. See www.directhealthcare-group.com for a complete list of distributors.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com