# EasySlide

## SystemRoMedic™

Instructions for use – English

Bruksanvisning – Svenska

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Návod na použitie – Slovensky

**REF**

2010  
2011  
2020  
2021

**REF**

2015

**REF**

2040  
2045

SWL: 300 kg/ 661 lbs

## Table of contents

English .....	3
Svenska .....	5
Norsk .....	7
Dansk .....	9
Suomi .....	11
Deutsch .....	13
Nederlands .....	15
Français .....	17
Italiano .....	19
Español .....	21
Český .....	23
Slovensky .....	25
Products .....	27
Accessories .....	27
Symbols .....	27

EasySlide is used for turning in bed, for transfers between bed and wheelchair/toilet, and for repositioning the user higher up in bed. EasySlide is a tubular, padded mat with a friction-reducing material on the inside.

EasySlide disposable is a versatile sliding mat that makes many common transfers, even the heavy ones, easier for both the user and the caregiver. The practical, hygienic package contains a roll with up to 60 individual EasySlide mats for short-term use. That means many easy transfers with maximum hygiene and safety. EasySlide disposable is a thin, pliable and very easy-to-use sliding mat that can be used to facilitate turning and repositioning higher up in bed, for supine and seated transfers, and for repositioning in the seated position. EasySlide disposable is an unbelievably versatile assistive device that can also be used to lift the user's legs in and out of bed, to assist dressing and undressing, and to facilitate application of a lifting sling.

## Check Safety

### Visual inspection



Check the condition and function of the assistive device regularly.

Always inspect the product after laundering (for the non-disposable EasySlides)

Check to ensure that seams and material are free from damage.

Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Friction test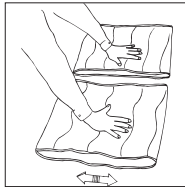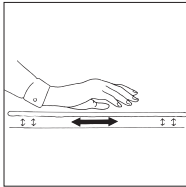

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

## Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

## Placement of EasySlide

Place the EasySlide under one buttock or under one shoulder.

For transfers, all of the user's body weight should be shifted over onto the EasySlide.


To remove, pull on the lower end of the EasySlide.

## Using EasySlide

1. EasySlide can be used to advantage for transfers to/from a wheelchair. Complement, preferably, with an EasyGlide.

2. EasySlide can be used under the shoulders as a friction-reducing assistive device to facilitate repositioning higher up in bed.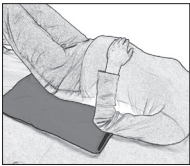

3. EasySlide can be used under the hips to facilitate lateral transfer.

Active users can use EasySlide themselves, but it can also be used by less active users with the aid of a caregiver.

The gap between the underlying surfaces (bed/gurney/show trolley) must not exceed 5 cm.

NOTE!

Always remember to lock wheels.

To prevent the risk of the user sliding/falling, never leave the EasySlide under the user after the completed transfer, and never position the user too close to the edge of the bed.

## Material

EasySlide: Polyester, microfibre, polyamide

EasySlide Disposable: Polyethylene

## Care of the product

Read the product label.

EasySlide

60-80° C  
140-176° F

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

EasySlide Disposable

Do not launder. Discard the product if it is soiled or if there is a need to use another product.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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## Accessories

Art. no	Product	Description
2051	Protective cover	For EasySlide 2010 ,2020, 2030
2052	Protective cover	For EasySlide 2011, 2021
2055	Disposable cover	For EasySlide 2010, 2020, 2030
2030	Bundle	EasySlide 2020 + Protective cover 2051

## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.