EasySlide

SystemRoMedic™

Instructions for use – English

Bruksanvisning – Svenska

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EasySlide is used for turning in bed and for transfers between bed and gurney/shower trolley.
EasySlide is a tubular, padded mat with a friction-reducing material on the inside.



Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test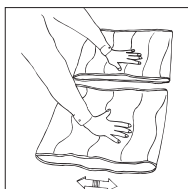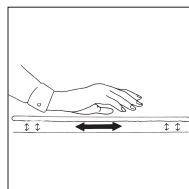

Low friction - Ensure that the surface is smooth and slides effortlessly.
Compare with a new product.

Always read the user manual


Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Using EasySlide

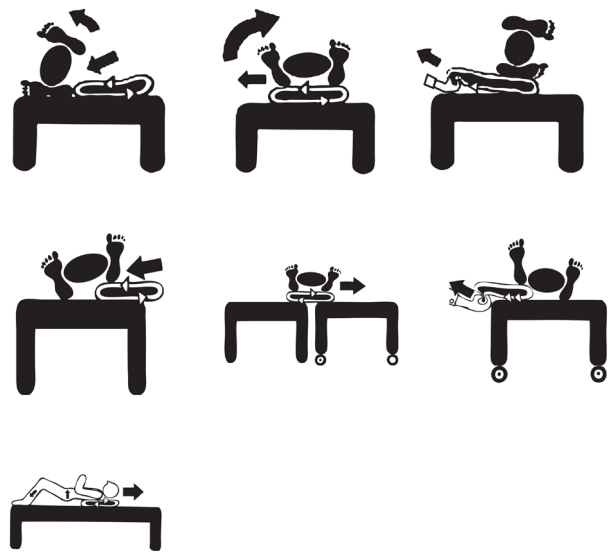Lean the user on one side. This is easier with the help of a draw sheet. Place the transfer slide under the patient. Remove the slide when the transfer is completed.

Note

The gap between the underlying surfaces (bed/ gurney/show trolley) must not exceed 5 cm.

NOTE!

Always remember to lock wheels.

To prevent the risk of the user sliding/falling, never leave the EasySlide under the user after the completed transfer, and never position the user too close to the edge of the bed.

Material

Polyester, microfibre, polyamide

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Úplný zoznam distribútorov nájdete na www.directhealthcaregroup.com.

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Accessories

Art. no	Product	Description
1051	Protective cover	For EasySlide 1020
1151	Protective cover	For EasySlide 1120
1251	Protective cover	For EasySlide 1220
1055	Dispoable cover	For EasySlide 1020
1155	Dispoable cover	For EasySlide 1120
1255	Dispoable cover	For EasySlide 1220

1030	Bundle	EasySlide 1020 + Protective cover 1051
1130	Bundle	EasySlide 1120 + Protective cover 1151
1230	Bundle	EasySlide 1220 + Protective cover 1251

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com