



RoMedicClips are used to facilitate positioning of the patient. RoMedicClips can be attached to most carry bars and can be used on all draw sheets. RoMedicClips are used in pairs and are placed on either side of the slingbar.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The slings are available in several different materials and in sizes from XXS to XXL. All models are safe and very easy to use and are rated for lifting patients weighing up to 300 kg. The choice of model and material depends on the patient's needs and the transfer situation.



Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.



Always read the manual



Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

Using RoMedicClips

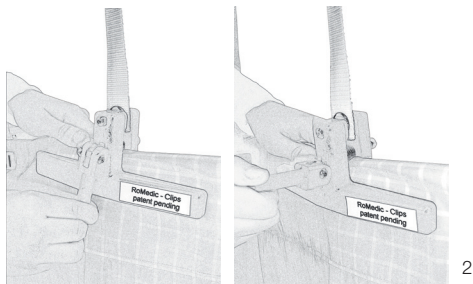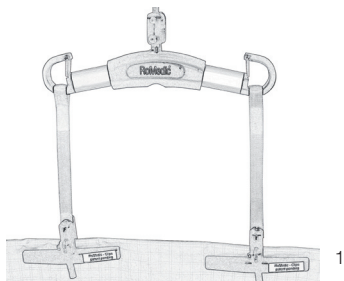

1. First, place the RoMedicClips on the slingbar by looping the suspension loops on to the slingbar hooks.

Fold the draw sheet at the point where the clip is to be applied.

2. Before positioning, place the folded edge of the draw sheet in the opening of the clip.

Place the clips as close to the patient as possible.

Clasp the draw sheet in the clip by pulling the locking handle from the open to the closed position.

The width of the clip opening is regulated by rotating the locking handle.

3. Raise the lift when the patient is ready for positioning. Exercise caution during this operation, and be prepared to interrupt positioning if anything appears to be out of place.

Complete the positioning when the patient is in the desired position.

Care of the product:

When necessary, clean RoMedicClips using warm water or rubbing alcohol.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Handicare AB
Veddestav. 15. Box 640 Tel: +46 (0)8 -557 62 200
SE-175 27 Järfälla Fax: +46 (0)8-557 62 299
SWEDEN

Internet: www.handicare.com
E-mail: info@handicare.se

