
# HighBackSling SystemRoMedic<sup>®</sup>

# **Trial Fitting Protocol**

D - 12 11	
Patient's	name:

#### Social insurance number:

HighBackSling is easy for the caregiver to apply and it gives the patient a feeling of security. It features leg sections, which means that the patient does not sit on the sling. Instead, the sling provides support from the tailbone up over the head and under the legs. HighBackSling is designed for situations requiring extra support and can be used for lifting from a recumbent position on the floor.

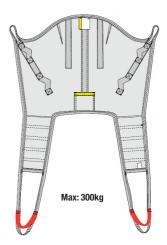



#### Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

Which size is used?   XX-Small   XX-Small	nall Small	Medium	Large	X-Large	XX-Large
Which material is used	Polyester net		Plasticized net		
Accessories	AdjustmentLoop		StrapsPadding		

### Which slingbar is used?

### Leg support placement:





Overlapped



Each leg separate

## Instructions for use

Can the sling be left under the patient in the wh	ieelchair?	Yes	No
Can the sling be used for transfers to/from the	toilet?	Yes	No
Can the sling be used in showering situations?		Yes	No
Loop alternatives/AdjustmentLoop:			
Have staff that use the assistive device: Read the instructions? Received practical instruction in patient transfers?	Yes Yes	No No	
Instructions to personnel: e.g.; Raise the bed side rail on the side towards which the patier Placement of legs/feet? Use bed functions.	nt is to be turned	. Placemen	t of head/ arms/hands?
Number of caregivers:			
Instructions to patient: e.g.; In what way can the patient participate actively in the transf	fer situation? Wh	ich instruct	ions are used?
Other:			
Trial-fitted by:		Date: Phone Phone	
Handicare AB			hanges or questions,



Internet: www.handicare.com

Fax:+46 (0)8-557 62 299

E-mail: info@handicare.se