Instructions for use – English

Bruksanvisning – Svenska

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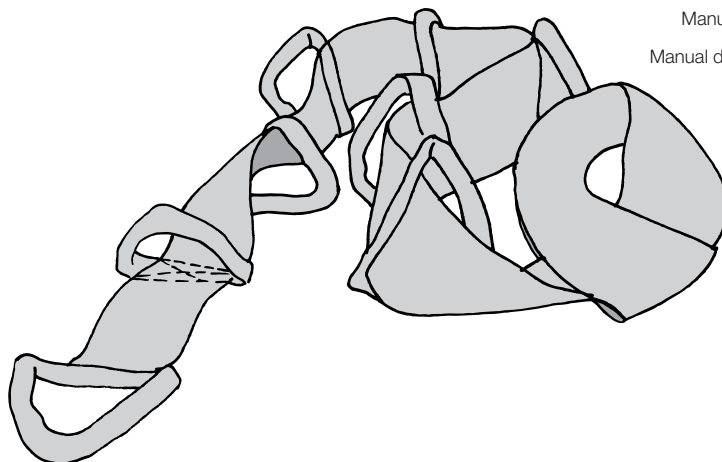
REF
6090

SWL: 150 kg/ 331 lbs

Table of contents

| | |
|---------------------------|----|
| English | 3 |
| Svenska | 5 |
| Norsk | 7 |
| Dansk | 9 |
| Suomi | 11 |
| Deutsch | 13 |
| Nederlands | 15 |
| Français | 17 |
| Italiano | 19 |
| Español | 21 |
| | |
| Product Information | 23 |
| Accessories | 23 |
| Symbols | 23 |

FlexiGrip is used as a helpful grip that facilitates more active raising of the user from a recumbent to a seated position. FlexiGrip is fastened to the bed's short side and has several handles on both sides to enable the user to 'climb' up to a seated position.

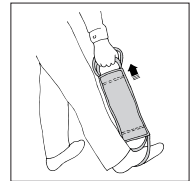
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

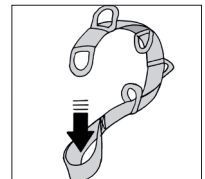
Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Placement of FlexiGrip

Fasten FlexiGrip by drawing the straps through the loops. Check to ensure that the fixture points will withstand the pressure of the load transferred by the straps.


Using FlexiGrip

The user can now 'climb' up to a sitting positioning with the help of FlexiGrip. The user can also use FlexiGrip as a support for sitting on the edge of the bed.

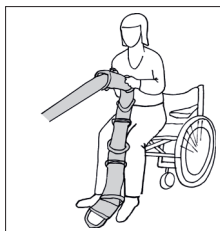To and from wheelchair with FlexiGrip

FlexiGrip can be used as support for transfers to and from a wheelchair. The user can perform low-level repositioning by grasping the FlexiGrip, raising the buttocks and using the feet to manoeuvre into a seated position.

Material

Polyester

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Accessories

| Art. no. | Product name |
|----------|-----------------|
| 6091 | Extension strap |

Symbols

| | | | |
|---|---|--|-------------------|
|  | This product complies with the requirements of the Medical Device Regulation 2017/745 |  | Visual Inspection |
|  | Medical Device |  | Read the manual |
|  | Caution |  | Product Code |
|  | Manufacturer information |  | Batch Code |

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.