FlexibleSling

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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Návod na použitie – Slovensky

Undivided leg support

Divided leg support

REF	42520002	42522002
	42520003	42522003
	42520004	42522004
	42520006	42522006
	42520007	42522007
	42520008	42522008

REF	42521003	42521403
	42521004	42521404
	42521006	42521406
	42521007	42521407
	42521008	42521408


SWL: 300 kg/ 660 lbs

Table of contents

English.....	3
Svenska.....	7
Norsk.....	11
Dansk.....	15
Suomi.....	19
Deutsch.....	23
Nederlands.....	27
Français.....	31
Český.....	35
Español.....	39
Slovensky.....	43
Symbols.....	47

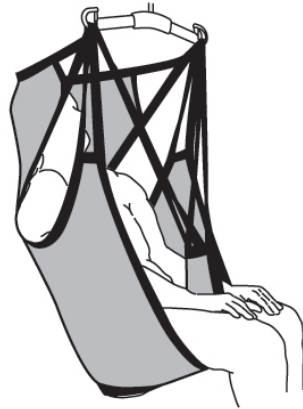Undivided leg support

Divided leg support

FlexibleSling is pliable and flexible around the body and provides a feeling of security. It is available in three models; one with divided leg supports, one with undivided leg supports and one with undivided leg supports and commode opening. FlexibleSling is designed to give a lot of support and to adapt to the shape of the patient's body. After use, it can be left in the wheelchair, moulded seat or similar.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully.

First, apply the sling's back loops to the slingbar, and then apply the leg support loops.

Use a slingbar that has been tested.

Note that there may be a risk of the patient tipping forward when a sling with standard (not separate) leg supports is used.

The lowest allowable safe working load always determines the safe working load of the assembled system.

Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

FlexibleSling with divided leg supports, Application of the sling when the patient is sitting in a wheelchair/chair or in bed

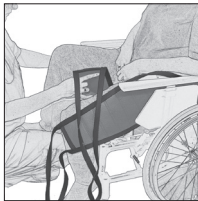
1

2

3

4

1. Hold the sling in one hand and grasp the lower edge of the sling with your other hand. This is easier if the patient leans forward or can be helped to lean forward slightly. Guide the sling down behind the patient's back, by sliding it along the back of the chair/wheelchair or along the bed, ideally, to the tailbone. The mid-section of the sling should follow the patient's spine.

2. Take care to guide the sling down to the patient's hips. Do this by pushing the sling into the space in the back corners of the wheelchair seat.

3. Then, work the leg sections out along the outside of the patient's thighs. Face the patient and pull the lower edge of the leg support, so that the sling slides into place around the hips. Light pressure against the patient's knees prevents him/her from sliding forward in the chair.

4. After checking to ensure that the leg supports are of equal length on both sides, place them under the patient's thighs. Cross the leg supports and hook them to the slingbar. Start raising the lift, when the straps become taut, stop and check that everything is ok before proceeding with the lift.

Removal of the sling when the patient is sitting

1

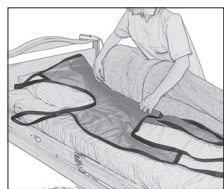
2


1. Fold the leg support and allow it to slide under itself when you pull it out at the outside of the thigh.

2. Then, pull the sling up behind the patient's back.

Application of the sling when the patient is lying in bed, on a stretcher or on the floor


1

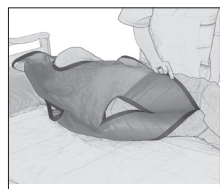2

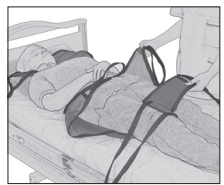1. Turn the patient towards you. Place the sling so that the product label is facing the underlying surface and the sling's mid-section follows the spine. The upper edge of the sling should be high enough so that the sling supports the head and the lower edge is level with the tailbone. Place the sling as far in under the patient as possible. Push the back-section loop under, so that you can reach it once you have turned the patient back onto the sling.

3

4

2. To reduce the number of application steps, you can fold the back section up against the patient's back and place the leg supports between the patient's knees when he/she is turned towards you, and then pull them forward when the patient is lying on the sling. Stop and check that everything is ok before proceeding with the lift.

3. Turn the patient back onto the sling and pull out the head support, back section and leg support.

4. Cross the leg supports and hook them to the slingbar. Start raising the lift.

FlexibleSling with undivided leg supports, Application of the sling when the patient is lying in bed, on a stretcher or on the floor

1

2

1. Turn the patient towards you. Place the sling so that the product label is facing the underlying surface and the sling's mid-section follows the spine. The upper edge of the sling should be high enough so that the sling supports the head and the lower edge is level with the tailbone. Place the sling as far in under the patient as possible. Push the back-section loop under, so that you can reach it once you have turned the patient back onto the sling.

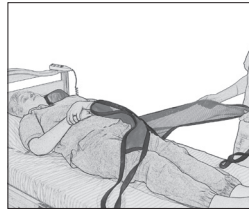
3

2. To ensure that the sling is correctly applied and that its mid-section is aligned with the patient's spine, you can fold the back section up against the patient's back.


3. Turn the patient back onto the sling and pull out the head support, back section and leg support. Connect the strap loops to the slingbar, starting with the upper strap loops, and then the leg supports. Start raising the slingbar, and then stop to ensure that everything is as it should be before lifting the patient from the underlying surface.

FlexibleSling with divided and undivided leg supports, Removal of the sling when the patient is recumbent

Turn the patient towards you and fold the sling in under the patient so that the fabric slides towards you when you pull it out after you have turned back the patient.

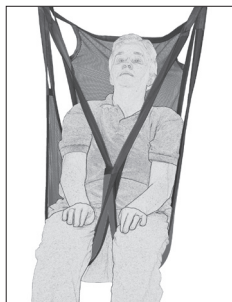Alternative application

The sling's high back provides good head support while allowing the patient to hold his/her arms inside or outside the sling.

For the model with divided leg supports, usually, the leg supports are crossed before they are hooked to the slingbar.

Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions.

The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on www.directhealthcaregroup.com.

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer.

The periodic inspection instructions for this product are found on www.directhealthcaregroup.com

Material:

Polyester net: An airy material that can be left under the patient, if necessary. Withstands high laundry temperatures.

Care of the product:

Read the product label.

Do not use rinsing agent.
To prolong product life, avoid
tumble-drying.

Size information

Slings can often be used for both 2 and 4-point suspension. It is essential to ensure that the width of the slingbar is appropriate for the sling size and the patient's status and capacity.

Use patient-specific slings to ensure safety and hygiene.

FlexibleSling, divided leg support

Size		XXS/Grey	XS/Orange	S/Red	M/Yellow	L/Green	XL/Purple
Rec. user	lbs	22-66	44-77	55-121	110-165	154-243	220-287
weight	kg	10-30	20-35	25-55	50-75	70-110	100-130
Sling overall	inch	29,5	32	35,5	40	40	41
height	cm	75	81	90	102	102	104
Sling seat	inch	8	10	14	18	22	25,5
measurement	cm	20	25	35	45	55	65

FlexibleSling, undivided leg support

Size		XS/Orange	S/Red	M/Yellow	L/Green	XL/Purple
Rec. user	lbs	44-88	66-154	132-198,5	176-286,5	242,5-397
weight	kg	20-40	30-70	60-90	80-130	110-180
Sling overall	inch	31,5	33,5	36,5	38	38,5
height	cm	80	85	93	96	98
Sling seat	inch	29,5	33,5	37	41,5	45,5
measurement	cm	75	85	94	105	115

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.