BRUGSANVISNING · USER MANUAL

XXL-Rehab Stand Tall gangbord  
XXL-Rehab Stand Tall

DK

UK

MAX 325 KG

DK

Kan med fordel anvendes til:

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## Instructions for XXL-Rehab Stand Tall

DK  
UK

### ⚠ CAUTION ⚠ CAUTION ⚠

- This product should not be used without proper instruction from a healthcare professional.
- Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.
- XXL-Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- If components are damaged or missing, contact XXL-Rehab immediately. Do NOT use substitute parts.
- XXL-Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from XXL-Rehab.

⚠ To ensure proper use of the XXL-Rehab Stand Tall, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with ⚠.

Also remember that in order to prevent the risk of injury to the user, the stand tall may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

### USE

The use of the XXL-Rehab Stand Tall is ideal in hospitals or nursing homes.

The XXL-Rehab Stand Tall is designed to provide support and security for persons weighing up to 325 kg when walking indoors.

⚠ Using the stand tall for persons weighing more than 325 kg may increase the risk of tilting or damage the stand tall.

⚠ The XXL-Rehab Stand Tall should be used on a plane hard surface with a maximum inclination of 3.5 degrees, as there may otherwise be a risk of tilting.

### DESCRIPTION

The XXL-Rehab Stand Tall is equipped with four horizontal, through-going bolts (two on each side) for height adjustment of forearm supports and hand supports. The bolts are locked with hand nuts.

### FINAL ASSEMBLY

The height of the stand tall's forearm supports and hand supports is adjustable by screwing the hand nuts off and pulling the two horizontal bolts out of the frame. Then set the forearm and hand supports at the appropriate position, push the bolts through the frame and screw on the hand nuts to secure the bolts.

⚠ The stand tall should not be used until the hand nuts have been screwed tightly onto the bolts, as there may otherwise be a risk of the forearm/hand supports loosening which may cause the user to fall.

Please ensure that the curved heads of the bolts abut closely against the frame, as there may otherwise be a risk that the hand nuts cannot be tightened correctly and therefore work loose.

The hand nuts ought to be facing forward on the stand tall to avoid injury to the user.

### DURING USE

Lock the brakes by bringing the locking bracket over the brake handles, while keeping the stand tall braked.

⚠ The brakes must be locked when the user gets up/sits down while using the stand tall as support.

⚠ The brakes may be difficult to activate for users with limited strength of fingers. Such users should be instructed in using the brakes with both hands.

⚠ The stand tall should be manoeuvred carefully across obstacles such as thresholds, as there may otherwise be a risk of tilting.

⚠ The stand tall is not designed for driving up and down ramps inclining more than 3.5 degrees, as there may otherwise be a risk of tilting.

⚠ The stand tall must only be used indoors and on plane hard surfaces to avoid the risk of tilting.

### MAINTENANCE AND CLEANING

The stand tall can be cleaned with water and ordinary cleaning agents.

Lock nuts and bolts must be checked on a regular basis and tightened, if necessary.

⚠ Non-compliance with the bolt inspection guidelines may lead to malfunction.

DK

UK

## WARRANTY

The XXL-Rehab Stand Tall is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended use; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by XXL-Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

Maximum weight capacity is	325 kg
Total length	73 cm
Total width	76 cm
Width between underarm support, adjustable	68-81 cm
Internal width	72 cm
Adjustable height	100-130 cm
Weight	20 kg
Front wheel	100 mm
Rear wheel	125 mm
Frame	Powder-lacquered chromium molybdenum steel
Grip	PU foam
Underarm support	PU
ISO – classification	ISO 12 06 12 - 06

XXL-Rehab Stand Tall is produced and tested in compliance with international standards DS/EN ISO 11199-3:2005 and DS/EN 12182:2000.

07/2014/JP

## MANUFACTURER

XXL-Rehab  
Fuglebækej 1D  
DK-2770 Kastrup  
Denmark  
Tlf.: +45 70 25 25 22  
Fax: +45 70 25 25 44  
[www.xxl-rehab.com](http://www.xxl-rehab.com)


# XXL REHAB®

Strongly engineered and functionally designed exclusively for bariatric users!


XXL-Rehab Minimaxx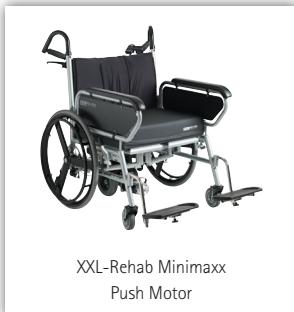

XXL-Rehab Minimaxx  
Push Motor

XXL-Rehab Manual  
Wheelchair - The Eclipse

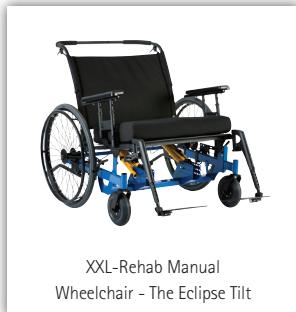XXL-Rehab Manual  
Wheelchair - The Eclipse Tilt

XXL-Rehab  
HD Electric Wheelchair


XXL-Rehab Wheelchair Cushions

XXL-Rehab Shower Bench

XXL-Rehab  
Bedside commode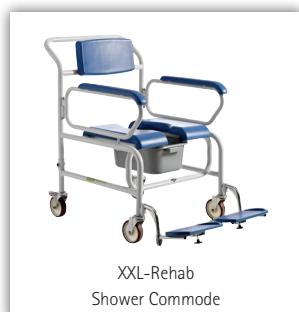

XXL-Rehab  
Shower Commode

XXL-Rehab  
Shower Commode Wheeler

XXL-Rehab  
Shower Commode Rise N Recline

XXL-Rehab  
Shower Commode Tilt

XXL-Rehab Barton Transferstol

XXL-Rehab Patient/Dinner chair

XXL-Rehab Rollator King


XXL-Rehab Rollator Support

XXL-Rehab Stand Tall

XXL-Rehab Bariatric walking frame

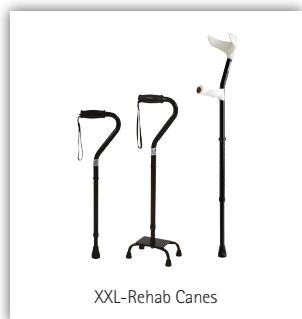XXL-Rehab Canes

XXL-Rehab Multifunktional Scale


XXL-Rehab Lift and Slings

XXL-Rehab 400 bariatric hospital bed

XXL-Rehab Home Care Bed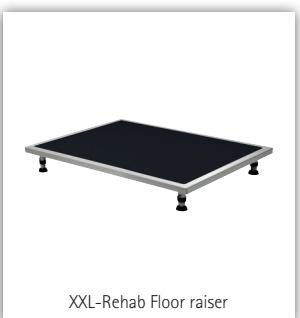

XXL-Rehab Floor raiser

XXL-Rehab  
Comfort Foam Mattress

XXL-Rehab  
Low Air Loss mattress

XXL-Rehab  
Alternating mattress

XXL-Rehab Bariatric suit

## BRUGSANVISNING · USER MANUAL

**XXL REHAB®**

XXL-Rehab · Fuglebækvej 1D · DK-2770 Kastrup · Denmark  
Telephone: +45 7025 2522 · Fax: +45 7025 2544  
[info@xxl-rehab.com](mailto:info@xxl-rehab.com) · [www.xxl-rehab.com](http://www.xxl-rehab.com)