

User Instruction for the Zenith Bariatric Commode X148



1. Introduction

This product is available with an oval potty with handle and caters for clients up to 381kg/60 stone. It is designed for ultimate comfort – with soft-feel polyurethane moulded over seat, back and arm pads and an adjustable back to cater for client reclined seating. The back is also detachable, as are the arms – for lateral transfer. The height can be simply adjusted by the use of remove/re-fix pin clips. The rear of the seat is scalloped to allow for personal hygiene. The transit wheels facilitate tipping of the frame and easy moving and the rubber feet allow for none-slip, none-marking.

Product Dimensions	
Seat height (from-to)	470mm-572mm
Width between arms	665mm
Seat to top of arms	235mm
Seat to top of back	420mm
Orifice depth	300mm
Maximum footprint	760mm x 595mm

2. Contents of the Box

- Zenith commode with pan (X148)
- Potty with handle (71019F/H)
- Extended Legs & Pin Clips
- Transit Wheels
- Polyurethane Moulded Over Seat
- Detachable Back & Arms
- User Instructions (this leaflet)

3. Assembly Instructions

- Carefully unpack your commode and examine the product to ensure it is complete.
- Before use check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur please contact your supplier to exchange and refrain from using the product. **DO NOT ATTEMPT TO USE IF PRODUCT IS DAMAGED.**


4. Correct Use

Wheels:

The product is supplied with two transportation wheels to allow easy manoeuvring. When the wheels have been supplied pre-fitted, they may have been positioned inwards for delivery – in this case they will need to be repositioned before product use.

To do this:

1. Remove the pin clip by pulling it outwards from the leg
2. Adjust the extension leg to the desired height by aligning another of the adjustment holes in the extension leg with the leg of the frame
3. Refit the pin clip ensuring it goes fully through and clips into position
4. Repeat for the other legs ensuring that all are at the same height

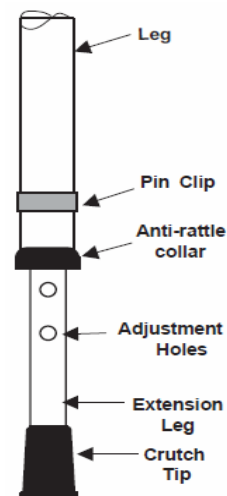

Arm Removal:

To remove each arm pull the spring plunger outwards whilst at the same time removing the arm by pulling upwards. To refit, pull the plunger outwards whilst pushing the arm into place ensuring it seats fully before releasing the plunger.



Backrest Adjustment:

To re-position the backrest, slacken the two thumbwheels underneath the seat and adjust the backrest in or out as required. Ensure that the thumbwheels have been re-tightened before use.



71019F/H Potty:

This product was developed for greater ease of use. The rear handle was designed intended to allow easy fitting and removal from the commode and greater control when carrying and emptying. The upper handle when put in the upright position locks into position for stability. To fold it down, push the handle vertically downwards to release the locking tabs before folding it backwards towards the rear handle.

- To fit the pan, ensure that the pan handles are lifted clear of the support brackets then slide the pan across the rear supporting brackets fitted to the commode until contact is made with the front of the bracket.
- To remove the pan, ensure the handles are clear of the support brackets and then slide the pan out from the rear of the commode.
- The upper handle will require to be folded down before fitting to the commode.



Handle folded down



Handle raised for carrying

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.

Use of power/jet washing & industrial washing machines:

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal moisture evaporates).
- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependant on the manufacturer's instructions.

6. Maintenance Requirements

- The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for users of a maximum of 380kg/60stone in weight.
- We recommend a full risk assessment should be carried out before use

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare
Cefndy Road
Rhyl
Denbighshire
LL18 2HG
Tel: 0044 1745 343877 (Quality Assurance Dept)
E-mail: cefndy.sales@denbighshire.gov.uk