Dyna-Form® Overlay Superior

Instructions for Use - English
Bruksanvisning- Svenska

REF

MAT2100001
MAT2100002
MAT2100003
MAT2100004

Table of Contents

English 03

Svenska 00

Intended Use of the Product

The Dyna-Form Overlay Superior is a pressure redistributing foam mattress intended to be used in the prevention or treatment of pressure ulcers. The mattresses is suitable for persons who are at high risk of pressure ulcers, including those with category 1 or category 2 pressure ulcers, alongside an individualised care plan.

Indications for Use

The Dyna-Form Overlay Superior is intended to be used by all kinds of users, including lay persons, in all healthcare environments including professional healthcare facilities and home healthcare.

The Dyna-Form Overlay Superior is an overlay mattress, and therefore should be used in combination with a secondary lower mattress. The Dyna-Form Overlay Superior should not be used as a pressure redistributing mattress on its own, without a secondary mattress below.

Contraindications for Use

There are no known contraindications for use.

Safety Information

Visual inspection

- Inspect the packaging for any damage.
- Check that the correct product has been delivered.
- Check to ensure that the mattress is free from damage.

Always Read the User Manual

- Always read the user manual before use.
- Keep the user manual where it is accessible to users of the product.
- Always ensure that you are referring to the latest version of the user manual.
- The most recent version of the user manual is available to download from the DHG website: www.dhg-healthcare.com
- It is strongly prohibited to modify the original product.

Warnings and Cautions

Warnings

- Which mattress is suitable for your individual use should be decided by your clinician who will also determine the most appropriate pressure ulcer prevention or treatment plan and the relevant nursing procedures.
- Always ensure that the mattress is the correct size for the required bed.
- The usage of additional waterproof mattress covers, electric blankets, sheepskins, or other similar items which may reduce the effectiveness of the pressure redistributing properties of the mattress or lead to an additional increase in body temperature such as to cause skin maceration is not advisable.
- As this is an overlay product, ensure that the mattress is securely fastened to the bottom mattress, using the elasticated straps provided, before a user lies on the surface.
- Ensure that no sharp objects damage the mattress or mattress cover.
- The printed side of the mattress cover must always be facing upwards. Head/foot alignment is indicated on the cover.
- Always ensure that the distance between the top surface of the Dyna-Form Overlay Superior and the top of the bed side rails is ≥ 20 mm.
- Inspect the inner and outer surfaces of covers, seams and zips regularly for signs of damage.
- If the cover is stained, soiled, or torn, the foam core should also be examined.

Cautions

- It is recommended that an audit of all mattresses is carried out at least annually.
- Mattresses can become damaged if stored incorrectly. It is advisable that a mattress is stored in a clean and dry environment. It is not advised that mattresses are stored near heating devices, such as electrical fires or radiators.
- Take care when handling the mattress to ensure no damage to the cover or foam.
- Avoid contact with abrasive surfaces, and do not drag the mattress.

Symbols

	Read the user manual
	Medical device
	This product conforms to the requirements of Medical Device Regulation (EU) 2017/745
	This product conforms to the requirements of UK MDR 2002
	General warning symbol
	Wash at 80°C
	Tumble dry on low
	Do not dry clean
	Do not bleach
	Do not iron
	No smoking
	Maximum user weight
	Do not use sharp instruments
	Do not use phenol

Installation

The mattress should only be installed and used by a clinician or a trained lay operator. A lay operator shall be considered trained once they have fully read this user manual.

Remove all packaging and protective material. Avoid using any sharp instruments to remove this packaging as this could damage the cover. If the mattresses has not been delivered vacuum packed, it can be used immediately. If the mattresses has been delivered compressed and vacuum packed, it is advised that the mattress should lay unpacked, on a flat surface, for at least 2 hours before it is used.

Place the Dyna-Form Overlay Superior on top of the lower mattress and use the four (4) elastic straps located in each corner of the mattress to secure the overlay to the lower mattress. It is extremely important that the overlay is secured to the lower mattress in this fashion prior to a user lying on the mattress.

Typically, the user will be placed in a supine position on the mattress.

Wipe the mattress down before covering the mattress with a loose-fitting sheet.

Cleaning and Maintenance

Mattress cleaning should take place before and after use, and between patients.

In the Case of Light Soiling

In the absence of gross contamination or unusual risk, the removal of dirt and spillages with clean, warm water and neutral detergent should be sufficient. However if required the cover can be wiped down with a solution of Sodium Hypochlorite diluted to 1,000ppm available chlorine. Ensure that the cover is rinsed with clean water using a single use non-abrasive cloth, and thoroughly dried. Ensure adequate ventilation.

In the Case of Heavy Soiling

In cases of gross contamination the mattress cover should be cleaned first with detergent and water then with a solution of Sodium Hypochlorite diluted to 10,000ppm available chlorine. Rinse well with clean water and a damp single use non-abrasive cloth. Ensure adequate ventilation.

Machine Laundering

If required, the Mattress Cover may be removed and machine-washed at a temperature of 80°C, for not less than 10 minutes.

Drying

To avoid shrinkage of the cover line dry in an indoor clean environment or tumble dry on a low heat setting not exceeding 40°C and for not longer than 10 minutes. Covers must be thoroughly dried before re-fitting to the mattress.

Maintenance

Inspect the inner and outer surfaces of covers, the seams, and zips regularly for signs of damage. If the cover is stained, soiled or torn, the foam core should also be examined.

It is recommended that an audit of all mattresses is carried out annually.

Re-Use, Storage and Disposal

Re-Use

The product is reusable. Before using the product again with a new user, it is recommended to clean the product using the above instructions.

Storage

It is advisable to store the mattress off the floor in a clean, dry environment. It is advisable to store the mattress flat. Do not store the mattress near heating devices. Do not store the mattress where it might be prone to damage.

Disposal

All contaminated mattresses must be disposed of as clinical waste in accordance with regional regulations and local legislation and guidelines.

Technical Information

Warranty	3 years cover, 3 years foam
Fire Testing	BS 7177, Low Hazard
Foam Density (kg/m ³)	39
Nominal Hardness (N)	120
Maximum User Weight	165kg
Weight of Product	6kg
Expected Life of Mattress	5 years

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Direct Healthcare Group Ltd
Direct Healthcare Group Sverige AB
Direct Healthcare Group Pty Ltd.,