Instructions for Use - English
Bruksanvisning – Svenska
Brugsvejledning – Dansk
Käyttöohje – Suomi
Brukermanual – Norsk
Handleiding – Nederlands
Manuel d'utilisation – Français
Gebrauchsanweisung – Deutsch
Manuale utente – Italiano
Manual de usuario – Español

REF

MAT05

Table of Contents

English	03
Svenska	07
Dansk	11
Suomi	15
Norsk	19
Nederlands	23
Français.....	27
Deutsch	31
Italiano	35
Español	39

Intended Use of the Product

The Dyna-Form Static Air HZ is a pressure redistributing foam mattress intended to be used in the prevention or treatment of pressure ulcers. The mattresses is suitable for persons who are at high risk of pressure ulcers, including those with category 1, category 2, or category 3 pressure ulcers, alongside an individualised care plan.

Indications for Use

The Dyna-Form Static Air HZ is intended to be used by all kinds of users, including lay persons, in all healthcare environments including professional healthcare facilities and home healthcare.

The Dyna-Form Static Air HZ is a mattress replacement system, and should be placed directly onto the bed surface.

Contraindications for Use

There are no known contraindications for use.

Safety Information

Visual inspection

- Inspect the packaging for any damage.
- Check that the correct product has been delivered.
- Check to ensure that the mattress is free from damage.

Always Read the User Manual

- Always read the user manual before use.
- Keep the user manual where it is accessible to users of the product.
- Always ensure that you are referring to the latest version of the user manual.
- The most recent version of the user manual is available to download from the DHG website: www.dhg-healthcare.com
- It is strongly prohibited to modify the original product.

Warnings and Cautions

Warnings

- Which mattress is suitable for your individual use should be decided by your clinician who will also determine the most appropriate pressure ulcer prevention or treatment plan and the relevant nursing procedures.
- Always ensure that the mattress is the correct size for the required bed.
- The usage of additional waterproof mattress covers, electric blankets, sheepskins, or other similar items which may reduce the effectiveness of the pressure redistributing properties of the mattress or lead to an additional increase in body temperature such as to cause skin maceration is not advisable.
- Ensure that no sharp objects damage the mattress or mattress cover.
- The printed side of the mattress cover must always be facing upwards. Head/foot alignment is indicated on the cover. Ensure that the Heel Zone area (identified by a symbol of two feet on the mattress cover) is located at the bottom of the bed where the user's heels will be placed.
- Always ensure that the distance between the top surface of the Dyna-Form Overlay Superior and the top of the bed side rails is $\geq 20\text{mm}$.
- Inspect the inner and outer surfaces of covers, seams and zips regularly for signs of damage.
- If the cover is stained, soiled, or torn, the foam core should also be examined.

Cautions

- It is recommended that an audit of all mattresses is carried out at least annually.
- Mattresses can become damaged if stored incorrectly. It is advisable that a mattress is stored in a clean and dry environment. It is not advised that mattresses are stored near heating devices, such as electrical fires or radiators.
- Take care when handling the mattress to ensure no damage to the cover or foam.
- Avoid contact with abrasive surfaces, and do not drag the mattress.

Symbols

	Read the user manual
	Medical device
	This product conforms to the requirements of Medical Device Regulation (EU) 2017/745
	This product conforms to the requirements of UK MDR 2002 for medical devices MDD/93/42 EEC
	General warning symbol
	Wash at 80°C
	Tumble dry on low
	Do not dry clean
	Do not bleach
	Do not iron
	No smoking
	Maximum user weight
	Do not use sharp instruments
	Do not use phenol

Installation

The mattress should only be installed and used by a clinician or a trained lay operator. A lay operator shall be considered trained once they have fully read this user manual.

The mattress packaging will include the mattress and its associated cover already fitted. Remove all packaging and protective material. Avoid using any sharp instruments to remove this packaging as this could damage the cover. If the mattresses has not been delivered vacuum packed, it can be used immediately. If the mattresses has been delivered compressed and vacuum packed, it is advised that the mattress should lay unpacked, on a flat surface, for at least 2 hours before it is used. If the mattress has been left in storage, it is possible that the heel zone cells may have deflated. Prior to placing a user on the mattress, ensure that the air-only heel zone cells are appropriately inflated by repeatedly applying pressure to the foam-in-air cells located in the main torso area of the mattress. This will begin to inflate the air-only heel zone cells. The mattress has a self-inflating system, and requires no input or maintenance to ensure inflation of the heel zone cells during regular use.

The mattress should be placed directly onto the bed surface. It is permissible to use the mattress with adjustable, profiling beds.

Typically, the user will be placed in a supine position on the mattress. The user of the mattress should not be an immobile user. Frequent movement by the user ensures that the heel zone remains inflated.

Wipe the mattress down before covering the mattress with a loose-fitting sheet.

If the cover needs to be removed, this is easily done by locating the zip in the bottom left corner and unzipping along the three (3) sides of the mattress. This will allow for the mattress cover to be completely removed.

Cleaning and Maintenance

Mattress cleaning should take place before and after use, and between patients. The mattress can be used by more than one person, provided it is decontaminated as per below instructions, in between each user.

In the Case of Light Soiling

In the absence of gross contamination or unusual risk, the removal of dirt and spillages with clean, warm water and neutral detergent should be sufficient. However if required the cover can be wiped down with a solution of Sodium Hypochlorite diluted to 1,000ppm available chlorine. Ensure that the cover is rinsed with clean water using a single use non-abrasive cloth, and thoroughly dried. Ensure adequate ventilation.

In the Case of Heavy Soiling

In cases of gross contamination the mattress cover should be cleaned first with detergent and water then with a solution of Sodium Hypochlorite diluted to 10,000ppm available chlorine. Rinse well with clean water and a damp single use non-abrasive cloth. Ensure adequate ventilation.

Machine Laundering

If required, the Mattress Cover may be removed and machine-washed at a temperature of 80°C, for not less than 10 minutes.

Drying

To avoid shrinkage of the cover line dry in an indoor clean environment or tumble dry on a low heat setting not exceeding 40°C and for not longer than 10 minutes. Covers must be thoroughly dried before re-fitting to the mattress.

Maintenance

Inspect the inner and outer surfaces of covers, the seams, and zips regularly for signs of damage. If the cover is stained, soiled or torn, the foam core should also be examined.

It is recommended that an audit of all mattresses is carried out annually.

Re-Use, Storage and Disposal

Re-Use

The product is reusable. Before using the product again with a new user, it is recommended to clean the product using the above instructions.

Storage

It is advisable to store the mattress off the floor in a clean, dry environment. It is advisable to store the mattress flat. Do not store the mattress near

heating devices. Do not store the mattress where it might be prone to damage.

Disposal

All contaminated mattresses must be disposed of as clinical waste in accordance with regional regulations and local legislation and guidelines.

Technical Information

Warranty	5 years
Fire Testing	BS 7177, Medium hazard
Foam Density (kg/m ³)	U-Core: 38-40 Insert: 38-40
Nominal Hardness (N)	U-Core: 200 Insert: 120
Minimum User Weight	0kg
Maximum User Weight	254kg
Weight of Product	10kg
Expected Life of Mattress	8 years
Sizes	Length: 120cm-220cm Width: 60cm-120cm Height: 15cm

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Direct Healthcare Group Ltd
Direct Healthcare Group Sverige AB
Direct Healthcare Group Pty Ltd.,