# Shower stool round

Manual - English

Manual - Svenska

Manual - Danske

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**Art.no.:** 200160001

**SWL: 150 kg/330 lbs**

**Manual no:** LI09509.409 ver.1 131120

**EN Visual inspection**

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## **EN - Please read the instructions before you start using this product**

### **Purpose**

This instruction applies to shower stool with round seat. The shower stool is mostly used in the bathroom. It can be used to sit on at the bathroom sink, but it can also be used as a shower seat. The stool is very suitable in places with little space.

### **Adjustment – see drawing, page 10**

The shower stool is height adjustable in 10 steps. The adjustment takes place by detaching the snaplocks from the legs one by one and selecting the desired height. Be sure to adjust the four legs to the same height and to press the snaplocks firmly onto the tubes again.

### **Maintenance**

To be wiped clean after use. Clean the stool with domestic cleaning materials without abrasive or corrosive properties. Solvents and chlorine should not be applied.

### **Caution**

- Max load: 150 kg.
- Only use the shower stool as described in the paragraph "Purpose".
- Do not stand on the shower stool!
- Do not push or pull the shower stool over the floor while the user is sitting on it.
- Check the correct position of the snaplocks before using the stool.
- Check the condition and the correct position of the snaplocks frequently.
- For optimal safety it is recommended to check the ferrules frequently and to replace them whenever necessary.
- Ensure that all legs have the same height.

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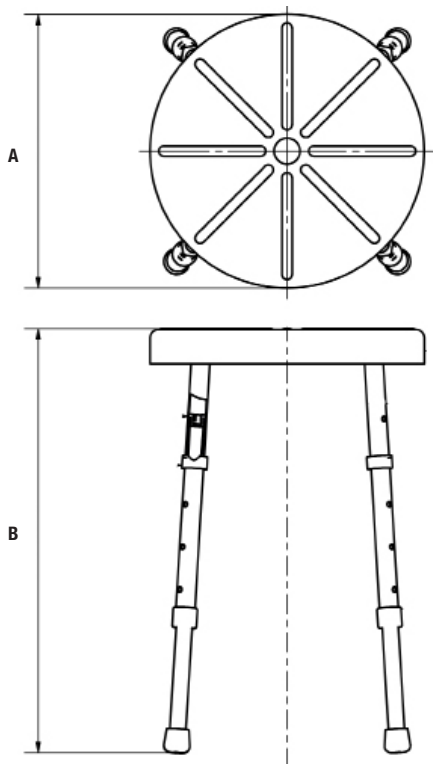
***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

The Handicare Group is one of the leading healthcare companies in Europe with own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, Poland, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various power and manual wheelchairs, scooters, seating systems, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise- and recline chairs, and bathing and toileting products.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com)

Handicare AB  
Veddestav. 15, Box 640  
SE-175 27 Järfälla  
SWEDEN

Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)

Internet: [www.handicare.com](http://www.handicare.com)