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EN - Please read the instructions before using the product.

These instructions apply to the armrest lock for the Combi chair.

Intended use

The armrest lock can be mounted as an accessory onto all Combi chairs, and insures that the armrests cannot inadvertently fold up, for example, in the Combi Manual Tilt Chair and the Combi Power Tilt Chair, when the chairs are tipped over backwards. The armrest lock can be used to advantage together with the front cross bar combi chair.

Installation

Loosen and remove the two front screws on the armrest. Mount the armrest lock on the armrest using the supplied longer screws (1) and the nylon rings (2). Place the two rings on both sides between the U-profile of the armrest and the armrest lock. The bend in the armrest lock must point backwards, as shown in the sketch.

Adjustment and use

Press down the armrest lock vertically under the armrest to insure against inadvertently swinging up of the armrest. Be sure that the armrest lock is pressed down completely to the frame for optimal use.

Pull the lock backwards, to lift up the armrest. The armrest can then be moved freely.

Maintenance

The armrest lock can be cleaned with ordinary cleaning products without polishing or etching properties. It is not recommended to use solvents or chlorine. Dry off with a clean cloth afterwards.

Warning!

- Must only be used as described under the section "Intended use".
- Should always be installed when the users are agitated or overly active.
- Check regularly that the screws are tightened.
- Make certain that the armrest lock is pressed completely down to the frame for optimal use.
- Be aware of the danger for being pinched when the armrest is being folded up or down.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com