# BackSupport

SystemRoMedic™

---

User manual – English

Bruksanvisning – Svenska

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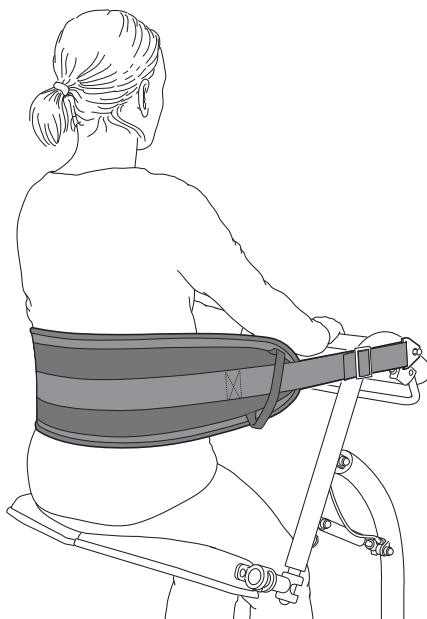
**REF**

70200037

---

**SWL: 170 kg/375 lbs**

## Table of contents

<b>Always read the user manual</b> .....	<b>3</b>
<b>English</b> .....	<b>4</b>
<b>Svenska</b> .....	<b>5</b>
<b>Norsk</b> .....	<b>6</b>
<b>Danske</b> .....	<b>7</b>
<b>Suomi</b> .....	<b>8</b>
<b>Deutsch</b> .....	<b>9</b>
<b>Netherlands</b> .....	<b>10</b>
<b>Français</b> .....	<b>11</b>
<b>Italiano</b> .....	<b>12</b>
<b>Español</b> .....	<b>13</b>

### **EN Always read the user manual**

Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

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English

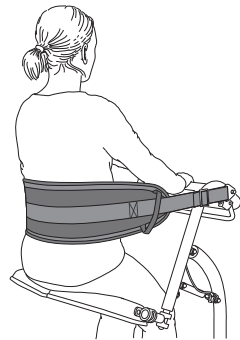
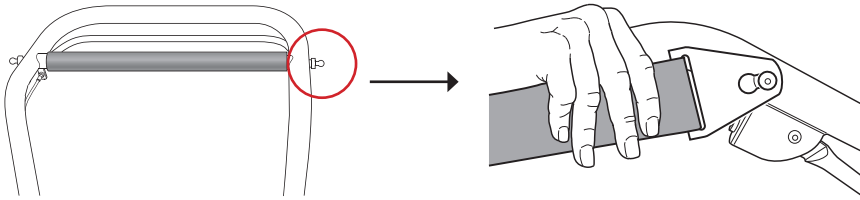
## NOTE!

Read the manual for QuickMove before use.


Adjust the length of the BackSupport to fit the user. Use of the strap should give the patient support and a sense of security.

## Assembling QuickMove

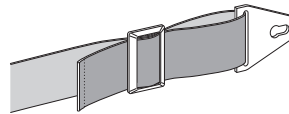Remove the clasps before laundering and replace them, as illustrated, after laundering.

## Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

**CE** Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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## Vida útil estimada

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# SystemRoMedic™

## Simple solutions for great results

SystemRoMedic™ is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

# Direct Healthcare Group

Advancing Movement & Health

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)