Backrest Linido

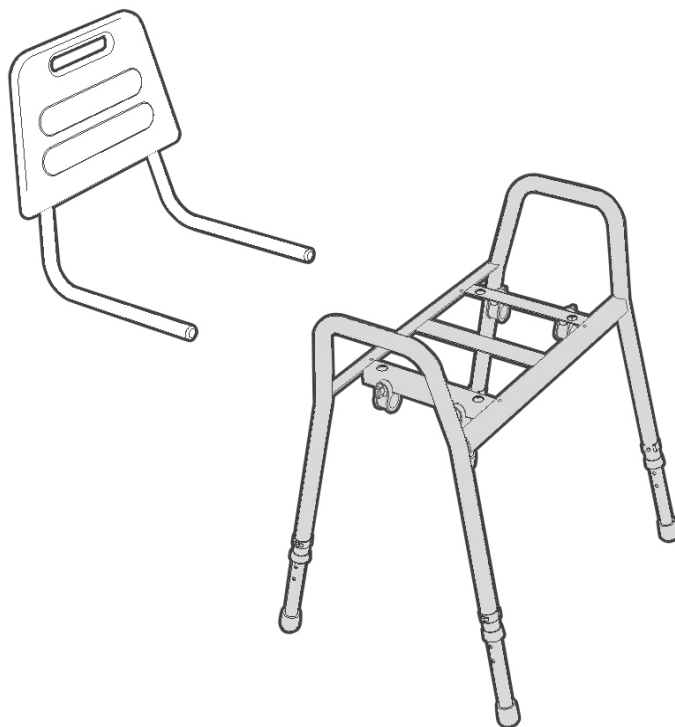
Manual - English

Manual - Svenska

Manual - Danske

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Art.no.: 200180312 / LI2002.0014-02

MAX: 150 kg/330 lbs

**EN Visual inspection**

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EN - Please read the instructions before you start using this product

Intended use

For optimal safety, Direct Healthcare Group's Shower stool soft seat (Art.no.: 200501312/200506312) and Shower stool plastic seat (Art.no.: 200500312) can be provided with a backrest.

The backrest is made of polyurethane, a material with good non-slip properties.

Assembly – see drawing

In case your shower stool has no welded rings under the seat, please follow the instructions A. In case the rings are already welded to the shower stool, please follow the instructions B.

Maintenance

To be wiped clean after use. Use domestic cleaning materials without abrasive or corrosive properties. Solvents and chlorine should not be applied.

The backrest slats can be detached for cleaning. Polyurethane can tolerate temperatures up to 85 °C / 3 minutes.

The backrest frame is made of stainless steel.

Warning!

- Max. load on stool with backrest: 150 kg.
- Only use the backrest as described in the paragraph "Intended use".
- When the backrest is mounted in position it is not necessary to place the stool against a wall.
- Check the correct position of the snaplocks before use.
- Check the condition of the snaplocks frequently. Regularly check that the snaplocks are correctly inserted into the tubing.

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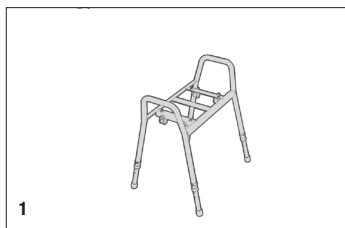
EN 1. Follow these instructions for a stool without welded rings. Remove the seat from the frame.

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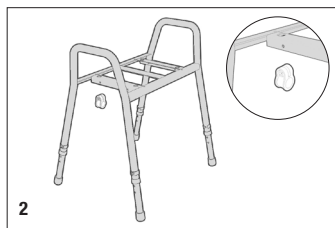
EN 2. Place the holder on the frame (4x). Make sure the holes are lined up.

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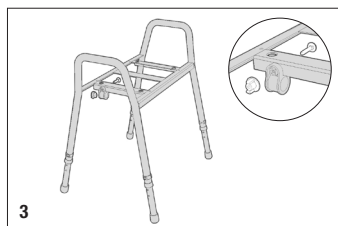
EN 3. Place the bolt through the frame and the holder, starting from the inside. Screw it together with the wingnut.

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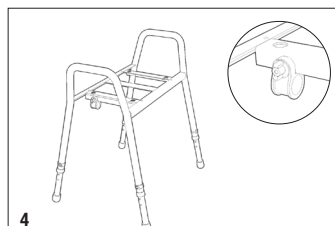
EN 4. Check if the wingnut is firmly tightened.

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EN 5. Mount the backrest by inserting the tubing through the holders.

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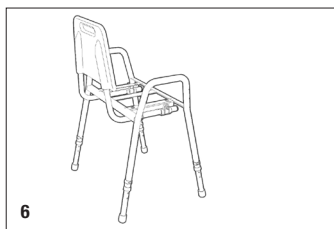
EN 6. Position the backrest to the correct seat depth. Attach the snaplocks, one on each side of the rings.

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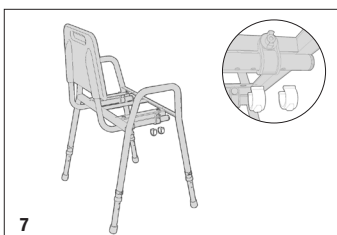
EN 7. Check if the clips are completely pressed against the tube.

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EN 8. Reattach the seat in its original position.

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EN 1. Follow these instructions for a stool with welded rings attached.

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EN 2. Mount the backrest by inserting the tubing through the holders.

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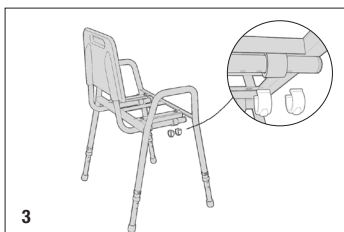
EN 3. Position the backrest to the correct seat depth. Attach the snaplocks, one on each side of the rings.

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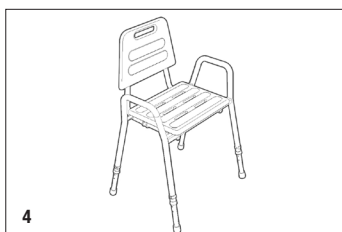
EN 4. Check if the clips are completely pressed against the tube.

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Maintenance

EN Clean regularly with normal household detergent. Do not use aggressive cleaners. Direct Healthcare Group is not liable for damage caused by improper mounting/use.

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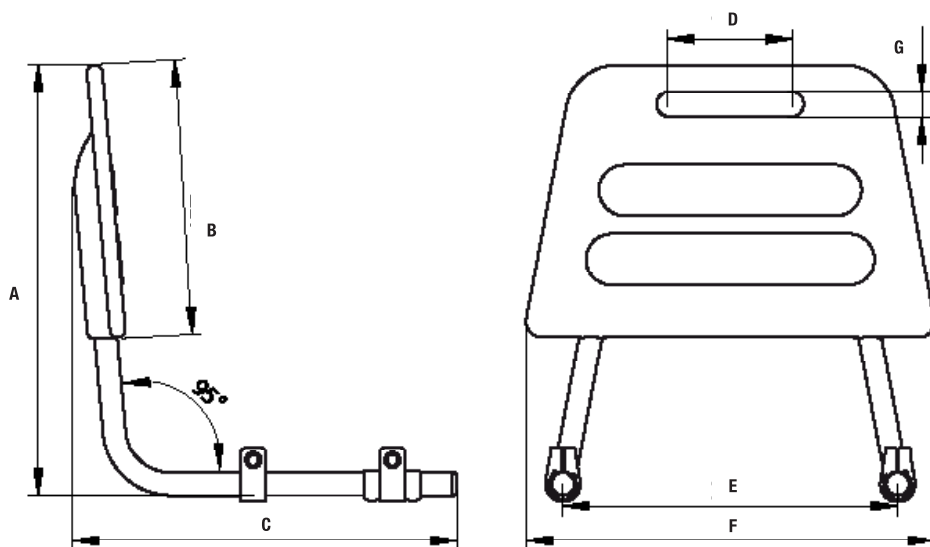
Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com