
Commode/shower chairs soft seat Shower chairs textile seat Swimming pool hoist chair

User manual - English
Bruksanvisning - Svenska
Handleiding - Nederlands

Linido

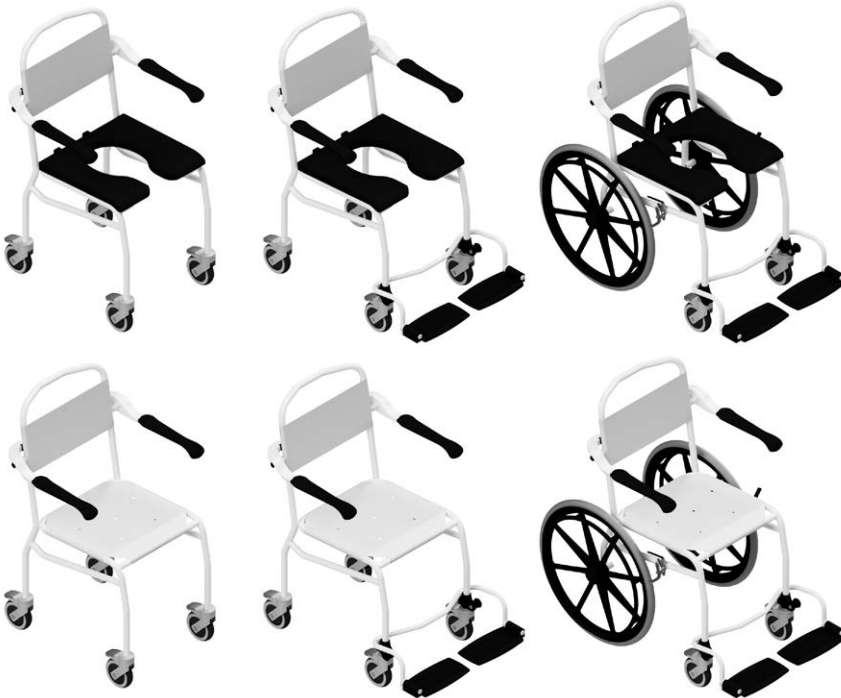REF

LI2136,
LI2137,
LI2138,
LI2140,
LI2141,
LI2142

 200 kg/440 lbs

Table of contents

English	3
Svenska	7
Nederlands	11
Assembly instructions / Monteringsanvisningar / Montage-instructies	16
How to use / Använda enheten / Hoe te gebruiken	17
Dimensions / Mått / Afmetingen	18
Symbols / Symboler / Symbolen	23
Accessories / Tillbehör / Accessoires	23

Commode/shower chairs soft seat

Shower chairs textile seat

Swimming pool hoist chair

Linido

Instructions for use
and user

1. INTENDED USE, USER AND ENVIRONMENT

The (Self-propelled) Shower and Toilet seating Aids are mobile chairs to be used for taking care of persons with little or reduced muscular function. Taking care includes for example actions of assistance with bathing, washing, showering and / or toilet activities. In the specific case of the Swimming Pool Hoist Chair, use in swimming pools and / or water related physical therapy sessions is part of the Intended use.

The devices are mobile to allow transport to and from the bathroom area and transfers and movement within the bathroom area. The devices allow the patient physical support during showering, washing, drying and dressing in a safe and comfortable sitting position. The armrests will give support for sitting up, and swivel away for easy sideways transfers.

All devices equipped with keyhole shaped seat surfaces allow the use in combination with a regular toilet.

All bathroom safety devices have a maximum safe working load as indicated on the labels and in the IFU. All devices can be tailored to the patient's and caregivers needs with the use of accessories.

The Bathroom Safety Products can be operated and propelled by the patient user or are to be operated by caregivers. Both should have access to and the ability to read and understand the IFU of the products. Apart from the advantages for the patient, the Bathroom Safety products reduce the need for manual lifting or handling of the patient, thus making the situation safer and reducing caregiver work-related injuries. It allows the caregiver to assist the patient elevated from the floor, which improves communication and eliminates the need for the caregiver to work very close to the floor. To new users, users with reading or cognitive disabilities, training or instruction on the proper use of the device is recommended. There are no manuals available for users with visual or cognitive disabilities.

The Linido Swimming Pool Hoist Chair is the only product in this product line with the additional intended use to ride or hoist a patient in and out of a swimming pool. The product may be driven into the water by a caretaker. The supporting surface of the product may be hoisted by being attached to a suitable and sufficiently strong lifting device. The lifting device may be attached by means of the 4 lifting eyes welded on to the chair.

Intended User

In general, the devices are to be used by:

- Conscious patients or disabled persons with reduced or little muscular function who are not subjected to the care of a caregiver. The patient is capable of performing part of the daily hygiene activities without assistance, but limited in endurance.
- Conscious patients or disabled persons with reduced or little muscular function who subjected to the care of a caregiver but may be able to contribute to the action or perform part of the action independently.
- Patient: Person undergoing a medical procedure. Person recovering from a medical procedure. Person suffering from physical impairments due to age related ailments such as with weak legs, low stamina, tremors and/or poor balance.
- Disabled person: Person with one or more impairments, one or more activity limitation, one or more participation restriction or a combination.
- Caregivers with medical training: healthcare professionals with medical education of different levels, who operate the device, assisting a patient. Have knowledge of intended medical indication and how to approach a patient. Are literate and can read and interpret instructions for use.
- Caregivers without medical training: laypersons with no medical education, who operate the device, assisting a patient. Are literate and can read and interpret instructions for use, have received training from DHG or DHG certified trainers on how to operate the device.
- Service personnel: technically savvy personnel at professional healthcare facility or assisting caregivers in home healthcare environment with repairs of the device. Service personnel have received training from DHG or DHG certified trainers on how to repair the device.
- DHG Service Technicians: technically savvy DHG personnel with expertise in repairing and servicing the device.

Intended Environment

Linido products and accessories are intended to be use in Home Healthcare environment and Professional Healthcare facility environment.

- Home Healthcare environment is limited to indoor domestic areas (where the products are used to alleviate or compensate for an injury, disability, or disease) and public use (such as public restrooms, swimming pools, hotels)
- Professional Healthcare facility environment includes long-term care (nursing homes/residential care, rehabilitation and geriatric facilities), clinics, limited care facilities and hospitals (patient rooms, emergency rooms, intensive care, surgery rooms except near HR surgical equipment, outside the RF shielded room)
- The intended use environment does not include extreme climate indoor areas, such as sauna's and spa's and outdoor areas such as beaches or lakes.

2. CONTRAINDICATIONS, PRECAUTIONS AND WARNINGS

We hope that our product meets or exceeds your expectations. This instruction manual contains the information that users of the product need to know for safe operation. In addition to the instructions for use, there is a technical manual that describes the structure of the device and contains information of a more technical nature and a list of spare parts.

PLEASE READ THIS INSTRUCTION MANUAL BEFORE USING THE PRODUCT.

Failure to comply with the following points of attention and the further descriptions in this manual can lead to dangerous situations. Improper and careless use can cause damage, and thus also lead to dangerous situations. In such situations, decommission the chair and contact the supplier.

Contraindications

- The device may not be used for patients above the maximum weight indicated on the label of the device and this user manual.
- It is not allowed to lift anything else besides the intended patient population.
- The device is only to be used according to the intended use. Any other purpose or use is not permitted.
- The device shall not be used on inclined surfaces with an angle of more than 5 degrees.

Precautions

- Treat the device with care and ensure a clean, tidy working environment.
- Check the condition and functions of the device regularly.
- Service and maintenance of the device is only permitted by demonstrably authorized persons. The lifetime of the device can be decreased by improper maintenance.
- The replacement structural parts of the device is only permitted by demonstrably authorized persons.
- Operating the device is only permitted by demonstrably authorized persons.
- When moving the device, always check that the space where the transfer takes place is free of obstacles.
The device must not jam or remain stuck.
- The device should be inspected periodically at least once a year.
- Always report defects on the device to the supplier.
- When lifting, always check that there is enough space above the device.
- Keep the manual where it is accessible to users of the device.
- Fold the armrests down when a patient is on the chair or during lowering / rising (even without patients present).
- For optimum Safety: tighten expander bolts on all wheels before use and again after 4 weeks. Check regularly thereafter.

Warnings

- Never leave the patient unattended during a transfer situation.
- Only use the device with approved accessories.
- For service and repair, use only original parts supplied by Direct Healthcare Group.

- Do not use the device if defective. If there is a red sticker with 'rejection' on the device, the device may not be used either.
- If deformations have occurred on the chair due to incorrect use or transport, it may no longer be used and must be returned to the supplier.
- Do not use any of the accessories when its condition is insufficient.
- The device must not be left or stored in a damp or humid environment.
- The device must not be cleaned using steam.
- The device may only be used indoors.
- Always keep the hands, legs and feet clear from the moving and folding parts.
- Resistance of ignition of the upholstered parts complies to BS-EN 1021-1 and -2. Please keep the device away for open fire, flames or direct sources of high heat.

3. BEFORE EVERY USE

1. Visually inspect the product if it is free from damage and properly assembled. The occurrence of the following must be checked.

- Cracks in the frame
- Cracks in the plastic parts
- A bended frame
- Bended legs
- Damage to the castors or brakes
- Tears or irregular holes in the (HT) cover material.

If any of these points apply, the product should be discarded, or parts must be replaced with spare parts. Dispose the discarded products or parts as environmentally as possible according to the local regulations.

2. Check the product functions by lifting, lowering or tilting the device.

3. Check movability in all directions. It should take no more than approx. 10 kg (100 N) of force to move the device. If it feels there is more force required, check whether there is anything blocking the device or if the brakes are still on.

The device is ready for use. Always maintain a neat and clean work environment.

Operation Control Forces

Operation Control	Maximum required Force
Hand control buttons	0,2 kg / 2 N
Foot Pedal	30 kg / 300 N
Castor brakes pedal	30 kg / 300 N
Moving/handling the device	10 kg / 100 N

Product Label

4. MAINTENANCE

The chair is designed to keep maintenance to a minimum. It is advised to follow the maintenance schedule below:

- Daily and weekly maintenance can be done by caregivers or service staff of an authorised reseller.
- Semi-annual maintenance should be performed by the technical service staff of an authorised reseller
- Annual inspection and any repair thereof should be performed by the technical service department trained to service and repair this equipment.

The expected lifetime of the product is 5 years.

4.1 Daily Maintenance

The following operations can be performed by the caregivers or service staff on a daily basis.

- Rinse or clean the parts after each use.
- It is recommended to dry the cover after each use for hygiene purposes.
- The device may be cleaned regularly using normal household detergent. If disinfection is required, use a product intended for cleaning of such products. Use a detergent with a maximum of 45% alcohol or max 5% Salicylic Acid.
- The product is not intended to be sterilised or cleaned by steam.
- Rinse with clean (mains) water after using detergents or cleaning agents and after use in or around swimming pools (if applicable).

 *When in doubt on the condition of the device, always contact the technical service of your authorised distributor!*

4.2 Weekly Maintenance

The following operations can be performed on a weekly basis.

- It is recommended to clean the chair weekly. This is best done with household detergent and a well-squeezed damp cloth.
- Check that all movable parts move easily and quietly.
- Check the readability of the serial number label, and the availability of a manual.

4.3 Semi-Annual Maintenance

The list below provides an overview of the maintenance that the technical service can perform on the chair every six months.

1. The movement of all moving parts of the chair should be smooth.
2. No damage to plastic parts is allowed.
3. The castors should turn easily and that the brake should work properly.
4. All bolts must be properly tightened and not loose.

 *Service and maintenance of the chair is only permitted by demonstrably authorized persons.*

 *If the chair does not comply with one or more of the above points, the lift must be temporarily put out of operation and an authorised distributor must be contacted.*

5. TROUBLESHOOTING

5.1 Troubleshooting table

Complaint	Possible Cause	Action
Brake does not work	Wheels broken.	Contact supplier or technical service.
Lifting function does not work.	Hydraulic pump broken.	Contact supplier or technical service.
Tilting function does not work.	Gas spring broken.	Contact supplier or technical service.
Armrest / side fence jams	Armrest mechanism broken	Contact supplier.

 *If the repair table does not resolve the complaint or in case of doubt, do not use the device and contact the supplier.*

5.2 Replacement/Removal of Parts

All parts that are replaced or removed (including batteries) and thus need to be discarded can be returned to DHG / Linido. It is also possible to offer these parts to the mechanic for destruction. We will ensure that all parts are processed in an environmentally friendly manner.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

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Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

DHG

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com