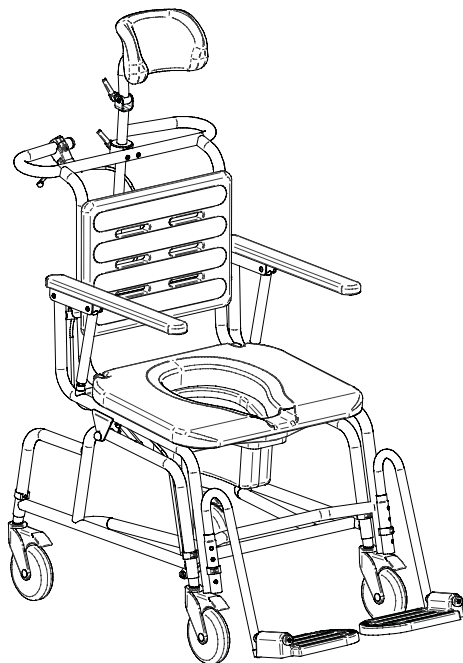
Combi, commode/shower chair, manual tilt Linido

User manual - English
Bruksanvisning - Svenska
Handleiding - Nederlands

REF

201941314

 150 kg/330 lbs

Table of contents

English	3
Svenska	11
Netherlands	19
Symbols / Symboler / Symbolen	27
Accessories / Tillbehör / Accessoires	28

1. INTENDED USE, USER AND ENVIRONMENT

The (Self-propelled) Shower and Toilet seating Aids are mobile chairs to be used for taking care of persons with little or reduced muscular function. Taking care includes for example actions of assistance with bathing, washing, showering and / or toilet activities. In the specific case of the Swimming Pool Hoist Chair, use in swimming pools and / or water related physical therapy sessions is part of the Intended use.

The devices are mobile to allow transport to and from the bathroom area and transfers and movement within the bathroom area. The devices allow the patient physical support during showering, washing, drying and dressing in a safe and comfortable sitting position. The armrests will give support for sitting up, and swivel away for easy sideways transfers.

All devices equipped with keyhole shaped seat surfaces allow the use in combination with a regular toilet.

All bathroom safety devices have a maximum safe working load as indicated on the labels and in the IFU. All devices can be tailored to the patient's and caregivers needs with the use of accessories.

The Bathroom Safety Products can be operated and propelled by the patient user or are to be operated by caregivers. Both should have access to and the ability to read and understand the IFU of the products. Apart from the advantages for the patient, the Bathroom Safety products reduce the need for manual lifting or handling of the patient, thus making the situation safer and reducing caregiver work-related injuries. It allows the caregiver to assist the patient elevated from the floor, which improves communication and eliminates the need for the caregiver to work very close to the floor. To new users, users with reading or cognitive disabilities, training or instruction on the proper use of the device is recommended. There are no manuals available for users with visual or cognitive disabilities.

Intended User

In general, the devices are to be used by:

- Conscious patients or disabled persons with reduced or little muscular function who are not subjected to the care of a caregiver. The patient is capable of performing part of the daily hygiene activities without assistance, but limited in endurance.
- Conscious patients or disabled persons with reduced or little muscular function who subjected to the care of a caregiver but may be able to contribute to the action or perform part of the action independently.
- Patient: Person undergoing a medical procedure. Person recovering from a medical procedure. Person suffering from physical impairments due to age related ailments such as with weak legs, low stamina, tremors and/or poor balance.
- Disabled person: Person with one or more impairments, one or more activity limitation, one or more participation restriction or a combination.
- Caregivers with medical training: healthcare professionals with medical education of different levels, who operate the device, assisting a patient. Have knowledge of intended medical indication and how to approach a patient. Are literate and can read and interpret instructions for use.
- Caregivers without medical training: laypersons with no medical education, who operate the device, assisting a patient. Are literate and can read and interpret instructions for use, have received training from DHG or DHG certified trainers on how to operate the device.
- Service personnel: technically savvy personnel at professional healthcare facility or assisting caregivers in home healthcare environment with repairs of the device. Service personnel have received training from DHG or DHG certified trainers on how to repair the device.
- DHG Service Technicians: technically savvy DHG personnel with expertise in repairing and servicing the device.

Intended Environment

Linido products and accessories are intended to be use in Home Healthcare environment and Professional Healthcare facility environment.

- Home Healthcare environment is limited to indoor domestic areas (where the products are used to alleviate or compensate for an injury, disability, or disease) and public use (such as public restrooms, swimming pools, hotels)
- Professional Healthcare facility environment includes long-term care (nursing homes/residential care, rehabilitation and geriatric facilities), clinics, limited care facilities and hospitals (patient rooms, emergency rooms, intensive care, surgery rooms except near HR surgical equipment, outside the RF shielded room)
- The intended use environment does not include extreme climate indoor areas, such as sauna's and spa's and outdoor areas such as beaches or lakes.

2. CONTRAINDICATIONS, PRECAUTIONS AND WARNINGS

We hope that our product meets or exceeds your expectations. This instruction manual contains the information that users of the product need to know for safe operation. In addition to the instructions for use, there is a technical manual that describes the structure of the device and contains information of a more technical nature and a list of spare parts.

PLEASE READ THIS INSTRUCTION MANUAL BEFORE USING THE PRODUCT.

Failure to comply with the following points of attention and the further descriptions in this manual can lead to dangerous situations. Improper and careless use can cause damage to the device, and thus also lead to dangerous situations. In such situations, decommission the chair and contact the supplier.

Contraindications

- The device may not be used for patients above the maximum weight indicated on the label of the device and this user manual.
- It is not allowed to lift anything else besides the intended patient population.
- The device is only to be used according to the intended use. Any other purpose or use is not permitted.
- The device shall not be used on inclined surfaces with an angle of more than 5 degrees.

Precautions

- Treat the device with care and ensure a clean, tidy working environment.
- Check the condition and functions of the device regularly.
- Service and maintenance of the device is only permitted by demonstrably authorized persons. The lifetime of the device can be decreased by improper maintenance.
- The replacement structural parts of the device is only permitted by demonstrably authorized persons.
- Operating the device is only permitted by demonstrably authorized persons.
- When moving the device, always check that the space where the transfer takes place is free of obstacles.
The device must not jam or remain stuck.
- The device should be inspected periodically at least once a year.
- Always report defects on the device to the supplier.
- When lifting, always check that there is enough space above the device.
- Keep the manual where it is accessible to users of the device.
- Fold the armrests down when a patient is on the chair or during lowering / rising (even without patients present).
- For optimum Safety: tighten expander bolts on all wheels before use and again after 4 weeks. Check regularly thereafter.

Warnings

- Never leave the patient unattended during a transfer situation.
- Only use the device with approved accessories.
- For service and repair, use only original parts supplied by Direct Healthcare Group.

- Do not use the device if defective. If there is a red sticker with 'rejection' on the device, the device may not be used either.
- If deformations have occurred on the chair due to incorrect use or transport, it may no longer be used and must be returned to the supplier.
- Do not use any of the accessories when its condition is insufficient.
- The device must not be left or stored in a damp or humid environment.
- The device must not be cleaned using steam.
- The device may only be used indoors.
- The tilting and lifting functions are solely intended for lifting and lowering. It should never be used as a support or handle.
- Always keep the hands, legs and feet clear from the moving and folding parts.
- Resistance of ignition of the upholstered parts complies to BS-EN 1021-1 and -2. Please keep the device away for open fire, flames or direct sources of high heat.

3. BEFORE EVERY USE

1. Visually inspect the product if it is free from damage and properly assembled. The occurrence of the following must be checked.
 - Cracks in the frame
 - Cracks in the plastic parts
 - A bended frame
 - Bended legs
 - Damage to the castors or brakes
 - Tears or irregular holes in the HT cover material.

If any of these points apply, the product should be discarded, or parts must be replaced with spare parts. Dispose the discarded products or parts as environmentally as possible according to the local regulations.


2. Check the product functions by lifting, lowering or tilting the device.
3. Check movability in all directions. It should take no more than approx. 10 kg (100 N) of force to move the device. If it feels there is more force required, check whether there is anything blocking the device or if the brakes are still on.

The device is ready for use. Always maintain a neat and clean work environment.

Operation Control Forces

Operation Control	Maximum required Force
Hand control buttons	0,2 kg / 2 N
Foot Pedal	30 kg / 300 N
Castor brakes pedal	30 kg / 300 N
Moving/handling the device	10 kg / 100 N

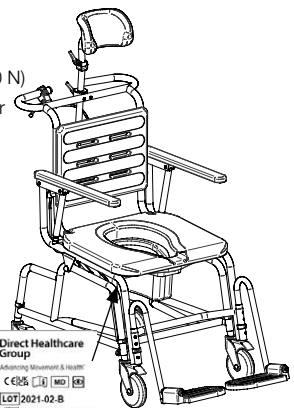Product Label

4. MAINTENANCE

The chair is designed to keep maintenance to a minimum. It is advised to follow the maintenance schedule below:

- Daily and weekly maintenance can be done by caregivers or service staff of an authorised reseller.
- Semi-annual maintenance should be performed by the technical service staff of an authorised reseller
- Annual inspection and any repair thereof should be performed by the technical service department trained to service and repair this equipment.

The expected lifetime of the product is 5 years.

4.1 Daily Maintenance

The following operations can be performed by the caregivers or service staff on a daily basis.

- Rinse or clean the parts after each use.
- It is recommended to dry the cover after each use for hygiene purposes.
- The device may be cleaned regularly using normal household detergent. If disinfection is required, use a product intended for cleaning of such products. Use a detergent with a maximum of 45% alcohol or max 5% Salicylic Acid.
- The product is not intended to be sterilised or cleaned by steam.
- Rinse with clean (mains) water after using detergents or cleaning agents and after use in or around swimming pools (if applicable).

 *When in doubt on the condition of the device, always contact the technical service of your authorised distributor!*

4.2 Weekly Maintenance

The following operations can be performed on a weekly basis.

- It is recommended to clean the chair weekly. This is best done with household detergent and a well-squeezed damp cloth.
- Check that all movable parts move easily and quietly.
- Check the readability of the serial number label, and the availability of a manual.

4.3 Semi-Annual Maintenance

The list below provides an overview of the maintenance that the technical service can perform on the chair every six months.

1. The movement of all moving parts of the chair should be smooth.
2. No damage to plastic parts is allowed.
3. The castors should turn easily and that the brake should work properly.
4. All bolts must be properly tightened and not loose.

 *Service and maintenance of the chair is only permitted by demonstrably authorized persons.*

 *If the chair does not comply with one or more of the above points, the lift must be temporarily put out of operation and an authorised distributor must be contacted.*

5. TROUBLESHOOTING

5.1 Troubleshooting table


Complaint	Possible Cause	Action
Brake does not work	Wheels broken.	Contact supplier or technical service.
Lifting function does not work.	Hydraulic pump broken.	Contact supplier or technical service.
Tilting function does not work.	Gas spring broken.	Contact supplier or technical service.
Armrest / side fence jams	Armrest mechanism broken	Contact supplier.

 *If the repair table does not resolve the complaint or in case of doubt, do not use the device and contact the supplier.*

5.2 Replacement/Removal of Parts

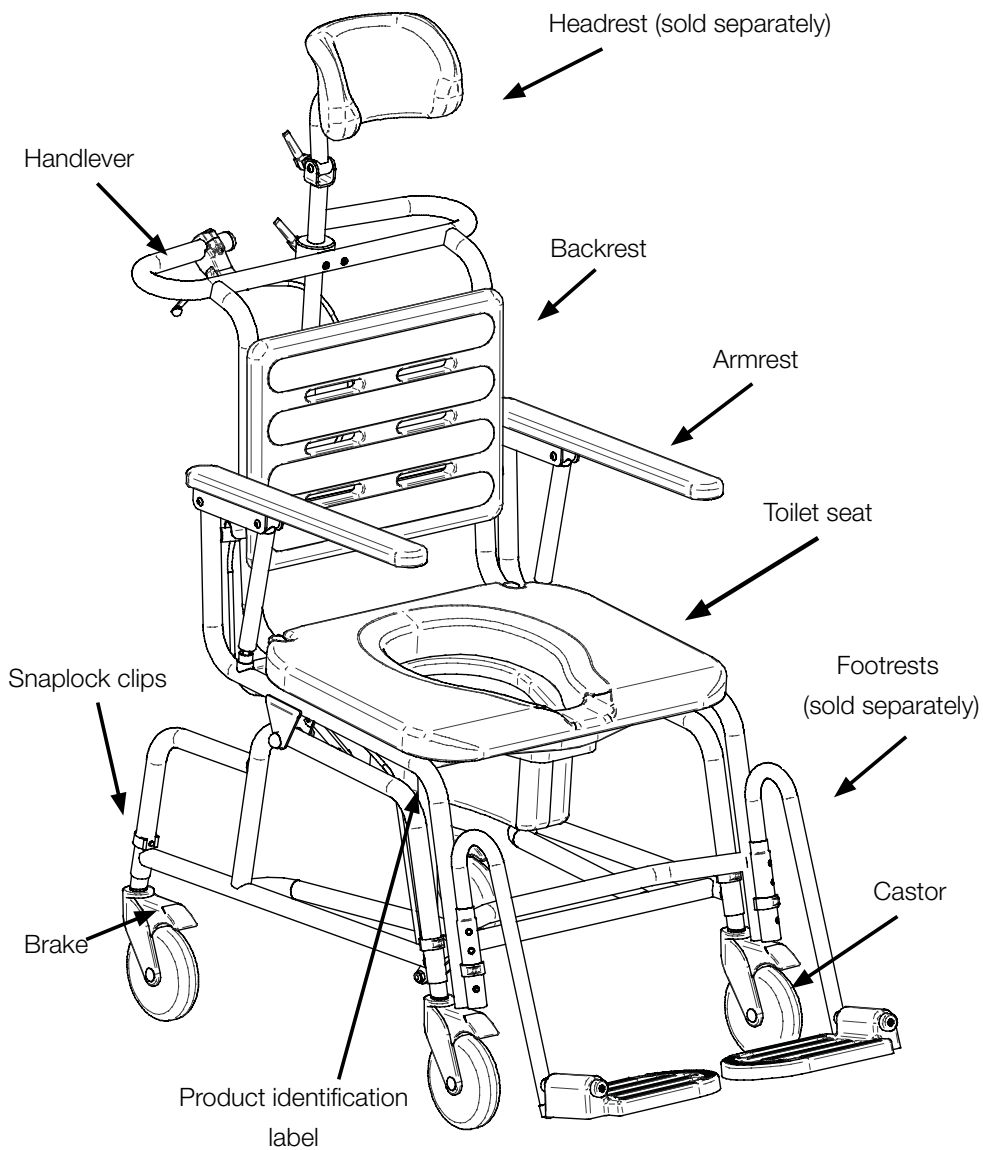All parts that are replaced or removed (including batteries) and thus need to be discarded can be returned to DHG / Linido. It is also possible to offer these parts to the mechanic for destruction. We will ensure that all parts are processed in an environmentally friendly manner.

6. ASSEMBLY

The Assembly, operation and adjustments should always be carried out by a user or caregiver who has read and understood the user manual.

Frame Assembly - see figure 1:

Remove the snaplocks and mount the backrest into the holes in the frame. Replace the snaplocks, ensuring they are inserted correctly into the steel tube with both pins correctly placed in the holes. It is possible, although not necessary, to detach the backrest when the Combi chair is used over a toilet.

Figure 1

Assembling the castors - see figure 2:

FRONT castors: Insert the inner-tube (castor) into the chair leg, aligning the larger hole of the castor tube with the hole in the leg tube. Insert the snaplock at the appropriate height with the black part facing to the front.

REAR castors: Insert the inner-tube (castor) into the chair leg, aligning the larger hole of the castor tube with the hole in the leg tube. Insert the snaplock at the appropriate height with the black part facing to the back.

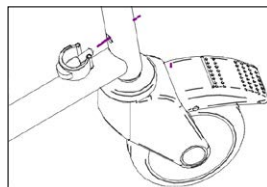
Figure 2

To ensure proper alignment of the castors, the snaplock pin will not fit into the tubes if the holes are mis-aligned.

IMPORTANT: Check that all four legs are the same height and that all of the snaplocks are inserted correctly into the frame before use.

Bedpan / Bucket rails

The bed pan / bucket rails can be used in combination with a keyhole (fig.3) or round shaped bucket (fig.4).

Figure 3

Figure 4

Figure 5

To assemble - see figure 6:

The two end pieces (B, on the drawing) rest on the rear of the seat frame. For keyhole shape bucket, insert the two front rods (A) into the inner holes (C). For all other bucket shapes use the outer holes (D). If no bucket or bed pan is used, the rails may be detached so the chair fits easily over a toilet.

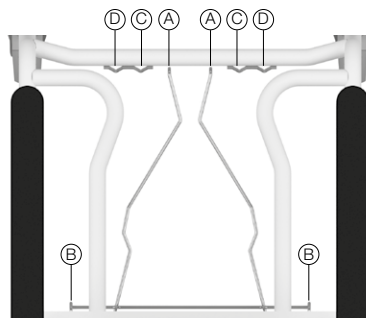
Figure 6

Individual Adjustment - see figure 7:

Attachment of castors & frame height adjustment:

The pin on the snaplock clip narrows in width from the front towards the end. The hole in the tubing is bigger on one side and narrower on the other side. This safety feature means that the snaplock clip can only be inserted in one way into the tubing, ensuring the castors to be mounted correctly.

FRONT castor: insert snaplock clip into tube hole from rear of tubing so pin points frontwards.

REAR castor: insert snaplock clip into tube hole from front of tubing so pin points backwards.

Height Adjustment:

By re-positioning the snaplocks on the chair legs, the Combi chair may be adjusted to 6 different heights.

To adjust the height of the chair:

1. Remove snaplock.
2. Select one of the 6 positions.
3. Reinsert snaplock.

Repeat for all 4 legs.

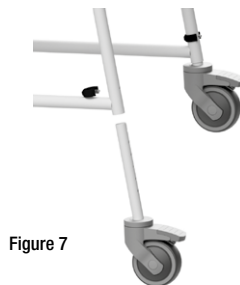
Figure 7

⚠ IMPORTANT: Check that all four legs are the same height and that all of the snaplocks are inserted correctly into the frame before use.

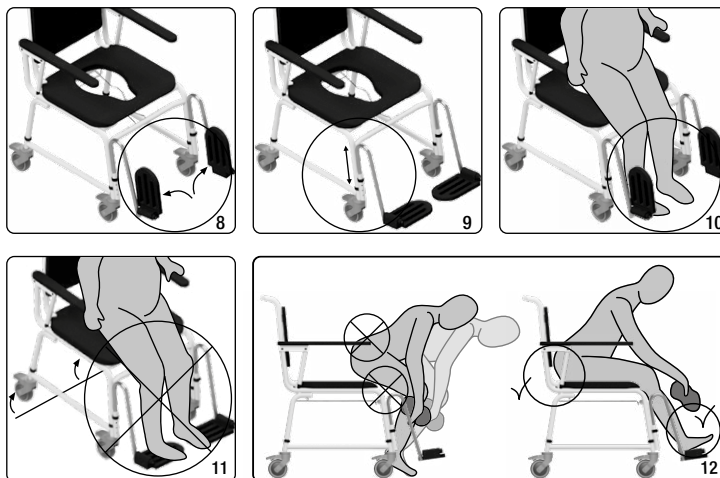
7. HOW TO USE

Tilting function

The Combi manual tilt chair can be tilted 5° forward and 25° backwards.

⚠ IMPORTANT: Before using the tilting function remember to lock all four castors.

The tilting function is operated by means of a gas piston activated via a hand lever on the pushbar. While the backwards tilt may be operated with the chair empty, the forward tilt requires a counterweight to facilitate the movement for the caregiver. Stand behind the chair and operate the hand lever. A small amount of downward force may be required to tilt the chair back depending on the weight of the user.

8. TECHNICAL SPECIFICATIONS

Total height:	37-46" (950-1180 mm)
Total length excl. footrests:	34.25" (870 mm)
Total length incl. footrests:	43" (1090 mm)
Total width:	21,2" (535 mm)
Backrest:	16 x 12" (410 x 310 mm)
Seat height interval:	20.2"-23.6" (520-620 mm), adjustable
Seat width:	20" (510 mm)
Seat depth:	17.1" (435 mm)
Seat hole width:	8.2" (210 mm)
Seat hole depth:	12.2" (310 mm)
Tilting angle:	25° backwards, 5° forwards

Armrest, length:	19" (480 mm)
Armrest, width:	2" (50 mm)
Width between armrests:	17" (430 mm)
Castors:	Ø 5" (125 mm), incl. brakes
Max. user weight:	330 lbs (150 kg)
Total weight:	24 kg / 53 lbs


Expected lifetime: 5 years

Materials

Frame:	Stainless steel
Surface coating frame:	Powder coating
Seat, Backrest, Armrests:	Polyurethane (PUR)
Snaplock Clips:	Chrome pin/Polypropylene

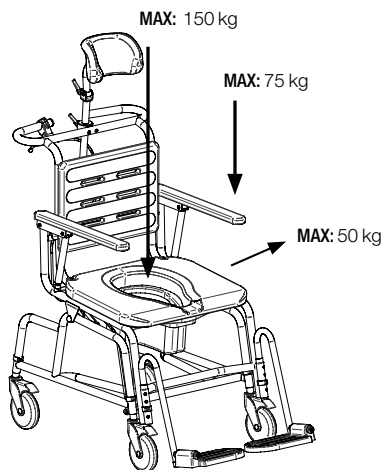(obstacle free) turning
circle diameter: 1500mm

IMPORTANT: This product can not be folded/dismantled for storage or transport.

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201941314 Combi Manual tilt

To always be ordered with:

200810004	Footrests	1 pair. For all Combi chair models and versions. Recommended for use with all models and versions of Combi chairs.
200360004	Headrest	For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.
200384004	Heel and calf strap	For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.
200382004	Hip belt	For all Combi chair models and versions.

Additional Accessories:

201131314	Soft backrest	For Combi tilt, manual and Combi tilt, powered.
201040314	Front cross bar	For all Combi chair models and versions with standard width. Recommended for use with the Armrest lock accessory.
201030310	Armrest lock	1 pair. For all Combi chair models and versions. Recommended for use with both versions of Combi tilt.
201070314	Lateral trunk support, right	For all Combi chair models and versions.
201072314	Lateral trunk support, left	For all Combi chair models and versions.
201060004	Amputee support	For all Combi chair models and versions.
201061004	Leg support	For Amputee support and all Combi chair models and versions.
853100604	Transport seat cover	For all Combi chair models and versions.
853100034	Bucket with lid	Keyhole shape, white plastic. For all Combi chair models and versions.
853100033	Bucket with lid	Round, grey plastic. For all Combi chair models and versions.

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Beensteun
201061004

Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

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Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

DHG

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com