Coral commode/shower stools

Manual - English

Manual - Svenska

Manual - Norsk

Manual - Danske

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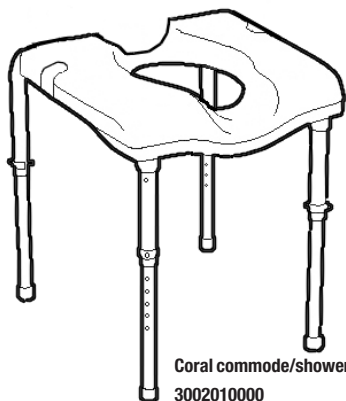
Manual - Español

Coral commode/shower stool
3002010000

Coral shower stool
3002010100

Art.no.:

3002010000, 3002010100

SWL: 150 kg/330 lbs

Table of contents

Visual inspection/before use	3
Read the manual	4
<hr/>	
English	5-10
Svenska	11-16
Norsk	17-22
Danske	23-28
Soumi	29-34
Deutsch	35-40
Nederlands	41-46
Français	47-52
Italiano	53-58
Español	59-64
<hr/>	
Dimensions	66-67
Accessories	68

**EN Visual inspection**

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
Table of contents

Using the product	5-9
- Important information	6
- Assembly	7
- Disassembly	8-9
With Coral, you are the designer!	10

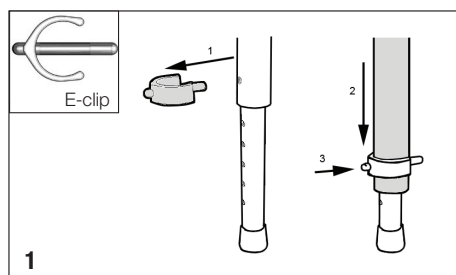Using the product

1. Adjust the height of the stool by repositioning the E-clips (without a user in the stool).

1. Remove the E-clips
2. Lower or lift the leg to desired height
3. Snap the E-clips in place
4. Ensure that all legs have the same height

Important information

Purpose

Coral commode and shower stool is a very versatile stool. It can be used as both a commode and a shower stool. Coral commode and shower stool is mostly used in the bathroom and can be equipped with arm supports and back support. Coral commode and shower stool is particularly suitable for immobile persons, who need assistance with toileting and personal hygiene. The stool is intended for short-term use only.

Maintenance / Cleaning

To ensure optimal operation, the following should be regularly reviewed:

- Check if the E-clips (see drawing, page 5) are securely fastened

The stool is to be wiped clean after use. Clean Coral commode and shower stool with non-abrasive, non-corrosive domestic cleaning agents. Solvents and chlorine should not be applied. The stool's surfaces can be disinfected with all disinfectants that are approved by Health Authorities.

The toilet rails can be detached for cleaning purposes.

Make sure to reattach all removed parts correctly.

Reuse

Follow the above procedure. Defective or worn parts must be replaced.

Caution

Max. load Coral commode and shower stool: 150 kg / 330 lbs.

Check the condition of the E-clips frequently.

Regularly check that the E-clips are inserted correctly into the tubing.

Regularly check that all parts are in place and secured.

If the height adjustment tubes are detached, it is important to mount them again correctly.

Make sure all 4 height adjustment legs have the same height.

The stool is only to be used on an even surface. Do not stand on the stool.

The bed pan/bucket is only to be inserted or removed, when the user is not in the stool.

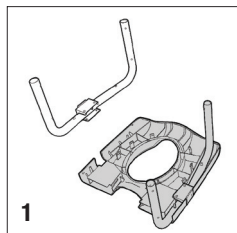
Do not place the stool in direct sunlight, as it might cause a hot surface on the seat.

Disposal


The plastic parts should be sorted as incinerated and stainless steel parts as metal.

Assembly


Coral commode/shower stools
3002010000, 3002010100

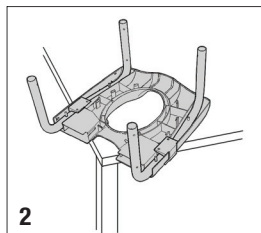
1
 1. Position the leg with cover (2x) on the seat. Make sure that all pins of the seat fully protrude the leg and the cover.

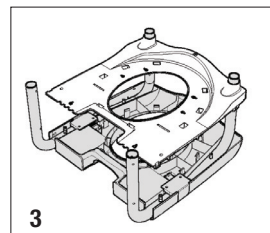
2
 2. Make sure that the leg and cover (2x) are well fitted in the seat. Lay this part on a table for easier assembly of the other parts.

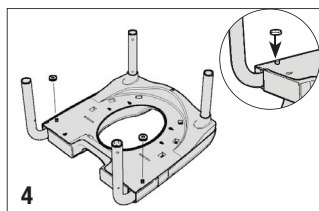
3
 3. Slide the base plate over the front legs. Make sure that the rear pins of the seat fully protrude the base plate. Push the base plate onto the seat until all clickers snap into place.

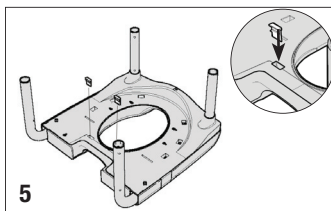
4
 4. Mount the safety lock (2x) on the protruding pin. Make sure that both safety locks are screwed on tightly.

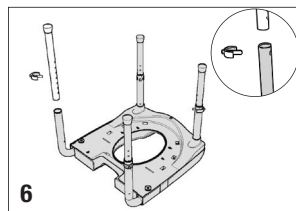
5
 5. Mount the safety key (2x) between the clickers at the back of the stool. Push downwards until it snaps in place.

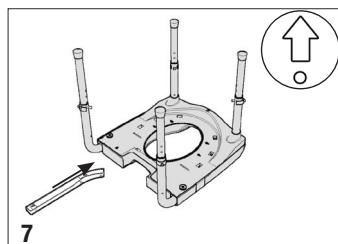
6
 6. Slide the height adjustment tube into the leg (4x). Adjust to the correct height and click the E-clip through the chair leg and height adjustment tube. Make sure the legs are mounted at the same height.

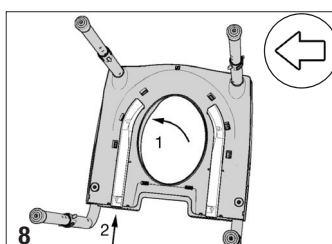7
Applicable for commode seat: 3002010000
 7. Place the rear pin of the toilet rail (2x) into the rear keyhole in the base plate. Push forward until it snaps in place.

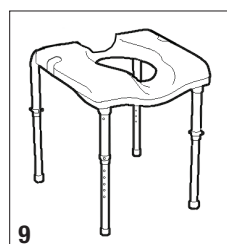8
Applicable for commode seat: 3002010000
 8. Rotate the toilet rail until the front pins align with the front keyholes. Push outward until it snaps in place.


9
 9. Turn the stool upright. Make sure that all parts are in place and secured.


Disassembly

Coral commode/shower stool

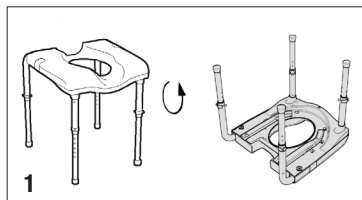3002010000, 3002010100

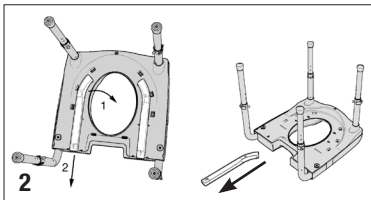
1. Turn the stool upside down and lay the stool with the seat on top of a table.


Applicable for commode seat: 3002010000

2. Remove the toilet rail (2x) by:

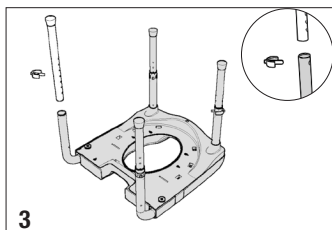
1. Pulling the rail inward until the upper part unlocks.
2. Pushing the rail backward (direction: back of the chair) until the rear pin unlocks.

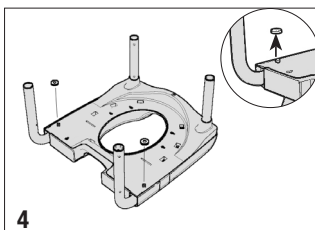
3. Remove the clip from the leg and remove the tube (4x).

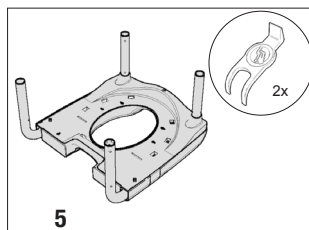
4. Remove the safety lock from the pin by unscrewing (2x).

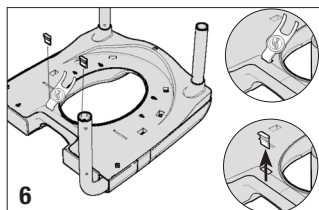
5. Use the two tools for disassembly, to unlock the base plate (see pictures 6 to 10).
Only use the included tools.

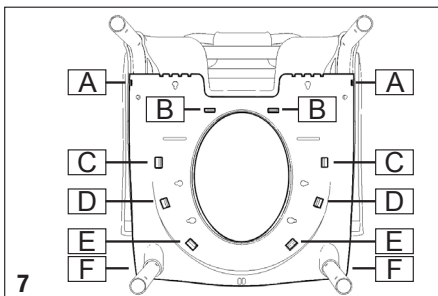6. Remove the safety keys at the backside of the base plate. Place the wedge end in the open space of the pocket. Remove the safety keys by rotating the wedge upwards (2x).

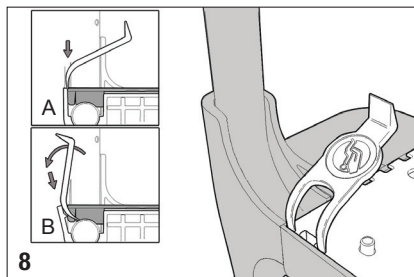
7. Unlock the clickers in alphabetical order (see pictures 7 to 10).


8. Pos A (2x):

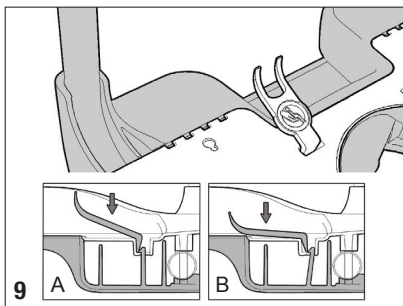- Fig 8A: Position the fork of the tool around the pocket. Make sure the tip of the fork is between the seat and the base plate.
- Fig 8B: Gently push/slide the tool downward between both parts until the clicker unlocks.


9. Pos B-E (2x);

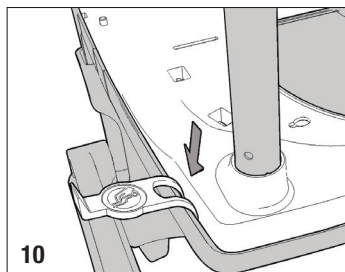- Fig 9A: Place the wedge end in the pocket by sliding along the back of the pocket. Tip of the wedge end touches the tip of the clicker. Tool and base plate are angled.
- Fig 9B: Press down the tool toward the base plate, while lifting the base plate, until the clicker unlocks.


10. Pos F (2x):

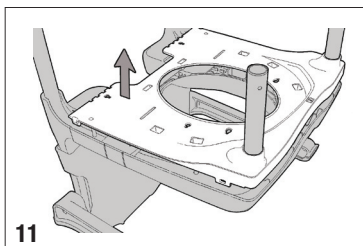- Position the tool with the tip of the fork between the seat and the base plate (at approximately 5cm/2" from the front corner of the seat).
- Press the tool gently between the seat and the base plate to unlock the clicker.

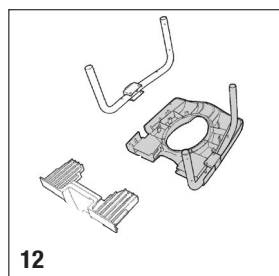11. Pull the base plate upward over the legs. Keep the base plate horizontal while pulling.

12. Remove the cover for back of seat and the legs off the seat.

With Coral, you are the designer!

With one stable and height-adjustable frame combined with optional extras and accessories, Handicare's Coral concept offers numerous possibilities to create individually adapted showering and toileting solutions. All combinations offer safe and flexible user-friendliness, high-quality stainless materials and a contemporary design.

When life changes, so does your Coral

For increased comfort and usability, Coral commode/shower stools can be equipped with back support and/or arm supports. The stool with a toilet seat can be equipped with a bedpan or bucket and soft toilet seats. The back and arm supports can also be replaced with a backrest with folding armrests. For even greater comfort, or to accommodate for changing needs and requirements, the chair can be fitted with castors and footrests. Finally, with the 24" self-propelling wheels adapter set, a Coral commode or shower chair fitted with the backrest with folding armrests can be conveniently transformed into a self-propelled version providing the user with increased independence in showering and toileting situations.

Recycling – an important part of the Coral concept

Naturally, an existing Coral configuration can also be adapted to a "smaller" solution or to fit another user. All dismantled parts and accessories can be reconditioned and re-used in new configurations for other users.

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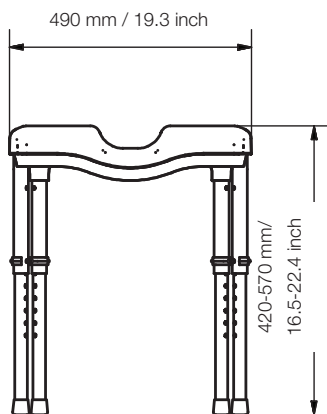
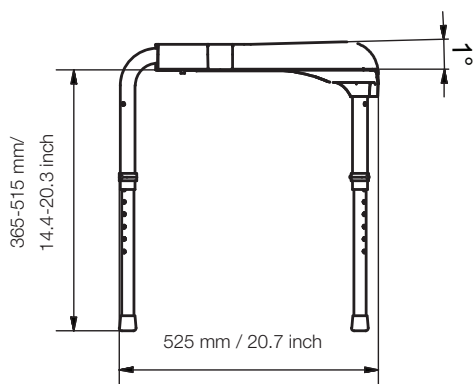
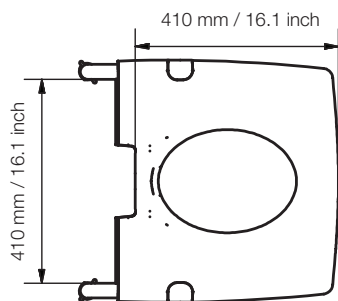
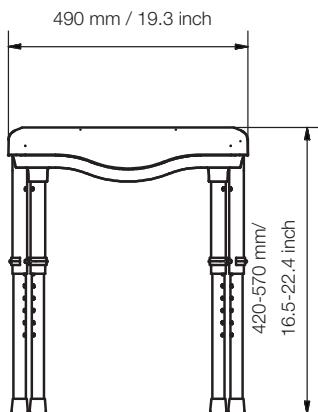
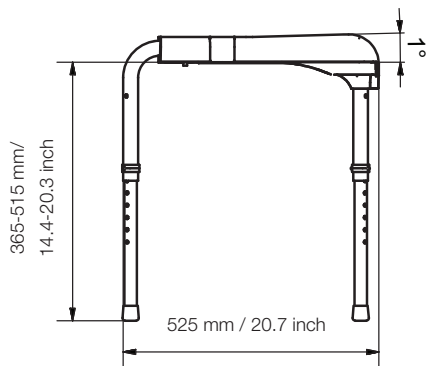
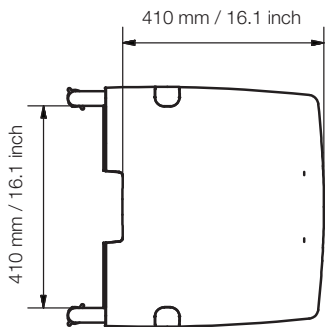
Accessories for increased comfort and/or usability

3002920802	Bedpan, with lid, stainless steel
3002920803	Bucket, with lid, stainless steel
3002920400	Soft toilet seat, PUR, oval opening
3002920401	Soft toilet seat, PUR, keyhole opening
3002920500	Transportation seat, PUR
3002800200	Shower seat, plastic, incl covers
3002800100	Toilet seat, plastic, incl covers and toilet rails

Accessories for creating more advanced configurations

3002800300	Back support, plastic
3002800500	Arm supports, plastic, 1 pair
3002920402	Soft toilet seat, PUR, oval opening, with cut-outs for arm supports
3002920403	Soft toilet seat, PUR, keyhole opening, with cut-outs for arm supports
3002920501	Transportation seat, PUR, with cut-outs for arm supports
3002820200	Backrest frame with folding armrests, plastic frame, incl 2 hinge pins
3002800400	Backrest cover standard, textile, polyester mesh
3002921100	Armrest lock, prevents folding function, 1 pair
3002920900	Front support, for Backrest frame with folding armrests 3002820200
3002820800*	Castors, twin castors, Ø 100 mm, incl legs, 4 pcs
3002820900*	Castors, twin castors, Ø 125 mm, incl legs, 4 pcs
3002800600	Swing-away footrests, plastic, incl fixing clips, 1 set
3002810300	Folding footrests, plastic, with tubes, incl fixings, 1 set
3002921400	Headrest, for Backrest frame with folding armrests 3002820200
3002920200	Calf support, textile, with Velcro
3002920100	Amputee support left, for left underleg, with adjustable knee angle (a piece), incl fixings
3002920101	Amputee support right, for right underleg, with adjustable knee angle (a piece), incl fixings
3002921300	Safety belt, for hip and chest
3002921500*	24" self-propelling wheels adapter set, complete adapter set
3002921700	Anti tip set, for self-propelling chairs

*Castors and self-propelling wheels can only be added to a Coral stool/chair fitted with a Backrest frame with folding armrests (3002820200).

For more information, please visit www.handicare.com or contact your local distributor.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve convenience for those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, Poland, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various power and manual wheelchairs, scooters, seating systems, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise- and recline chairs, and bathing and toileting products.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com

Handicare AB
Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se

Internet: www.handicare.com