# Protective Cover Disposable Cover

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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Protective Cover

Disposable Cover

**REF**

1051 1055

1151 1155

1251 1255

1451 2055

1481 9138

2051 9168

2052

**SWL for protective cover for EasySlide: 300 kg/660 lbs**

**SWL for protective cover for RollerSlide: See board**

## Table of contents

<b>English</b> .....	<b>3</b>
<b>Svenska</b> .....	<b>4</b>
<b>Norsk</b> .....	<b>5</b>
<b>Dansk</b> .....	<b>6</b>
<b>Suomi</b> .....	<b>7</b>
<b>Deutsch</b> .....	<b>8</b>
<b>Nederlands</b> .....	<b>9</b>
<b>Français</b> .....	<b>10</b>
<b>Italiano</b> .....	<b>11</b>
<b>Español</b> .....	<b>12</b>
<b>Product information</b> .....	<b>13</b>

# Protective Cover Disposable Cover

SystemRoMedic™

Different protective covers are available for SystemRoMedic's EasySlide and RollerSlide. Protective Cover is a multi-use cover that is made of Ultra Low Friction (ULF) material. Disposable Cover, a single-use cover that is made of plastic, can be used to advantage in e.g., emergency care or on isolation wards. The multi-use protective cover for RollerSlide is also a glide cover, since it enables the slide to glide freely.

## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

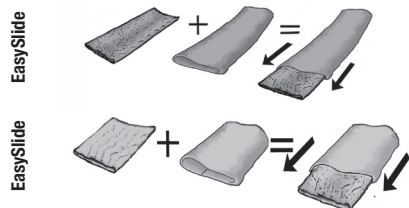
## Always read the user manual

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product. Always make sure that you have the right version of the manual.


The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

## Placement

Disposable Covers are on convenient rolls and are torn off one by one.

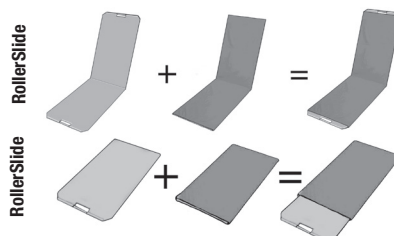Disposable Cover and Protective Cover are pulled over EasySlide or RollerSlide (see images).

When using Disposable Cover on RollerSlide, remember that a glide cover is always needed, since it is what allows the slide to glide freely.

**NOTE!** DisposableCover cannot be laundered, but must be discarded after use.

## Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC. Contact your local distributor if you have any questions about the product and its use. See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

## Expected lifetime

Up to 10 years of normal use

# Protective Cover Disposable Cover

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## Simple solutions for great results

SystemRoMedic™ is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

# Direct Healthcare Group

Advancing Movement & Health

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)