EasyGlide

User manual - English

Bruksanvisning - Svenska

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REF
5000

SWL: 150 - 250 kg/330 - 550 lbs

Table of contents

English	4
Svenska	6
Norsk	8
Dansk	10
Soumi	12
Deutsch	14
Nederlands	16
Français	18
Italiano	20
Español	22

EasyGlide, extra long is a sliding board which is used for supine transfers of users from one bed or stretcher to another. EasyGlide, extra long is used when the gap is max. 15 cm/5,9". For safety reasons, EasyGlide, extra long should have a firm underlying surface on both sides when transferring.

The rounded corners and the smooth, low-friction top surface provide for easy sliding transfers and make the sliding board very easy to position and remove. The double anti-slip tape on the underside prevents the board from sliding against the underlying surface during transfer.

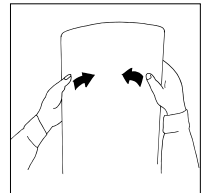
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that material is not worn or faded. If there are signs of wear, the product must be discarded.

Stability test

Carefully bend the assistive device to check strength and durability. The board should be both elastic and rigid. Discard the EasyGlide when its strength and durability are diminished.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

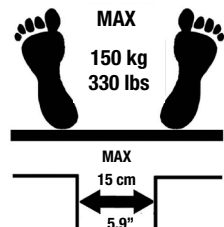
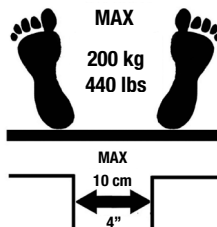
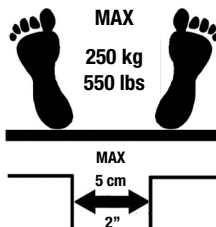
The most recent editions of user manuals are available for downloading from our website, www.handicare.com.


Important information

For the sake of safety, both underlying surfaces should be firm when transferring between bed and gurney.

Do not leave the user unattended during a transfer situation.

Transfer from bed to bed

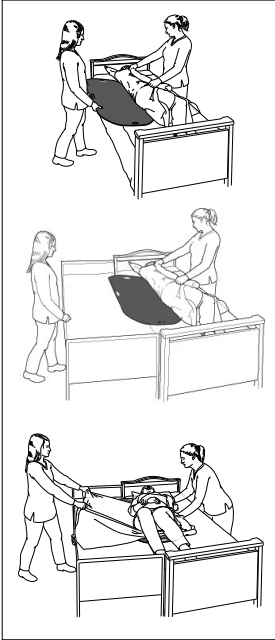EasyGlide is placed under the hips and shoulders.

The user is turned on one side. A draw sheet can be used to facilitate the transfer. Guide the EasyGlide in under the user, between the bed sheet and the draw sheet. Press EasyGlide down into the mattress somewhat and push in.

Ensure that the weight of the hips and shoulders is centred on EasyGlide.

Transfer on transfer slide. Place the transfer slide between the two underlying surfaces. By using a draw sheet, the caregiver's reach is extended, allowing a better working stance.

Make sure the transfer slide is between the two underlying surfaces and that the gap between the beds does not exceed 15 cm/5,9". Lean the user towards the transfer slide and pull the draw sheet to transfer the user.

To remove, lean the user to one side and pull EasyGlide out.

Material

Polyethen

Care of the product

The surface of the product can be cleaned with warm water or a soap solution. If disinfection is needed, 70 % ethanol, 45 % isopropanol or similar should be used.

CE Medical Device Class I

The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Vida útil estimada

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Making everyday life easier

More than 30 years in the service of simplicity

Handicare's unique easy transfer concept is the market's widest and most complete range of clever, easy-to-use and safe assistive devices for all types of patient transfers and manual handling requirements.

The philosophy behind the concept is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity.

Through a unique combination of training and a complete range of efficient transfer aids, we offer improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Designed and produced by Handicare

For more than 30 years, Handicare has offered simple solutions for great results, and an easier everyday life for both users and personnel in the care sector. During 1984-2010 under the name of RoMedic and, as of January 1, 2011 under the name of Handicare.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide.

Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

handicare

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com