EasySlide

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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

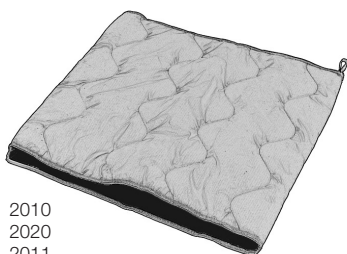
2010
2020
2011
2021

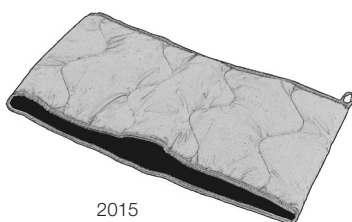2015

REF

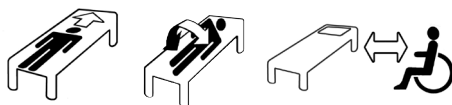2010
2020
2011
2021
2015

SWL: 300 kg/660 lbs

Table of contents

English	3-4
Svenska	5-6
Norsk	7-8
Danske	9-10
Suomi	11-12
Deutsch	13-14
Netherlands	15-16
Français	17-18
Italiano	19-20
Español	21-22
<hr/>	
Products/Accessories	23

EasySlide is used for turning in bed, for transfers between bed and wheelchair/toilet, and for repositioning the user higher up in bed. EasySlide is a tubular, padded mat with a friction-reducing material on the inside.

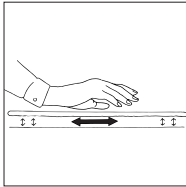
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

Placement of EasySlide

Place the EasySlide under one buttock or under one shoulder. For transfers, all of the user's body weight should be shifted over onto the EasySlide.

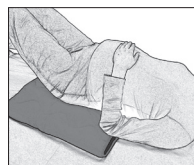
To remove, pull on the lower end of the EasySlide.

Using EasySlide

1. EasySlide can be used to advantage for transfers to/from a wheelchair. Complement, preferably, with an EasyGlide.

2. EasySlide can be used under the shoulders as a friction-reducing assistive device to facilitate repositioning higher up in bed.

3. EasySlide can be used under the hips to facilitate lateral transfer. Active users can use EasySlide themselves, but it can also be used by less active users with the aid of a caregiver.

The gap between the underlying surfaces (bed/gurney/show trolley) must not exceed 5 cm.

NOTE!
Always remember to lock wheels.

To prevent the risk of the user sliding/falling, never leave the EasySlide under the user after the completed transfer, and never position the user too close to the edge of the bed.

Material

Polyester, microfibre, polyamide

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Accessories

Art. no	Product	Description
2051	Protective cover	For EasySlide 2010 ,2020, 2030
2052	Protective cover	For EasySlide 2011, 2021
2055	Dispoable cover	For EasySlide 2010, 2020, 2030

2030	Bundle	EasySlide 2020 + Protective cover 2051
------	--------	--

Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

handicare

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com