Instructions for use – English

Bruksanvisning – Svenska

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
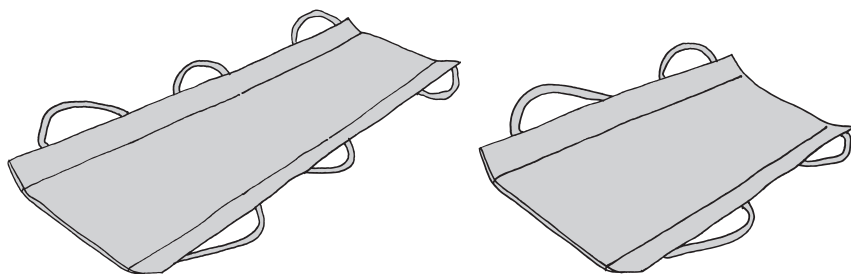
REF

4011

4012

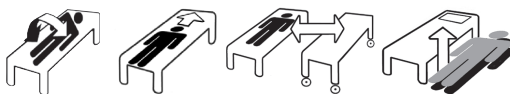4016

SWL: 150 kg/ 330 lbs

Table of contents

English	3
Svenska	5
Norsk	7
Dansk	9
Suomi	11
Deutsch	13
Nederlands	15
Français	17
Italiano	19
Español	21
Product information	23
Symbols	23

FlexiMove is used in different transfer and repositioning situations such as turning in bed, repositioning higher up in bed, bed to bed/bed to shower trolley, and lifting from the floor. FlexiMove features either four or six handles. The choice depends on the user's need for support and required number of caregivers.

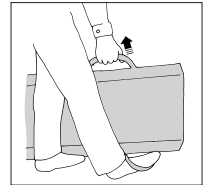
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Always read the user manual

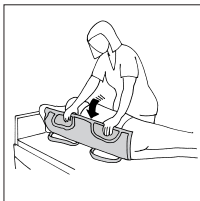
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.


The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Placement of FlexiMove


The user is turned on one side and the FlexiMove is guided in with the handles folded under the slide. The user is then turned on the back and the handles are drawn out.

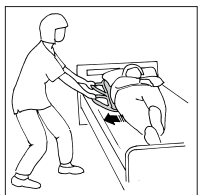Turning in bed

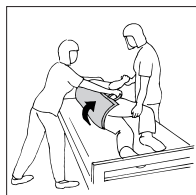
Use the handles to pull the user closer to the edge of the bed. Work in a walking stance with knees slightly bent, straight arms and straight back.


Fold in the handles, and then turn the user in the opposite direction, where one caregiver provides support. If you are working alone, raise the bed rails. Turn the user using the legs and pushing the user away from you.

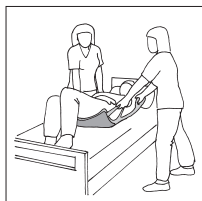Higher up in bed

One caregiver works on either side of the bed. Repositioning is easier if the user is able to bend his or her knees to push higher up in bed. Work in a walking stance with straight arms and straight back. Caregivers should coordinate their efforts to perform a smooth transfer.

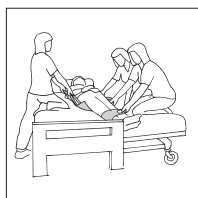Bed to bed/bed to shower trolley


Caregivers should work in pairs (or more), depending on the user's needs. Place beds as close together as possible. Pull the feet over first, and then help each other to manoeuvre the user over to the bed/shower trolley.

Lifting from the floor

NOTE!

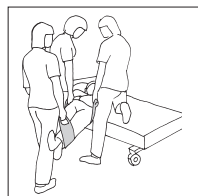A mechanical lift should always be used, if possible. When lifting manually, never lift more than is appropriate. Caregivers should work in pairs (or more), depending on the user's needs. Plan the transfer so as to minimize time taken, lifting and distance.

Material

Polyamid, nylon

Care of the product

Read the product label.

Cleaning: Remove the slide and padding and then clean surfaces with disinfectant. Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

**Direct Healthcare
Group**

Advancing Movement & Health®

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.