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## **EN - Please read the instructions before using the product.**

### ***These instructions apply to the headrest standard Combi chairs.***

#### **Purpose**

The Headrest can be used on all Combi chair models. The contoured padding lends support to the user's head and neck. The headrest is angle and height adjustable.

#### **Assembly**

Before the headrest can be assembled the black covers in the backrest need to be removed. Screw the cylindrical metal bracket onto the backrest pushbar using the 2 screws (3) and metal bushes (4) provided. Slot the tubing into the bracket and tighten the hand lever.

#### **Adjustment**

The Headrest can be height adjusted by loosening and tightening the lower hand lever (1). Angle adjustment can be achieved by loosening and tightening the upper hand lever (2).

#### **Maintenance**

Clean with regular cleaning agents without corrosive or abrasive properties. We advise against the use of chlorine and solvents.

#### **Caution!**

- Use only as described under "Purpose"
- Make sure to tighten the hand levers after adjustment.
- A headrest should always be used on a tilt chair.
- The end stop/clip at the headrest may not be removed. This end stop ensures the safety and the maximum height of the headrest.

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# Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## Direct Healthcare Group

Advancing Movement & Health<sup>®</sup>

Direct Healthcare Group B.V.  
Weteringweg 9,  
2641 KM Pijnacker, The Netherlands  
Tel: +31 (0)15 369 54 40  
Fax: +31 (0)15 369 71 64  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)  
[info.nl@directhealthcaregroup.com](mailto:info.nl@directhealthcaregroup.com)

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)  
[info.se@directhealthcaregroup.com](mailto:info.se@directhealthcaregroup.com)