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EN - Please read the instructions before using the product.

These instructions apply to the hip belt for Combi chairs.

Intended use

The hip belt fits all Combi chairs.

The hip belt is well suited for users who need trunk support, and can be adjusted to the individual user.

Installation

Remove the backrest by removing the clips on both sides (1). Lift up the backrest. Place the hip belt from the bottom up on to the backrest. Reinstall the backrest by placing it down into the holes in the frame. Put both clips back into the holes in the frame (1).

Adjustment and use

The hip belt can be adjusted to the individual user by loosening or tightening the strap on the front (2) or on the back of the hip belt (3).

Maintenance

The hip belt can be removed and machine washed at up to 30 degrees Celcius. It should not be dried in a tumbler dryer.

The hip belt can be cleaned with common cleaning solutions without abrasive or etching properties. It is not recommended to use solvents or chlorine.

Warning!

- Maximum weight of user: 100 kg.
- The hip belt must only be used as described under the section "Intended use".

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com