Hip belt

Accessory for Combi chairs

Linido

User manual - English
Bruksanvisning - Svenska
Brugsanvisning - Dansk
Brukerhåndbok - Norsk
Käyttöohje - Suomi
Handleiding - Nederlands
Benutzerhandbuch - Deutsch

REF 200382004

MAX 100 kg/220 lbs

Table of contents

English	5
Svenska	6
Dansk	7
Norsk	8
Suomi	9
Nederlands	10
Deutsch	11

**EN VISUAL INSPECTION**

Inspect the product regularly. Check to ensure that material is free from damage.

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EN BEFORE USE

Ensure that the product is correctly mounted/ installed.

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**EN ALWAYS READ THE USER MANUAL**

Always read the user manual. Keep the user manual where it is easily accessible for users of the product.

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Linido

Instructions for use
and user

EN - Please read the instructions before using the product. *These instructions apply to the hip belt for Combi chairs. Always consult the manual of the main product for additional product descriptions, intended use specifications, general warnings, precautions and contra-indications and maintenance and service information.*

Intended use

The hip belt fits all Combi chairs.
The hip belt is well suited for users who need trunk support, and can be adjusted to the individual user.

Installation

Remove the backrest by removing the clips on both sides (1). Lift up the backrest. Place the hip belt from the bottom up on to the backrest. Reinstall the backrest by placing it down into the holes in the frame. Put both clips back into the holes in the frame (1).

Adjustment and use

The hip belt can be adjusted to the individual user by loosening or tightening the strap on the front (2) or on the back of the hip belt (3).

Maintenance

The hip belt can be removed and machine washed at up to 30 degrees Celsius. It should not be dried in a tumbler dryer. The hip belt can be cleaned with common cleaning solutions without abrasive or etching properties. It is not recommended to use solvents or chlorine.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

DHG

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.