MoveMaster

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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REF
2090

Table of contents

English	3
Svenska	5
Norsk	7
Dansk	9
Suomi	11
Deutsch	13
Nederlands	15
Français	17
Italiano	19
Español	21
Product information	23
Symbols	23

MoveMaster is used for transfers between bed and wheelchair/portable toilet. MoveMaster is an open slide mat with three handles, one of which is longer than the other two. The different handles and the open design enable alternative working stances for the caregiver.

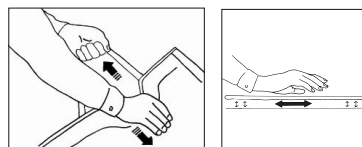
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

Always read the user manual

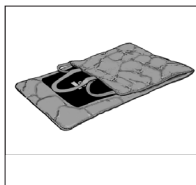
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Using MoveMaster

MoveMaster is a friction-reducing assistive device that can be used to advantage together with an EasyGlide to facilitate transfers in a seated position. MoveMaster is folded double and slides against itself to move the user over the underlying surface.

Place MoveMaster under the user using either one hand or with the help of an EasyGlide. EasyGlide can function as a bridge between the bed/wheelchair/shower chair. Lean the user slightly in the opposite direction when positioning the MoveMaster. During transfer, the user is leaned in the direction of transfer to reduce the contact surface under the 'sit bones' (the lowest bony prominences or ischial tuberosities), thereby sliding more easily.

To remove the MoveMaster, lean the user in the opposite direction, hold the upper section of the MoveMaster, and then pull the lower section in the other direction.

Material

Polyamide, polyether

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.