OneWaySlide

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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REF

2074

2075

2076

2077

SWL 2074, 2075, 2076: 150 kg / 330 lbs

SWL 2077: 200kg / 440 lbs

**Direct Healthcare
Group**

Advancing Movement & Health[®]

Table of contents

English	3
Svenska	5
Norsk	7
Danske	9
Suomi	11
Deutsch	13
Netherlands	15
Français	17
Italiano	19
Español	21

Size and safe working load	23
---	----

Symbols	23
----------------------	----

OneWaySlide is used for repositioning farther back and sideways in a chair/wheelchair. OneWaySlide is designed to glide easily backward, but not forward. To make manoeuvring easier for caregivers, OneWaySlide features handles on both sides and at the back. It is placed so that the user remains sitting on it. OneWaySlide is available in several sizes.


Check Safety

Visual inspection


Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load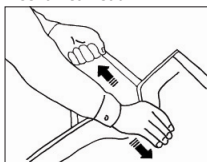

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Friction test

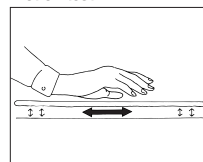Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product. →

High friction - Ensure that the surface is rough and prevents the transfer aid from moving. Compare with a new product. ←

Always read the user manual


Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.


Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.

Placement of OneWaySlide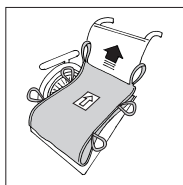



Place OneWaySlide so that the open section is facing the back of the wheelchair. Place the upper section so that it sticks up a bit on the backrest of the wheelchair. This makes it easier for the caregiver to reach the handles.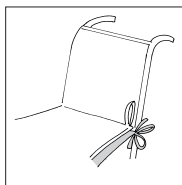


Tie the OneWaySlide to the rear frame tubing of the wheelchair.

Alternative grips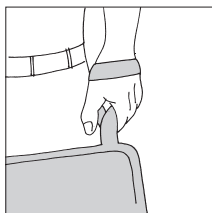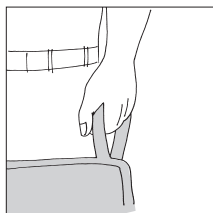


The caregiver can either simply grasp the handle, or first loop it around the wrist. Remember to work with knees bent, back and arms straight, and shift the weight.

Using OneWaySlide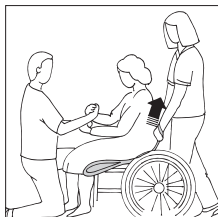


Positioning is easier if the patient can lean forward. If the user is unable to lean forward, one caregiver can assist from the front while supporting the patient's balance.

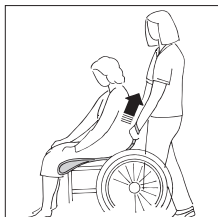If the user can lean forward and has good balance while seated, only one caregiver is required. Grasp the appropriate handles and pull the user back by drawing the OneWaySlide upward along the backrest of the wheelchair.

If the user is heavy or if the chair has a high back, two caregivers can assist, one working on either side. Coordinate your efforts.

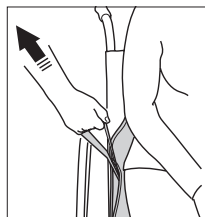A caregiver can easily help make small positioning changes sideways using the handles on each side of the OneWaySlide

! Raising the user's feet from the floor will relieve some of the weight of the thighs from the seat, thereby making repositioning easier.

Material

Polyester, polyamide, cotton

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

**Direct Healthcare
Group**

Advancing Movement & Health®

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.