# OneWaySlide

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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REF

2069

2070

**SWL: 150 kg/ 330 lbs**

## Table of contents

<b>English</b> .....	<b>3</b>
<b>Svenska</b> .....	<b>5</b>
<b>Norsk</b> .....	<b>7</b>
<b>Danske</b> .....	<b>9</b>
<b>Suomi</b> .....	<b>11</b>
<b>Deutsch</b> .....	<b>13</b>
<b>Nederlands</b> .....	<b>15</b>
<b>Français</b> .....	<b>17</b>
<b>Italiano</b> .....	<b>19</b>
<b>Español</b> .....	<b>21</b>
<hr/>	
<b>Size</b> .....	<b>23</b>
<b>Symbols</b> .....	<b>23</b>

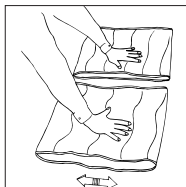
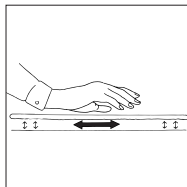
OneWaySlide is used for repositioning farther back in a chair/wheelchair. OneWaySlide is designed to glide easily backward, but not forward. It is placed so that the user remains sitting on it.

## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

High friction - Ensure that the surface is rough and prevents the transfer aid from moving. Compare with a new product.

## Always read the user manual

Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

## Placement of OneWaySlide

Check to ensure that the OneWaySlide lies securely on the seat cushion. A rough material on the top of the seat cushion gives the best result.

Place the OneWaySlide as indicated by the arrow on the product label and allow it to hang about 10 cm/3,9 inch over the front edge of the cushion.

NOTE! Place OneWaySlide according to instructions on product label. Incorrect placement may entail a risk of the patient sliding out of the chair.

## Using OneWaySlide

If the user slides forward in the wheelchair, the caregiver can help them to lean forward and provide counter pressure, thereby enabling the user to slide back in the chair.

Make sure the knees are raised to the same level as the hips. This can be done with the aid of a FootStool.

## Material

Polyester, polyamide, cotton

## Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumbler-drying for optimal service life.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

# Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.