# ReadyBelt

SystemRoMedic™

Manual - English

Manual - Svenska

Manual - Norsk

Manual - Dansk

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
**REF**

16023

16024

16025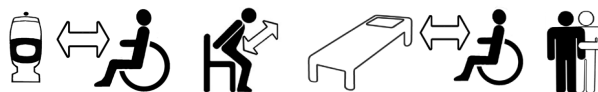

**SWL: 150 kg/330 lbs**

## Table of contents

<b>English</b> .....	<b>3</b>
<b>Svenska</b> .....	<b>5</b>
<b>Norsk</b> .....	<b>7</b>
<b>Dansk</b> .....	<b>9</b>
<b>Deutsch</b> .....	<b>11</b>
<b>Français</b> .....	<b>13</b>
<hr/>	
<b>Size</b> .....	<b>15</b>

ReadyBelt is a belt that is designed to provide the user with support and help during mobility training and different transfer situations. With ReadyBelt, the caregiver can easily and in an ergonomically correct manner assist the user during rising and during transfers, for example, between bed and wheelchair and between wheelchair and toilet or car. Several ergonomically designed handles provide the caregiver with a secure grip, allowing the ideal working stance in every situation. ReadyBelt is not to be used as an assistive device for lifting.

ReadyBelt is available in different sizes and can therefore be used with most users. The belt is placed around the user's waist or hips and secured in place with a clasp which is very easy to open, close and tighten. ReadyBelt is soft and comfortable for the user to wear and a non-slip surface on the inside of the belt prevents it from sliding up.

## Functional inspection


### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Mechanical load

Test the friction of the material by pressing it against the underlying surface and rolling it back and forth (1). Note whether it still glides as easily as it did. Compare with a new product if possible.

Pull on the handles to check seams and strength (2).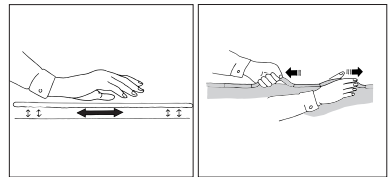


## Always read the manual

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.


## Placement of ReadyBelt

1. Apply the belt around the patient, below the waist but above the hips.
2. Fasten the belt and tighten it.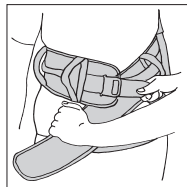

## Removing ReadyBelt

Release the clasp, first loosening any straps, and then remove the belt.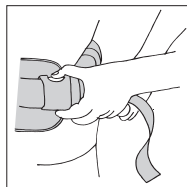


## Using ReadyBelt

Caregivers can work in pairs or alone, depending on the patient's needs, since ReadyBelt has several vertical and horizontal handles. Cross-grips at the back provide support and enable a good working stance (1 and 2).

When two caregivers are assisting, it is appropriate if one caregiver stands on each side, grasping the cross-grips at the back and holding the patient by the hand at the front (3).

ReadyBelt provides good support during transfers between wheelchair and bed (4).

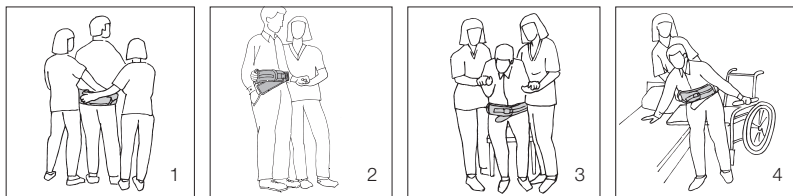## Material

Polyester, Polyethylene, Polypropylene

## Care of the product

Read the product label.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

**CE** Medical Device Class I

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors. Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

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## Simple solutions for great results

SystemRoMedic™ is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

# Direct Healthcare Group

Advancing Movement & Health

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)