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EN - Please read the instructions before using the product.

The instructions apply to seat with forward tilt for Combi chairs for hip patients.

Intended use

The seat with hardware can be installed on the Combi chair self propelled, Combi chair standard, Combi chair low, and Combi chair wide. The seat is installed on the frame, and permits the use of the Combi chair by hip patients or trauma patients who either cannot bend at the hips, or who have reduced hip flexibility after an operation. It can be adjusted to 5 degrees or 10 degrees.

Installation

Remove the standard seat from the Combi chair. Remove the snaplock clips (1) in the backrest, and remove the backrest. Fold up the armrests.

Remove the holders (2) from the seat with hardware by loosening the screws (3) and the disc (4) using an Allen key. Take the positioning bolts (5) out of the notches to remove the holders. Remove the screws (6) and the spacers (7) from the holders. The holders (2) can now be installed on the back of the frame. Place the holders between the armrest and backrest. Replace the spacers (7) into the holders and tighten the screws (6). To install the seat, insert the two rods (8) on the seat through the holes in the holder (2). Install the screws (3) and the disc (4) with the Allen key. Snap the front of the seat (9) tightly down over the frame. Place the backrest and press the snaplock clips (1) back into the frame.

Adjustment and use

When the seat is to be tilted, pull the positioning bolts on both sides out of the notches, so that the seat can be adjusted to the desired angle and height.

Then reinstall the positioning bolts into the notches on both sides. The seat can be adjusted to 3 different heights.

When cleaning, lift the front edge of the seat up 90 degrees, so that the bottom surface can easily be cleaned.

Maintenance

The hardware and seat can be cleaned with common cleaning solutions without abrasives or etching properties. It is not recommended to use solvents or chlorine.

Warning!

- Seats with hardware must only be used as described under the section "Intended use".
- Never stand on the seat - either in the level or angled position.
- Always use footrests on Combi chairs that are outfitted with seat angle hardware for hip patients.
- Check regularly that all knobs and screws are properly tightened.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health[®]

Direct Healthcare Group B.V.
Weteringweg 9,
2641 KM Pijnacker, The Netherlands
Tel: +31 (0)15 369 54 40
Fax: +31 (0)15 369 71 64
www.directhealthcaregroup.com
info.nl@directhealthcaregroup.com

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.directhealthcaregroup.com
info.se@directhealthcaregroup.com