Soft backrest Accessory for Combi chairs Linido

User manual - English
Bruksanvisning - Svenska
Brugsanvisning - Dansk
Brukerhåndbok - Norsk
Käyttöohje - Suomi
Handleiding - Nederlands
Benutzerhandbuch - Deutsch

REF 201130314 / 201131314

MAX: 150 kg/330 lbs

Table of contents

English	5
Svenska	6
Dansk	7
Norsk	8
Suomi	9
Nederlands	10
Deutsch	11

**EN VISUAL INSPECTION**

Inspect the product regularly. Check to ensure that material is free from damage.

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EN BEFORE USE

Ensure that the product is correctly mounted/ installed.

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**EN ALWAYS READ THE USER MANUAL**

Always read the user manual. Keep the user manual where it is easily accessible for users of the product.

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Accessory for Combi chairs

Instructions for use and user

EN - Please read the instructions before using the product. These instructions apply to the soft backrests for Combi chairs. Always consult the manual of the main product for additional product descriptions, intended use specifications, general warnings, precautions and contra-indications and maintenance and service information.

Intended use

The soft backrests fits all Combi chairs with a maximum user weight of 150 kg.

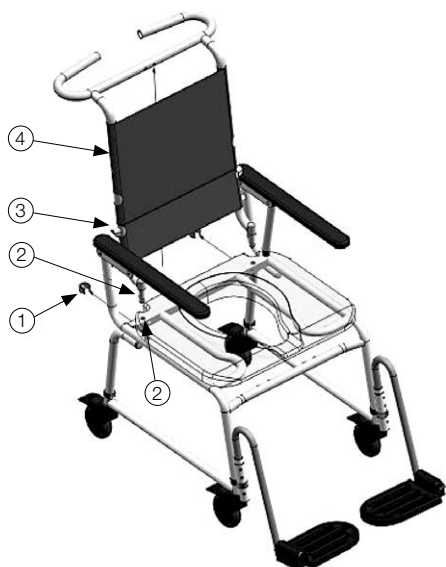
The backrest is available in two variations:

- soft backrest standard size:
part number 201130314
- soft backrest high for Combi tilt chairs:
part number 201131314

The backrest is well suited for users who need a flexible and adjustable back support that can be adjusted to the individual. It is used in place of the standard backrest.

Installation

Remove the existing backrest by removing the clips on both sides (1). Lift up the backrest. Install the soft backrest by placing it down into the holes in the frame (2), securing it over the pins in the frame (3). Put both clips back into the holes in the frame (1).

Adjustment and use

The upholstery can be adjusted to the individual user by loosening or tightening the Velcro straps (4) on the back of the upholstery. Be certain that the Velcro straps hold when the desired adjustment is achieved.

Maintenance

The upholstery on the backrest can be removed and machine washed at up to 80 degrees Celsius. It should not be dried in a tumbler dryer. Remember to stick the Velcro straps together when washing. The backrest can be cleaned with common cleaning solutions without abrasive or etching properties. It is not recommended to use solvents or chlorine.

Warnings

- Maximum weight of user: 150 kg.
- Be certain that the Velcro straps hold after the upholstery has been adjusted.
- Be certain that the upholstery is balanced on the backrest, so that the Velcro can attach equally on both sides.
- Always be certain that the Velcro is sticking well and is tight.

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Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use. See www.dhg-healthcare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.dhg-healthcare.com

DHG

Moving Health Forward

Legal manufacturer

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200

UK Representative

Direct Healthcare Group Ltd.
Whitey Court
Western Industrial Estate
Caerphilly CF83 1BF
United Kingdom

Contact and more info

info@directhealthcaregroup.com
www.dhg-healthcare.com

Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.