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## **EN - Please read the instructions before using the product.**

### ***These instructions apply to the soft backrests for Combi chairs.***

#### **Intended use**

The soft backrests fits all Combi chairs with a maximum user weight of 150 kg.

The backrest is available in two variations:

- soft backrest standard size: part number 201130314
- soft backrest high: part number 201131314

The backrest is well suited for users who need a flexible and adjustable back support that can be adjusted to the individual. It is used in place of the standard backrest.

#### **Installation**

Remove the existing backrest by removing the clips on both sides (1). Lift up the backrest. Install the soft backrest by placing it down into the holes in the frame (2), securing it over the pins in the frame (3). Put both clips back into the holes in the frame (1).

#### **Adjustment and use**

The upholstery can be adjusted to the individual user by loosening or tightening the Velcro straps (4) on the back of the upholstery.

Be certain that the Velcro straps hold when the desired adjustment is achieved.

#### **Maintenance**

The upholstery on the backrest can be removed and machine washed at up to 80 degrees Celsius. It should not be dried in a tumble dryer. Remember to stick the Velcro straps together when washing.

The backrest can be cleaned with common cleaning solutions without abrasive or etching properties. It is not recommended to use solvents or chlorine.

#### **Warning!**

- Maximum weight of user: 150 kg.
- The backrest must only be used as described under the section "Intended use".
- Be certain that the Velcro straps hold after the upholstery has been adjusted.
- Be certain that the upholstery is balanced on the backrest, so that the Velcro can attach equally on both sides.
- Always be certain that the Velcro is sticking well and is tight.

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# Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group B.V.  
Weteringweg 9,  
2641 KM Pijnacker, The Netherlands  
Tel: +31 (0)15 369 54 40  
Fax: +31 (0)15 369 71 64  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)  
[info.nl@directhealthcaregroup.com](mailto:info.nl@directhealthcaregroup.com)

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)  
[info.se@directhealthcaregroup.com](mailto:info.se@directhealthcaregroup.com)