Soft shower seat

Linido

Manual - English

Manual - Svenska

Manual - Norsk

Manual - Dansk

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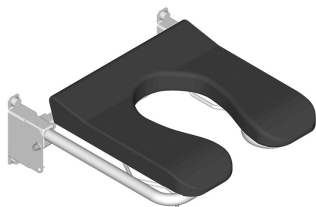
Manual - Español

LI2221.002x-xx


LI2223.021x-xx

Art.no.:

LI2221.002x-xx

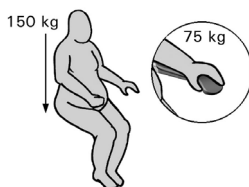LI2223.021x-xx

Max: 150 kg/330 lbs
Max: 75 kg/165 lbs

Direct Healthcare
Group

Advancing Movement & Health®

**EN Visual inspection**

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
Art. no:	Product name:	Description:
LI2806.0000-00	Wall mounting set	For wall mounting of all variants of Soft shower seat without backrest.
LI2805.0000-00	Wall mounting set	For wall mounting of all variants of Soft shower seat with backrest.
LI2880.0006-02	Insulation set M8 x 4	For electrical insulation and covering of fixingpoints, spare part for Soft shower seat without backrest.
LI2881.0006-02	Insulation set M8 x 5	For electrical insulation and covering of fixingpoints, spare part for Soft shower seat with backrest.
LI2214.001x-xx	Floorsupport for Soft shower seat	For all variants of Soft shower seat, for users weighing more than 100 kg.
LI2215.1014-xx	Height adjustment system	For Soft shower seat without backrest.
LI2215.1024-xx	Height adjustment system	For Soft shower seat with backrest.
LI2603.3xxx-xx	Hinged arm support	For side support in combination with Soft shower seat without armrests, for larger users weighing more than 125 kg.
LI02979.430	Bucket	With lid and handle, stainless steel, Ø300mm, for all variants of Soft shower seat.
LI02979.411	Bedpan	With lid and handle, stainless steel, Ø300mm, for all variants of Soft shower seat.

Dimensions/Måttbeskrivning/Målbeskrivelse/Mitat/Maßbeschreibung/Afmetingen/Dimensioni/Dimensiones

A	625 mm	24,6 inch
B	450 mm	17,7 inch
C	Ø6.5 mm	0,2 inch
D	Ø10 mm	0,4 inch
E	425 mm	16,7 inch
F	490 mm	19,2 inch
G	525 mm	20,6 inch
H	20 mm	0,7 inch
I	665 mm	26,1 inch
J	260 mm	10,2 inch
K	415 mm	16,3 inch
L	415 mm	16,3 inch
M	560 mm	22 inch
N	205 mm	8 inch
O	175 mm	6,8 inch
P	140 mm	5,5 inch
Q	165 mm	6,4 inch

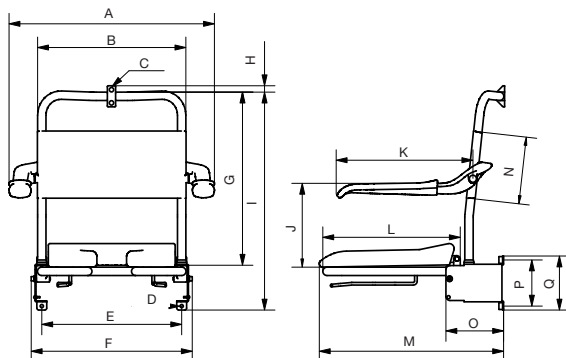Linido

Direct Healthcare Group's Linido product line is a wide range of high quality safety products designed to provide ergonomic support, comfort and increased independence for the user in various bathing, showering and toileting situations.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com

Direct Healthcare Group

Advancing Movement & Health®

Direct Healthcare Group
Weteringweg 9, 2641 KM Pijnacker

Tel: +31 (0)15 369 54 40

Fax: +31 (0)15 369 71 64

www.directhealthcaregroup.com

info.nl@directhealthcaregroup.com