
# TurnTable38


System**RoMedic**

Instructions for use – English  
Bruksanvisning – Svenska  
Brukermanual – Norsk  
Brugsvejledning – Dansk  
Käyttöohje – Suomi  
Gebrauchsanweisung – Deutsch  
Handleiding – Nederlands  
Manuel d'utilisation – Français  
Manuale utente – Italiano  
Manual de usuario – Español  
Instrukcja obsługi – Polski  
Návod k použití – Český  
Návod na použitie – Slovensky  
Instruções de utilização - Português

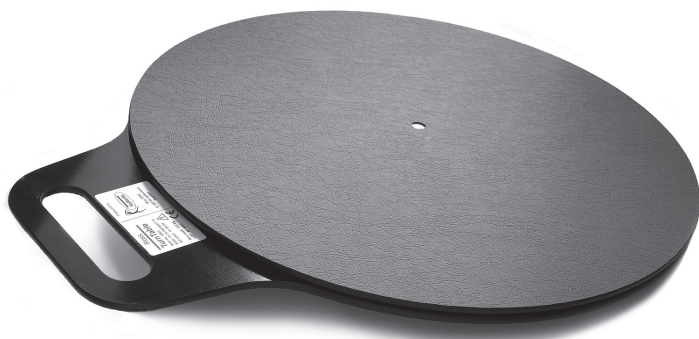**REF**  
5085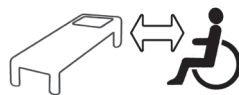

SWL: 135 kg / 300 lbs

## Table of contents

Always read the manual .....	3
English .....	5
Svenska .....	6
Norsk .....	7
Dansk .....	8
Suomi .....	9
Deutsch .....	10
Nederlands .....	11
Français .....	12
Italiano .....	13
Español .....	14
Polski .....	15
Český .....	16
Slovensky .....	17
Português .....	18
Description .....	19
Symbols .....	19

### **EN Always read the user manual**

Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

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The TurnTable can be used for seated transfers between wheelchair and bed, toilet or chair/armchair. For increased security, the TurnTable can be used together with FlexiBelt/EasyBelt. TurnTable is available in three sizes with handles or without handle for tight spaces.

## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

## Special requirements and conditions

### To transfer with TurnTable, the patient must be able to:

- support weight while standing
- stand up and maintain balance (with support)
- understand instructions

## Placement and use TurnTable

Place the TurnTable on the floor beside the user. Secure the TurnTable in place, before raising the user, by placing the ball of your foot on the TurnTable and your heel on the floor. Steer the TurnTable by pushing with the front of the foot and braking with the heel. When the user is about to sit down, secure the TurnTable in place as described above.

When transferring from bed to wheelchair, place the wheelchair close to the bed, so as to minimize transfer distance.

When transferring to the toilet, place the wheelchair close to the toilet, so as to minimize transfer distance. Pull the user's pants up/down when the user has risen slightly from the seat. Preferably, two caregivers should assist.

## Care of the product

Clean TurnTable with pressurized air or water. If necessary, disinfect with 70% alcohol. Teflon spray or a similar product can be used to lubricate between the plates so that they glide easily against each other.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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### Vida útil prevista

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## Description

Art. No	Name	Description
5082	TurnTable	Ø 300 mm/11.8 inch. Without handle. Max 135 kg/297 lbs
5083	TurnTable	Ø 380 mm/14.9 inch. Without handle. Max 135 kg/297 lbs
5084	TurnTable	Ø 450 mm/17.7 inch. With handle. Max 250 kg/551 lbs
5085	TurnTable	Ø 380 mm/14.9 inch. With handle. Max 135 kg/297 lbs

## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

# SystemRoMedic™

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.